


Kingkraft Vacuum Support Cushions **Operating Instructions**

IMPORTANT

Water or moisture must NOT be allowed to enter the air extraction bulb, valve, connecting pipe or the Cushion itself. The ingress of moisture will seriously affect performance of the Cushion and will also invalidate the guarantee.

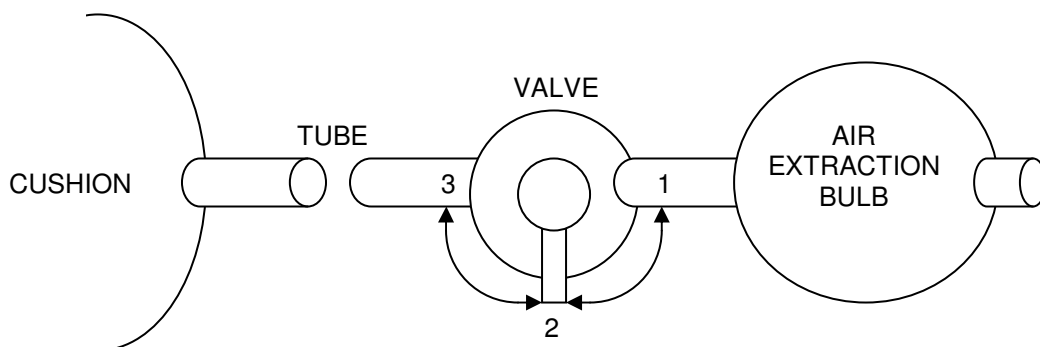

1. Turn valve to position 1 - leave cushion to stand for two minutes to allow air pressure to balance.
2. Check that the surface to which the cushion is to be attached is clean and damp. Fix the Cushion into position ensuring the suckers are firmly secured.
3. Place person or object on the Cushion in the desired position.
4. If necessary, further mould the Cushion into the required shape.
5. Turn the valve to position 2.
6. Operate the air extraction bulb to vacuum-extract air until the correct degree of support is obtained.
7. Turn the valve to position 3 to lock the vacuum.
8. To release the vacuum, turn to position 1. The Cushion can be left in its moulded shape if required.