



# Instructions for use Komfort & Kozy Bather



**PLEASE READ THESE INSTRUCTIONS CAREFULLY BEFORE USE**

# IMPORTANT SAFETY INSTRUCTIONS



## WARNING!

Please read these instructions carefully before first using your Kingkraft product, and then keep them safely for future reference. Please also make sure that each person and any carer who uses the Komfort or Kozy Bather is also aware of their contents and follows the instructions each time they use the unit.

## Komfort & Kozy Bather sizes

Width	480mm
Support Bars	130mm (not available on Kozy)

**Weight limit 120kg (19 stone)**

### Kingkraft Komfort

A 692mm  
B 334mm  
C 340mm



### Kozy Bather

A 504mm  
B 334mm  
C 340mm



# Komfort & Kozy Bather

The Komfort Bather can be used in most standard baths and some Kingkraft baths. The Kozy Bather can be used in all Kingkraft baths and has a smaller height profile to be used in conjunction with our Changing Tops.

## 1.1 Intended Use Statement

The Kingkraft Bathers are intended to be used in Domestic properties and nursing homes by Occupational Therapists, parents, carers and general public.

They are intended to assist in the bathing of children and adults. The person sits in a supported position and allows the carer/parent to wash and interact with ease.

The person must never be left unattended whilst in the Komfort and Kozy Bather at any time.

## 1.2 Intended use of accessories

Only specific Kingkraft accessories are intended to be used on the Komfort and Kozy Bather.

## 1.3 Classification

This product is a Class 1 Medical Device

## Technical information:

- Weight load = 120kg
- Weight of product – 5kg (please ensure your ability to lift the product in and out of the bath)

# Instructions for use



## WARNING!

The Komfort and Kozy Bather are not supplied in a sterile state.

The suitability of the person to use the Bathers should be assessed by a competent person.

Before use the product should be visually checked for damage.

The mesh cover and wedges (if using) should be firmly held in place by the velcro. If the mesh is damaged in any way do not use and contact Kingkraft.

The Komfort Bather should be placed in the bath checking that the rubber arms are securely rested on the sides of the bath. **DO NOT USE IF THE BATH DOES NOT HAVE LEVEL SIDES.** The sides of the bath should also be checked to ensure the rubber arms can rest on a flat surface. Ensure that the Komfort Bather cannot move around more than an inch (25.4mm) either way.

The seat should be placed at the 'head' end of the bath away from the tap. The seat should not be used with the tap behind the users head.

Any use of a lifting hoist to place the person in the Komfort or Kozy Bather should be assessed by a competent person and is designed to be used under normal bathroom conditions. The Kozy Bather is designed to sit in the bath, ensure that the seat is flat in the bath - if the seat rests on the curve of the bath, this could damage the bath



## WARNING!

**Never leave a person unattended in the Komfort Bather.**

The Komfort and Kozy Bather frame should be wiped clean with a soft cloth and soap based product after use, the mesh cover can be removed and machine washed. See cleaning instructions on page 6 for further advice.

PLEASE DO NOT HAVE NAKED FLAMES NEAR THIS PRODUCT

# Accessories



## WARNING!

**The HS25 Head Support is made from PVC!**

Please make sure the user will not have any allergic reactions to the material before use.


## Fitting of the HS25 Head Rest Accessory

(to be used with Komfort Bather only)

The HS25 vacuum support cushion is placed into the mesh cover through the Velcro opening and the vacuum pump is threaded out through the hole. The mesh cover is then positioned to suit the users head and then secured in place by the Velcro strap around the back of the frame.

### IMPORTANT

Water or moisture must NOT be allowed to enter the air extraction bulb, valve, connecting pipe or the Cushion itself. The ingress of moisture will seriously affect performance of the Cushion and will also invalidate the guarantee.

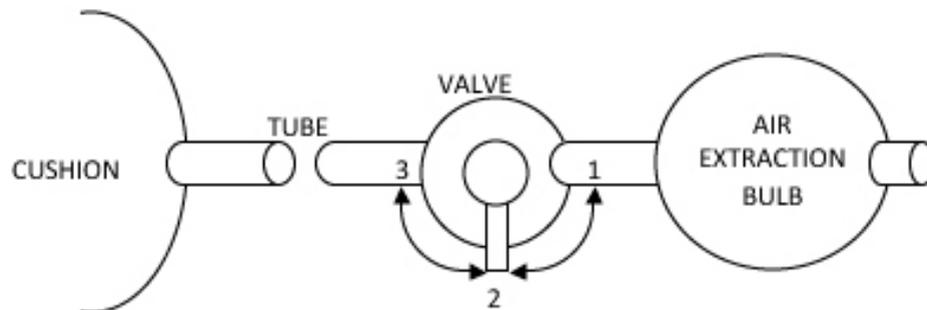

1. Turn valve to position 1 - leave cushion to stand for two minutes to allow air pressure to balance.
2. Check that the surface to which the cushion is to be attached is clean and damp. Fix the Cushion into position ensuring the Velcro are firmly secured.
3. Place person or object on the Cushion in the desired position.
4. If necessary, further mould the Cushion into the required shape.
5. Turn the valve to position 2.
6. Operate the air extraction bulb to vacuum-extract air until the correct degree of support is obtained.
7. Turn the valve to position 3 to lock the vacuum.
8. To release the vacuum, turn to position 1. The Cushion can be left in its moulded shape if required.

## Fitting of the wedges to the Komfort & Kozy Bather

Place the wedges in the desired position and stick to the velcro strap. The wedge is intended as a support and isn't load bearing. It should not be treated as a siderail.

# Safety notes & cleaning instructions – mesh cover



**Please ensure that all users and Carers are aware of these instructions.**

These instructions cover the general use of Kingkraft mesh cover products and should be read in conjunction with the specific instructions for the product in use.



## **WARNING!**

The bathing cushions are made from Polyester.  
Please make sure the user will not have any allergic reactions to this material before use.

To reduce the risk of scalding, always check the water temperature before a person is in the bath, and run the cold tap first and top up with warm water or use a suitable thermostatic mixer valve system. Care must be taken to ensure that if running hot water into the bath, this is mixed thoroughly with the cold in the area of the Komfort and Kozy Bather as well as the rest of the bath.

When placing the seat in the bath, care should be taken to ensure the surface is free from grease or oil or soap deposits. Ensure that the Komfort and Kozy Bather is secured before use.

Be aware that the cover may become detached from the Komfort and Kozy Bather frame in use through a variety of causes such as interference by the user or others, excessive movement, or turbulent water inflow to the bath.

The Komfort and Kozy Bather may move inadvertently if subjected to sufficient force by the user. Extreme care is needed to ensure the that product is correctly specified and used.

**The bather must never be left unattended in the Komfort and Kozy Bather. If in any doubt about their safe use, or the degree of secure support required, do not attempt to use. Please contact Kingkraft immediately for advice.**

Always follow the instructions provided and in the case of any doubt or for further advice please contact Kingkraft Ltd.

# CLEANING INSTRUCTIONS FOR MESH BATH SEAT COVER

Please ensure that all users and carers are aware of the instructions.

Please note that when cleaning the mesh cover, non abrasive materials should be used. Under no circumstances should acid or a bleach based disinfectant be used. Such use will invalidate the guarantee.

Kingkraft recommend the mesh cover should be cleaned regularly - this can be removed and washed at 40°C. .

This product comes with one year manufactures warranty.



Kingkraft Ltd, 26D Orgreave Crescent,  
Dore House Industrial Estate, Sheffield S13 9NQ  
phone 0114 2690697 fax 0114 2695145  
email [info@kingkraft.co.uk](mailto:info@kingkraft.co.uk) web [www.kingkraft.co.uk](http://www.kingkraft.co.uk)



---

**WE PLAN + WE MANUFACTURE + WE INSTALL + WE MAINTAIN**

---