

BODY STRAP AIRWALKER® WASHING INSTRUCTIONS



Machine Wash Separately Cold



Non-Chlorine Bleach as Needed



Tumble Dry Low Remove Promptly



Do Not Iron

⚠️ Remove Rotator and Body Strap Before Washing!

Body Strap Dissassembly



Slide Fleece



Press Buckle Lever




Release Strap



Remove Rotator

Remove Strap

Reassembly Overview
Detailed Steps on Next Page

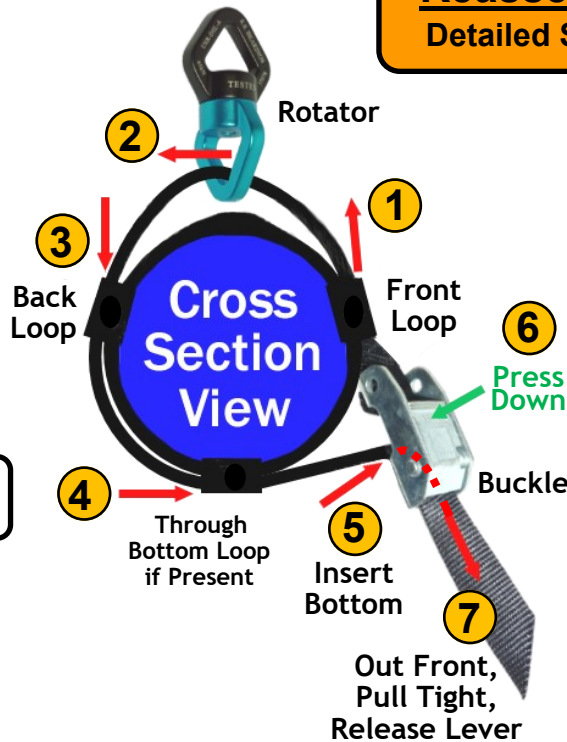

NOTE: Loops are Webbing or Stainless Steel D-Rings



⚠️ WARNING!
Strap Must be secured through bottom of Buckle to Hold Weight



Detailed Body Strap Assembly



**! Loops are Webbing or Stainless Steel
Do Not Hang Airwalker by Loops!**

Flip Over & Pull Tight



Slide Cover

7 Proper Body Strap Buckle Assembly

