


INSTRUCTIONS FOR USE

PRESS 2 - PRESSURE ACTIVATED SAFETY LANCET

The PRESS 2 pressure activated safety lancet is for capillary blood sampling use. It has a pre-loaded needle activated by contact which allows safety for all medical staff and the patient.




STEP 1

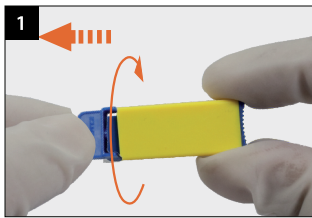

Carefully twist off the cap.
Caution : It is important to completely twist the cap at 360° before removing it.

STEP 2

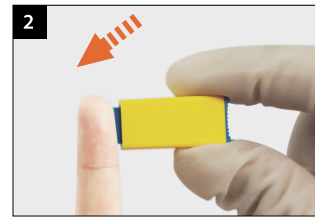

Press the lancet body firmly against the puncture site to activate the device.

STEP 3



Dispose of the used lancet in a suitable sharps container.


MODE D'EMPLOI

PRESS 2 - LANCETTE SÉCURITAIRE À DÉCLENCHEMENT PAR PRESSION

La lancette sécuritaire à déclenchement par pression PRESS 2 est utilisée pour le prélèvement sanguin capillaire. Son aiguille pré-chargée, déclenchée par pression permet d'assurer la sécurité pour tout le personnel ainsi que pour le patient.




ÉTAPE 1

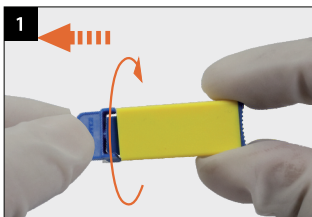

Tourner le capuchon.


Attention : il est important de complètement tourner le capuchon de 360° avant de l'enlever.

ÉTAPE 2

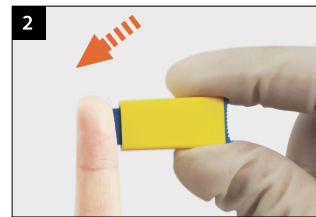

Pour déclencher la lancette, presser fermement le dispositif contre le site de ponction.

ÉTAPE 3

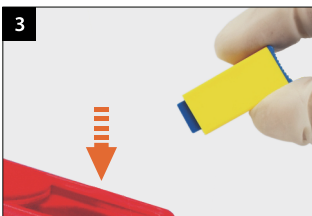

Jeter la lancette souillée dans un récipient pour objets tranchants.