

# INSTRUCTIONS FOR USE


## PRESS 4 - PRESSURE ACTIVATED SAFETY LANCET



The PRESS 4 pressure activated safety lancet is for capillary blood sampling use. It has a pre-loaded needle activated by contact which allows safety for all medical staff and the patient.




### STEP 1

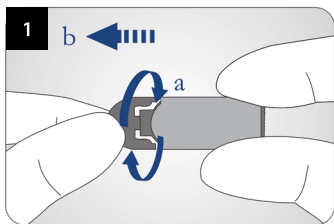


Twist off the cap first and then pull it straight out.

### STEP 2

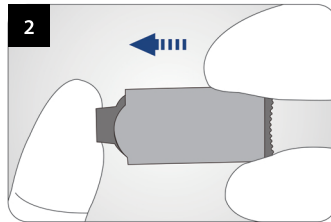

Press the lancet body firmly against the puncture site to activate the device.

### STEP 3

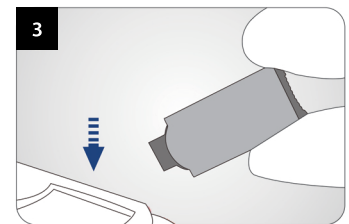

Dispose of the used lancet in a suitable sharps container.

### EASIER TWIST-OFF CAP



Front




Back



*Visual confirmation of used lancet*

Trigger

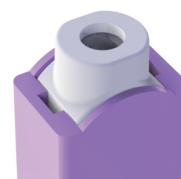

Before use



After use



## MODE D'EMPLOI


### PRESS 4 - LANCETTE SÉCURITAIRE À DÉCLENCHEMENT PAR PRESSION



La lancette sécuritaire à déclenchement par pression PRESS 4 est utilisée pour le prélèvement sanguin capillaire. Son aiguille pré-chargée, déclenchée par pression permet d'assurer la sécurité pour tout le personnel ainsi que pour le patient.




#### ÉTAPE 1

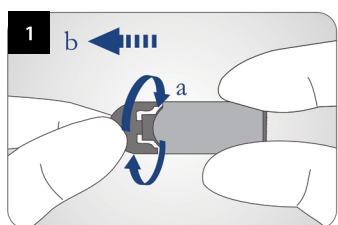


Tourner le capuchon, puis tirer pour le retirer.

#### ÉTAPE 2

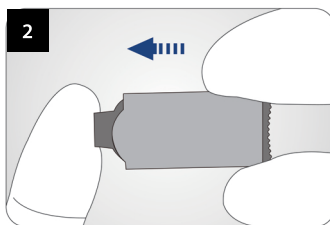


Pour déclencher la lancette, presser fermement le dispositif contre le site de ponction.

#### ÉTAPE 3

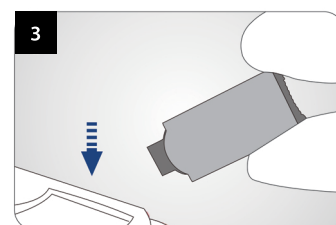

Jeter la lancette souillée dans un récipient pour objets tranchants.

## CAPUCHON PLUS FACILE À TOURNER

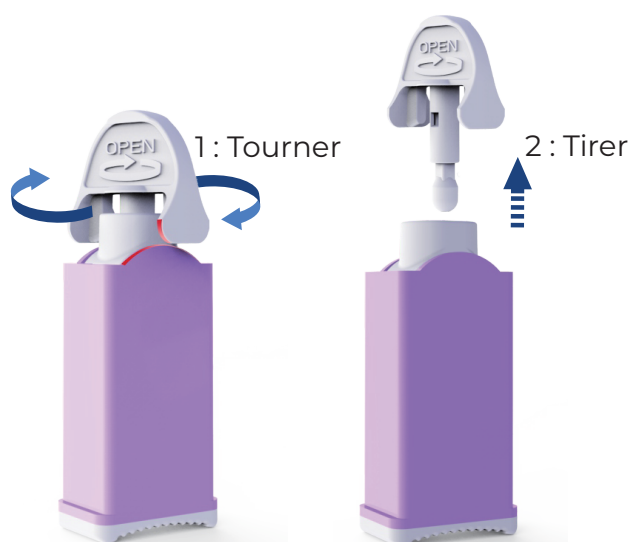

Devant



Derrière



Confirmation visuelle que la lancette a été utilisée

Mécanisme



Avant usage



Après usage

