

INSTRUCTIONS FOR USE


FLEX 3 - DEPTH ADJUSTABLE SAFETY LANCET



With the 3 adjustable depth settings of 1.3mm, 1.8mm and 2.3mm, Flex3 is the answer to the clinical needs of healthcare professionals and also, allows safety for all medical staff and the patient.




STEP 1

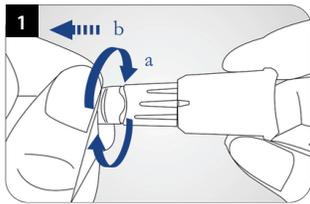

Twist off the cap, then pull it gently.


Caution : It is important to completely twist the cap at 360° before removing it and do not apply force to pull it out.

STEP 2

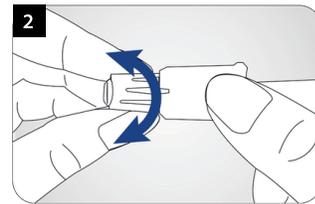


Rotate the adjustable tip (white part) to set the desired depth.

STEP 3

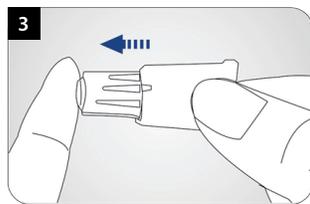

Place the lancet body firmly against the puncture site to activate the device.

STEP 4

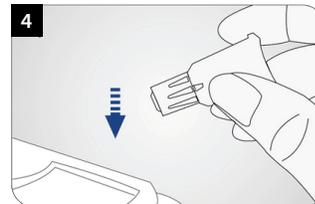

Dispose of the used lancet in a suitable sharps container.



MODE D'EMPLOI


FLEX 3 - LANCETTE SÉCURITAIRE À PÉNÉTRATION AJUSTABLE



Avec ses 3 niveaux de profondeur réglables, soit 1.3mm, 1.8mm et 2.3mm, Flex3 permet de répondre à l'ensemble des besoins cliniques des professionnels de la santé en plus d'assurer la sécurité pour tout le personnel ainsi que pour le patient.




ÉTAPE 1

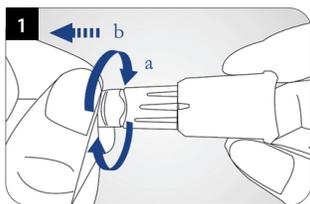

Tourner le capuchon, puis tirer doucement pour le retirer.


Attention : il est important de complètement tourner le capuchon de 360° avant de l'enlever et de ne pas appliquer de force en le retirant.

ÉTAPE 2

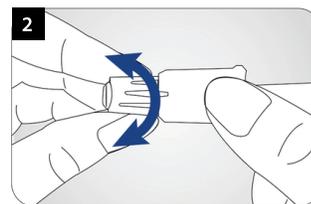


Tourner l'extrémité blanche afin de régler la profondeur désirée.

ÉTAPE 3

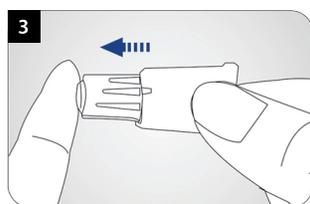

Pour déclencher la lancette, presser fermement le dispositif contre le site de ponction.

ÉTAPE 4

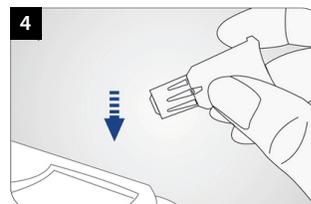

Jeter la lancette souillée dans un récipient pour objets tranchants.

