INSTRUCTIONS FOR USE


DOMREX™ SAFETY BLOOD COLLECTION SET

The DOMREX™ Safety Blood Collection Set is intended to be used for insertion into a patient's vascular system for blood collection or short term IV administration (up to 2 hours) under the direct supervision of a trained medical professional. It helps in the prevention of accidental needlestick injury through the use of an active safety feature.

FOR BLOOD COLLECTION


1. Verify that the packaging is not damaged. Open the device packaging.
2. Make sure the Female Luer Adapter is securely attached to the Male Luer Adapter. Remove the protective tube piercing needle cap and securely attach the Male Luer Adapter on the holder.
3. Remove the needle cap.
4. Perform venipuncture by holding the wings.
5. Withdrawing the DOMREX™ Safety Blood Collection Set
 - A. Apply pressure to site using your fingers as shown.
 - B. Grasp the translucent yellow safety shield grip area with the thumb and index finger while grasping the tubing at the same time.
6. Safety shield activation: **One-hand activation:** Hold tubing in hand and advance translucent yellow safety shield with the thumb and index finger until a "click" is heard or a falling indicating that the safety mechanism is locked in place over needle tip.
7. Dispose of all used materials in an appropriate container and wash hands.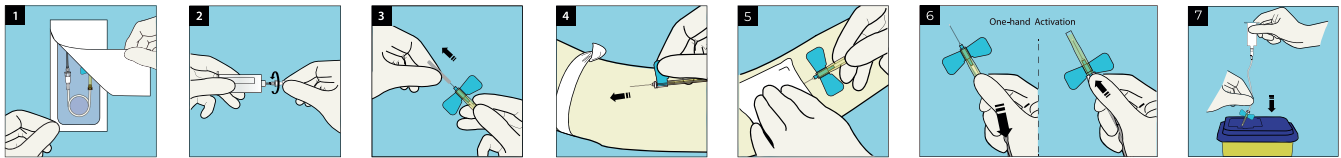

FOR BLOOD COLLECTION USING A SYRINGE

1. Remove Luer Adapter and attach syringe.
2. Follow blood collection steps 3-7.

NOTE: Use the transfer device to transfer blood from the syringe to the blood collection tube.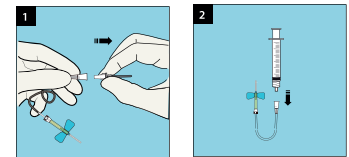


FOR SHORT TERM IV ADMINISTRATION (up to 2 hours)

1. Remove Luer Adapter and attach IV line or syringe to the Female Luer Adapter.
2. Prime set in accordance to the institution's procedure.
3. Follow blood collection steps 3-7.

Administer medication. It is recommended that this device be changed per the institution's procedures, but no longer than 2 hours.

NOTE: This device is for use in a professional healthcare facility, the home or a transport (ambulatory) environment and should always be used under the direct supervision of a trained medical professional.

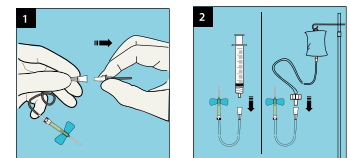CAUTION:

- After activating the safety mechanism, promptly dispose of the product in an approved disposal container.
- While the safety mechanism is being activated, keep hands away from the needle tip at all times.
- There is a possibility of blood and/or other fluids leaking out of the product when removing the Luer Adapter. Use precaution.
- Use the device immediately after opening the blister package.
- Do not use if the packaging is damaged.
- Store in a cool, dry environment, away from direct sunlight, high heat and high humidity.
- Be careful when using and when disposing of the device to avoid accidental needlestick injuries.
- The product is for single use only. DO NOT REUSE.

Non-Toxic / Non-Pyrogenic / Sterile if unopened and/or undamaged / Single Use Only
Not made with DEHP / Not made with natural rubber latex

WARNING

- Do not forcefully pull on the wing and/or perform any related action.
- Rough handling of the device may cause damage to the device.
- Do not forcefully release and/or destroy the safety mechanism/safety shield after it has been activated.

UNIVERSAL PRECAUTIONS:

Handle all biological samples and blood collection sharps such as lancets, needles, luer adapters and blood collection sets in accordance to the policies and procedures of your facility. Immediately seek appropriate medical attention in the event of any exposure as body fluids may transmit viral hepatitis, HIV (AIDS) and/or other blood borne pathogens. Utilize built-in used needle protector if the blood collection device provides one. Discard all blood collection sharps in biohazard containers approved for their disposal.

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