

PRESS 4


PRESSURE ACTIVATED SAFETY LANCET



The PRESS 4 pressure activated safety lancet is for capillary blood sampling use. It has a pre-loaded needle activated by contact which allows safety for all medical staff and the patient.




STEP 1

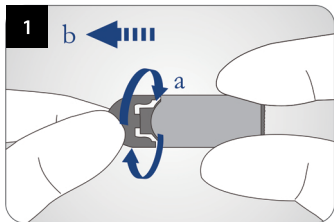


Twist off the cap first and then pull it straight out.

STEP 2

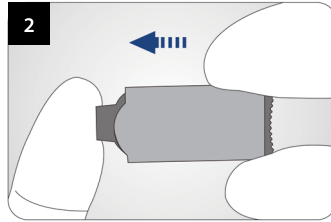

Press the lancet body firmly against the puncture site to activate the device.

STEP 1

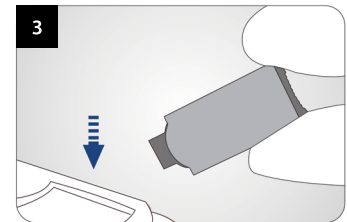

Dispose of the used lancet in a suitable sharps container.

EASIER TWIST-OFF CAP



Front

Back



Visual confirmation of used lancet

Trigger

Before use

After use

