

# DOMREX™

## SAFETY NEEDLES AND SYRINGES



### STEP 1



To open the package, locate the end marked **PEEL HERE**. Separate the heat-sealed edges by spreading the tabs with the thumb and index finger of each hand.


*Never push on the syringe to tear the packaging.*

### STEP 2



Once opened, ensure that the capped needle is securely anchored on the syringe which may become loose during transport.

### STEP 3

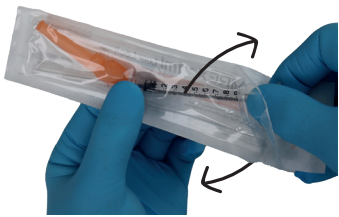

**Grasp the base** of the capped needle with one hand and the other end of the syringe with the other hand. **Push lightly** on the syringe and screw in by turning the syringe clockwise.

**Caution:** DO NOT PUSH OR SCREW TOO HARD as this may affect the syringe-needle junction.

### STEP 4



**Move the safety mechanism away** from the needle and toward the syringe before removing the needle cap. Perform the injection using established technique.

For user convenience, the needle position (bevel up) is oriented to the position of the safety mechanism.

### STEP 5

#### ACTIVATION WITH THE THUMB




OR

#### ACTIVATION ON HARD SURFACE



Once the injection is complete, use one of the 2 one-handed techniques presented above to activate the safety mechanism which has **3 anchor points**. Activation is confirmed by an **audible "Click"**.

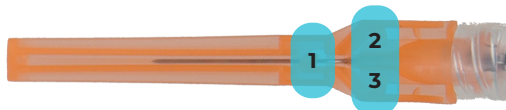

### STEP 6

Dispose of the device in a bin provided for this purpose such as DOMREX™ Sharps Container.

