

Magnetic Urine Bottle Upgrade Instructions





This guide will instruct you on the correct installation steps to modify your Cuddy or Cuddy Lite to use a magnetic urine bottle.

The latest version of these instructions is available by scanning the QR Code



You will also need a #2 Phillips Screwdriver.

Inside the Kit you will find:


2x screws 	1x Magnetic Urine Bottle 	1x Reed Switch 	1x Switch Housing 
--	---	--	--

Remove the Spring Contacts



Prepare the Reed Switch

Push the reed switch inside the housing and prepare the wires for mounting by placing them over the screws.

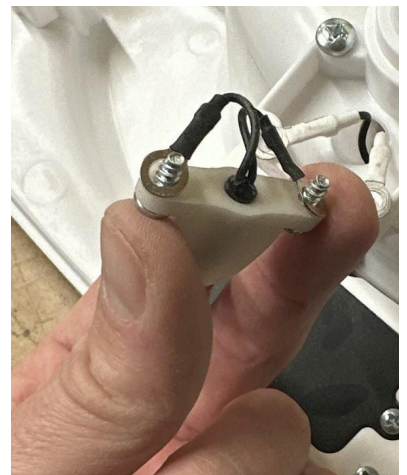

Attach the reed switch circuit board

Push the reed switch inside the housing and prepare the wires for mounting by placing them over the screws.



Check it works!

Invert the bottle to raise the float and hold the top of the bottle near the sensor to turn on the LED.

