
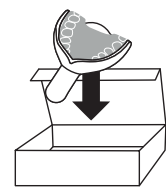
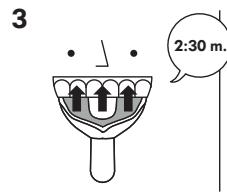
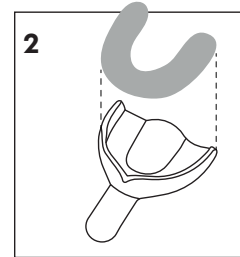
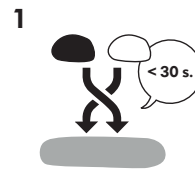


## MOULDING INSTRUCTIONS (ENG)

Please read the following instructions carefully if you want your mould to be useful. Read them all through before you start and take the mould as fast as possible, because the putty hardens quickly.

- Brush your teeth & put on gloves
- Mix 1x colored + 1x white impression putty for max. 30 sec. until solid color (no white spots)
- Roll quickly into a ball and then into a thick cylinder shape, without creating folds. Do this step fast
- Put the mixed putty softly into the tray, make sure it's not folded anywhere (!). Do this step fast
- Push the tray firmly onto your teeth (with your thumbs) in one move. Make sure your teeth are fully covered with putty, don't scrape the tray with your teeth. DON'T wiggle or move the tray until putty is fully hardened.
- Leave in for 2,5 minutes (02:30). Don't bite on the tray, just hold the front of the tray with one hand.
- Remove by pulling it down gently, don't wiggle it left to right – only softly front to back if necessary!
- Still struggling? Watch a YouTube video.

### DENTÅLIMPRES

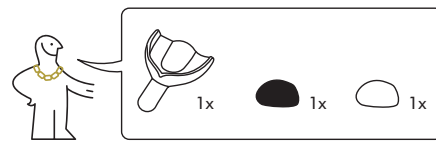

FOR THE VISUAL LEARNERS ;)

GOOD



All teeth are well defined and the gum lines are visible. Your impression should look something like this! :)

NOT GOOD



Client didn't push the tray up onto their teeth in one smooth move. It's moved too much. You can see the putty is cracked and the teeth and gum lines aren't defined enough. You can also see the (white) tray at the front, because the teeth were scraped onto it. Avoid this.