ACART



Assembly Instructions

Medium Duty Hamper with Foot Pedal



Model #HS-50 (692) - Assembly Instructions

TOOLS INCLUDED:

- (2) 7/16 wrenches
- (2) 9/16 wrenches





1 person assembly



1

Install casters to base legs. Place one washer under base tube and one above. Slide smaller wrench between base and caster to hold in place, then use larger wrench to tighten acorn nut on top until secure.

Note: Be sure to install locking casters at front of base, where base tubes end with a black cap



2

Install uprights by inserting vertical male poles into female base tubes.



WARNING

- When moving hamper, always proceed slowly
- Secure bag before use
- Before using foot pedal, confirm that brake is engaged and lid is not obstructed
- Make sure casters roll easily and are free of debris







Model #HS-50 (692) - Assembly Instructions

3

Install lid by placing female tubes onto vertical male poles.



4

Install pedal wires by inserting one curved end of wire through middle hole in foot pedal frame. Insert other end of wire into hole at back of lid. Be sure pedal wire ends are facing inward. Repeat on other side of hamper.

Note: Lid height when opened is adjustable via the different holes in foot pedal frame.



Your Medium Duty Hamper is now ready for use.





