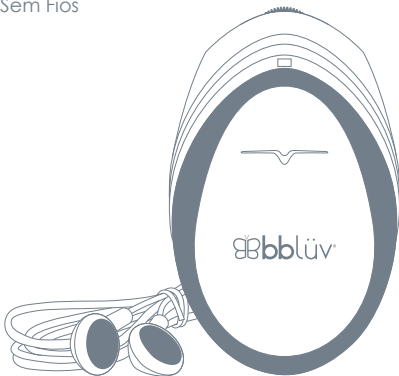
# Echö

- EN Wireless Fetal Doppler
- FR Moniteur foetal sans fil
- ES Doppler fetal inalámbrico
- IT Doppler fetale wireless
- DE Kabelloser Baby-Doppler
- PT Doppler Fetal Sem Fios

B0113 / JPD-100smini - Instruction manual v.5

*Please read the instruction manual carefully before use.*

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
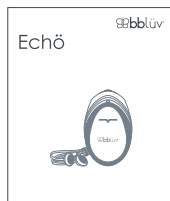
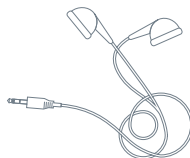
Shenzhen Jumper Medical Equipment Co., Ltd. D Building, No. 71, Xintian Road,  
Fuyong Street Baoan, Shenzhen Guangdong 518103, CN

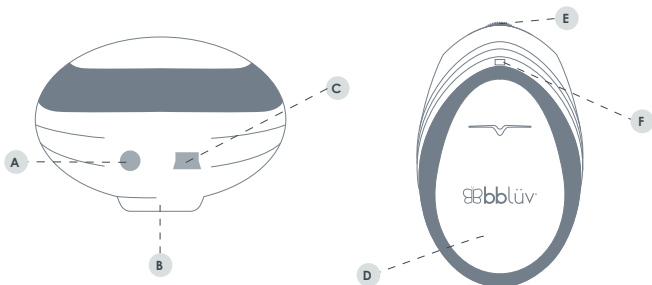MedPath GmbH Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany

**A**

Headphone jack (Audio out)  
Prise casque (sortie audio)  
Toma para auriculares (salida de audio)  
Jack per cuffie (uscita audio)  
Kopfhöreranschluss (Audioausgang)  
Tomada de auscultadores (Saída de áudio)

**B**

Transducer Probe (Baby Heartbeat receiver)  
Sonde du transducteur (récepteur de battements cardiaques du bébé)  
Sonda transductora (receptora de los latidos del bebé)  
Sonda trasduttore (Ricevitore battito cardiaco del bambino)  
Schallkopfsende (Empfänger für den Herzschlag des Babys)  
Sonda transdutor (Recetor do batimento cardiaco do bebé)

**C**

Mini-USB USB jack (Recording)  
Mini-prise USB (enregistrement)  
Minifoma USB (grabar)  
Jack USB mini-USB (registrazione)  
Mini-USB-Buchse (Aufnahme)  
Tomada Mini-USB (Gravação)

**D**

Battery Cover  
Couvercle du boîtier des piles  
Tapa de la batería  
Coperchio della batteria  
Batteriefachabdeckung  
Tampa da bateria

**E**

On/Off/Volume control  
Marche/Arrêt/Contrôle du volume  
Control de volumen/encendido/apagado  
Manopola acceso/spento/controllo volume  
Ein/Aus-Lautstärkeregler  
Ligar/Desligar/Controlo de volume

**F**

Power On Indicator  
Indicateur de mise sous tension  
Indicador de encendido  
Indicatore di accensione  
Netzkontrollleuchte  
Indicador de alimentação

**G**

In compliance with the essential requirements of European Medical Device Directive 0482.  
Conforme aux exigences essentielles européennes de la directive 0482 sur les dispositifs médicaux.  
Cumple con los requisitos europeos esenciales de la Directiva 0482 de Dispositivos médicos.  
Conforme ai requisiti essenziali europei della Direttiva sui dispositivi medici 0482.  
In Übereinstimmung mit den wesentlichen Anforderungen der europäischen Richtlinie 0482 für Medizingeräte.  
Em conformidade com os requisitos essenciais europeus da Diretiva de Dispositivos Médicos 0482.

**H**

Do not dispose of this product with ordinary household waste.  
Ne jetez pas ce produit avec les ordures ménagères ordinaires.  
No elimine este producto con los desperdicios habituales del hogar.  
Non smaltire il prodotto insieme agli altri rifiuti domestici.  
Entsorgen Sie dieses Produkt nicht im normalen Hausmüll.  
Não elimine este produto junto com resíduos domésticos normais.

**I**

IEC 60417-5333, Type BF applied part with IEC 60417-5031, Direct current.  
IEC 60417-5333, Pièce appliquée de type B avec IEC 60417-5031, Courant continu  
IEC 60417-5333, parte de contacto tipo BF con IEC 60417-5031, corriente directa.  
IEC 60417-5333, Parte applicata di tipo BF con IEC 60417-5031, Corrente continua.  
IEC 60417-5333, Angewandtes Teil Typ BF mit IEC 60417-5031, Gleichstrom.  
IEC 60417-5333, peça aplicada Tipo BF com IEC 60417-5031, Corrente contínua.

**J**

**WARNING** - Refer to instruction manual  
**AVERTISSEMENT** - Reportez-vous au manuel d'instructions  
**ADVERTENCIA** - Consulte el manual de instrucciones  
**AVVERTENZA** - Consultare il manuale di istruzioni  
**WARTUNG** - Siehe Gebrauchsanweisung  
**AVISO** - Consultar o manual de instruções

**K**

Protected from the touch of fingers and objects greater than 12 millimeters.  
Protégé du toucher des doigts et des objets de plus de 12 millimètres.  
Protegido del toque de dedos y objetos superiores a 12 milímetros.  
Protetto dal tocco di dita e oggetti superiori a 12 millimetri.  
Geschützt vor Berührung mit den Fingern und Gegenständen größer als 12 Millimeter.  
Não tocar com os dedos ou objetos maiores que 12 milímetros.

# Echö

|    |                |    |
|----|----------------|----|
| EN | English .....  | 6  |
| FR | Français.....  | 17 |
| ES | Español .....  | 28 |
| IT | Italiano ..... | 39 |
| DE | Deutsch.....   | 50 |
| PT | Português..... | 61 |

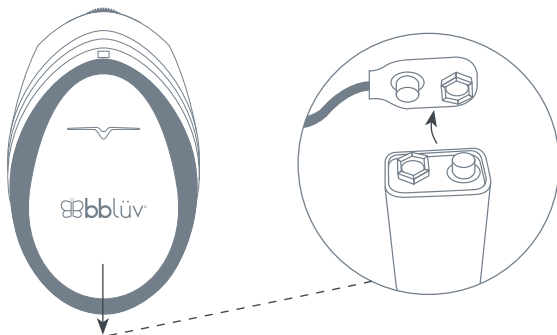
## BEFORE USE

The Echö is a portable lightweight heartbeat monitor. It is mainly used to detect the sound of the fetal heartbeat (SFH). Always consult your physician before using the Echö. Do not use for diagnosis purposes.

## INSTRUCTIONS

### INSTALLING THE BATTERY

Open the battery cover **D**, and install a 9V battery (not included) into the battery compartment. Put the battery cover back on.


**NOTE:** When not in use, remove the battery from the Echö and store it in a cool and dry environment. This will protect both the monitor and the battery.


## CONNECTING THE EARPHONE

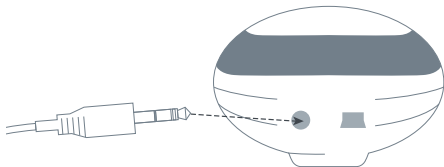It is possible for one or two people to listen to the baby's heartbeat at the same time.

**Single listener** - connect one earphone set to the headphone jack **A**.

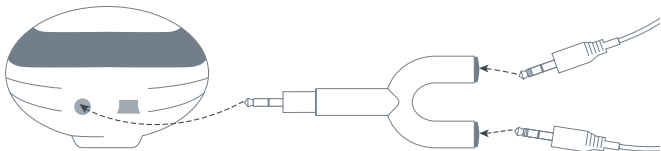**Double listener** - connect the splitter, to the headphone jack **A** first, and then connect each earphone set **A** to one of the splitter's output jacks (*Second earphone set not included*).

**NOTE:** You can also use a personal speaker using a standard 3.5 mm stereo audio jack.

## PREPARING THE SURFACE

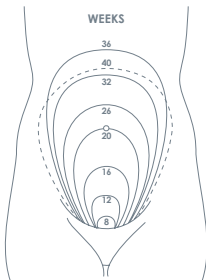
In a sitting position or laying down, remove any clothing on your belly to be directly on the skin.

1. You can use the Echö on dry skin or, for a better experience, you may use some Ultrasound Transmission Gel or a personal lubricating gel which reduces the static caused by moving the monitor around on the belly. Baby oil, coconut oil, virgin olive oil or petroleum jelly (e.g. Vaseline) also work well.

**NOTE:** We do not recommend to use of colored gels or lotions. Caution: Be sure not to get gel into the area around the On/Off/Volume knob as this could damage the Echö.

## STARTING THE ECHÖ AND LISTENING TO BABY'S HEARTBEAT

1. Power On the Echö with the On/Off/Volume control **E**. The Power indicator light **F** will turn on to indicate the Echö is on.
2. Hold the monitor to your heart first to test it. (*Your heartbeat is easier to distinguish compared to the baby's heartbeat*). Refer to the Troubleshooting section if you are not sure it is working.
3. After checking that the Echö is working properly, hold it to your abdomen to listen to your baby's heartbeat. The baby's heart sounds like a galloping horse at approximately twice the maternal rate. A wind-like sound will be heard from the placenta. Between 8 to 16 gestational weeks it is possible to hear your baby's heartbeat if conditions are optimal but you may not be able to detect it that early on. After the 16th gestational week you should have no problem finding the heartbeat. As the baby grows, the heartbeat becomes stronger and it will be easier for you to detect it. Look at the gestational chart below where you should be able to find your baby's heartbeat as it grows each week.

**NOTE:** Keep in mind that each mother and baby is different so not being able to detect the heartbeat early is not a reason for concern. If you are not able to detect the baby within 10 minutes, it is recommended to take a break and to try again in the next few days. However, if you are concerned about any issues relating to your pregnancy you should contact your healthcare provider for assistance.


4. Move the Echö very slowly to find the optimum placement based on the baby's current position.
5. After the heartbeat is detected, adjust the volume **E** to a comfortable level and enjoy.
6. After you are done listening, turn off **F** the power and gently wipe the probe **B** clean.

**NOTE:** We recommend keeping the session to less than 10 minutes at a time. While the Echö is considered safe for "continuous use" by regulatory bodies including the Food and Drug Administration (FDA), we recommend limiting use in short intervals, infrequently and primarily after the first trimester. The Echö is NOT intended to be used for diagnostic or medical purposes and is not to be used as a substitute for regular prenatal care by a licensed doctor. Contact your healthcare provider for assistance if you are concerned.

## RECORDING

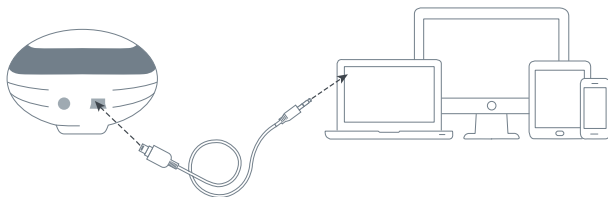- Connect the Computer Recording Cable to the Echö using the USB Mini plug **C** and connect the other end to a recording device. Recording devices could be your computer using the microphone jack or any external recording equipment using the audio in jack. You can use a basic free recording software that usually comes with your computer or available online.
- Another way to get a recording is to connect the Echö to an external audio speaker and then use your smart phone or tablet to record a live video. Then you can share the video with your loved ones right from your phone.
- You may also record the mother's heartbeat at the same time and keep this recording to play it to soothe your newborn baby at daycare and at home.

## MAINTENANCE

The transducer is fragile and must be handled with care. Please check that the Echö does not have visible signs of damage that may affect user safety or monitoring capability before use.

- Carefully inspect the equipment for physical damage.
- Examine the case, connectors and accessories making sure every part is clean and is not cracked or damaged in any way.
- After the visual inspection, if the Echö and/or its accessories are damaged, replacement is needed.
- Before cleaning, turn off the Echö **E**, disconnect the earphones **A** and the recording cable **C** and remove the battery.
- If gel is used, clean it from the transducer probe **B** with a soft cloth. You can disinfect the probe by wiping it with 70% ethanol or alcohol wipes and then drying with a soft dry cloth.
- Do not use abrasives, solvents, bleach, steam or high temperature cleaning processes to sterilize the Echö as any of these can damage it.
- Do not immerse in water or other liquids to clean the Echö.
- Do not disassemble the Echö.
- Checking the battery level

**NOTE** - The Echö requires no calibration.

## TROUBLESHOOTING

What appears to be a malfunction may not always be serious. If your Echö does not perform as expected, consult the table below to see if the problems can be corrected.

If you are unable to resolve the problem, contact bblüv Customer Service at **1-855-808-2636** or by email at **info@bbluvgroup.com**

**NOTE:** Make sure to always test the Echö with your own heartbeat first to make sure it is working correctly

| PROBLEM           | CAUSE                             | SOLUTION                                    |
|-------------------|-----------------------------------|---------------------------------------------|
| Audio scream      | Audio volume too high             | Turn down the volume                        |
|                   | Too much gel on the probe surface | Use less gel                                |
|                   | Battery is low                    | Replace the battery                         |
| Weak sound output | Audio volume too low              | Turn up the volume                          |
|                   | Insufficient gel                  | Add gel                                     |
|                   | Battery is low                    | Replace the battery                         |
| Low sensitivity   | Incorrect probe position          | Move the probe to find the correct position |
|                   | Insufficient gel                  | Add gel                                     |

## FAQ

### CONFIGURATION ISSUES

#### Q: The Echö doesn't turn on

**A:** Install a new battery and make sure the On/Off/ Volume **E** knob is turned on. Verify that the battery connector is connected fully onto the battery terminals. Make sure you've removed the clear protective wrapper off of the battery.

#### Q: The power indicator LED **F** is weak

**A:** Install a new battery.

#### Q: There is no sound coming through the earphones

**A:** Make sure the power indicator LED **F** is on. Check that the earphone wire is pushed all the way into the headset jack **A** of the Echö. Check all the wires to be sure they are in good condition. Be sure to turn up the volume using the On/Off/Volume **E** knob. If it is still not working, install a new battery.

**Q: Uncomfortable noise is coming through the earphones**

**A:** Interference can be caused by nearby electronic devices (e.g. high frequency electronics, cell phones, cordless phones, etc.). Also, when you can hear some slight noises occur when moving the probe from one location to another, you might need to apply some more gel. You can also install a new battery, a weak battery can cause noise.

**Q: How soon can I hear the baby?**

**A:** We recommend to use the Echo after the 16th week of pregnancy. Before this time, there are a lot of variables which may not let you hear your baby yet, such as the position of your placenta. Keep in mind that the baby is still tiny before 16 weeks of gestation. The Echö works more like a laser pointer than a floodlight in that you will hear the baby when it is perfectly lined up. If you are off slightly you'll miss the baby's heart and won't hear anything. As the baby grows every week, you will be able to find the baby's heart rate more easily.

**Q: I'm finding it difficult to find baby's heartbeat after 16 weeks**

**A:** First check to see if you can hear your own heartbeat to be sure the Echö is working correctly. If you can not hear your own heartbeat, you should troubleshoot before attempting to locate the baby's heartbeat. Move the probe very slowly around your abdomen to make it easier to find the optimum placement to hear your baby's heartbeat. The baby can sometimes be located behind the placenta making it difficult to find. You might not have found the optimum placement of the probe to hear the baby's heartbeat, take a break for a day and try again. Don't apply too much pressure to your abdomen.

**Q: The battery cover doesn't stay closed**

**A:** When putting the battery back into the Echö, run the wires down the right side between the battery holder and the inside wall with the clip end of the battery towards the On/Off/Volume **E** knob. This allows the battery to sit lower in the compartment, so the battery cover **D** will close easier and more completely.

**Q: How do I record the heartbeat?**

**A:** Please review the instructions in RECORDING section.

**WARNINGS**

The following is a list of product safety alerts. You must read, understand and heed these safety alerts before attempting to operate the Echö.

- **Fire and explosion hazard:** do not operate the Echö in the presence of flammable gases to avoid possible explosion or fire hazard.
- **Practice the ALARA principle:** We recommend that exposure to ultrasound is kept as low as reasonably achievable principle. This is considered to be good practice and should be observed at all time.
- **Temperature/Humidity/Pressure extremes:** Exposing the Echö to extreme environmental conditions outside of its operating parameters may compromise the ability of the product to function properly.
- **Battery disposal:** Recycle or dispose of the battery in accordance with all federal, state and local laws. To avoid fire and explosion hazard, do not burn or incinerate the battery.
- **Possible Radio Frequency (RF) susceptibility:** radio frequency from cellular phones, CB radios and FM 2-way radio may cause interference with the Echö.

**INTENDED USE**

The Echö is intended to detect the fetal heartbeat rate. You can also use it to listen to the fetal heartbeat.

## TECHNICAL SPECIFICATIONS

| ECHÖ   |                                                                     |
|--------|---------------------------------------------------------------------|
| Model  | B0113                                                               |
| Safety | Complies with IEC 60601-1, EN 60601-1, EN60601-1-2, IEC 61266 norms |

| CLASSIFICATION                                                |                                                                     |
|---------------------------------------------------------------|---------------------------------------------------------------------|
| Anti-electric shock type                                      | Internal powered equipment                                          |
| Anti-electric shock degree                                    | Complies with IEC 60601-1, EN 60601-1, EN60601-1-2, IEC 61266 norms |
| Classification of protection against harmful ingress of water | Ordinary protection IPX4 (probe)                                    |
| Methods of sterilization or disinfection                      | None required                                                       |
| Degree of safety in presence of flammable gases               | Equipment not suitable for use in presence of flammable gases.      |
| Mode of operation                                             | Continuous operation                                                |
| EMC                                                           | Group   Class B                                                     |

| TECHNICAL PARAMETERS                                       |                                                                                                                           |
|------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------|
| Ultrasound: Ultrasonic emitting frequency                  | 3MHz<br>Overall sensitivity at the distances 8 in. from the face of the probe (Doppler frequency: $500 \pm 50\text{Hz}$ ) |
| Target velocity                                            | 4 in./s ~ 16 in./s: $\geq 90\text{d}$                                                                                     |
| Spatial-peak temporal-peak acoustic pressure               | $\leq 1\text{MPa}$                                                                                                        |
| Output power                                               | $< 20\text{mW}$                                                                                                           |
| Effective area of the ultrasonic transducer active element | 1 sq in. $\pm 0.1$ sq in.                                                                                                 |
| Acoustic coupling medium for normal use                    | ph: 5.5~ 8                                                                                                                |
| Acoustic impedance                                         | $\leq 1.7 \times 10^5 \text{g} / \text{cm}^2\cdot\text{s}$                                                                |
| Working mode                                               | Continuous wave Doppler<br>Audio Output Power : $< 0.5 \text{ W}$                                                         |
| Audio Out Socket                                           | $\Phi 0.14$ in./USB interface                                                                                             |
| Recommended battery type                                   | 9V DC alkaline battery (IEC Type No. 6F22 or equivalent)                                                                  |
| Stand-by time                                              | $> 4$ hours                                                                                                               |
| Recommended gestation for use                              | 16 gestational weeks or later, depending on pregnancy                                                                     |

| PHYSICAL CHARACTERISTIC |                                                      |
|-------------------------|------------------------------------------------------|
| Size                    | 0.04 in. (Length) * 2 in. (Width) * 2.8 in. (Height) |
| Weight                  | 2.5 oz (only main unit, not including battery)       |

#### ENVIRONMENTAL REQUIREMENTS:

| OPERATING CONDITIONS : |                              |
|------------------------|------------------------------|
| Temperature            | 41°F to 104°F (50°C to 40°C) |
| Humidity               | 0 to 80% RH, non-condensing  |
| Atmospheric pressure   | 86kPa to 106kPa              |

| STORAGE AND SHIPPING CONDITIONS |                               |
|---------------------------------|-------------------------------|
| Temperature                     | -4°F to 131°F (-20°C to 55°C) |
| Humidity                        | 10 to 93% RH, non-condensing  |
| Atmospheric pressure            | 50kPa to 106kPa               |

#### LEGAL NOTE

The Echö is FDA approved, compliant with the ISO 80601-2-56 and EN60601-1-2:2007 norms and is subject to particular precautions with regards to electromagnetic compatibility. For detailed description of EMC requirements, please contact bblüv customer service.

**NOTE - Portable and mobile radio frequency communications equipment can affect medical electrical equipment.**

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