


5 Function Skincare System (Vacuum, Spray, High Frequency, Ultrasonic, Brush) (SKU16165)

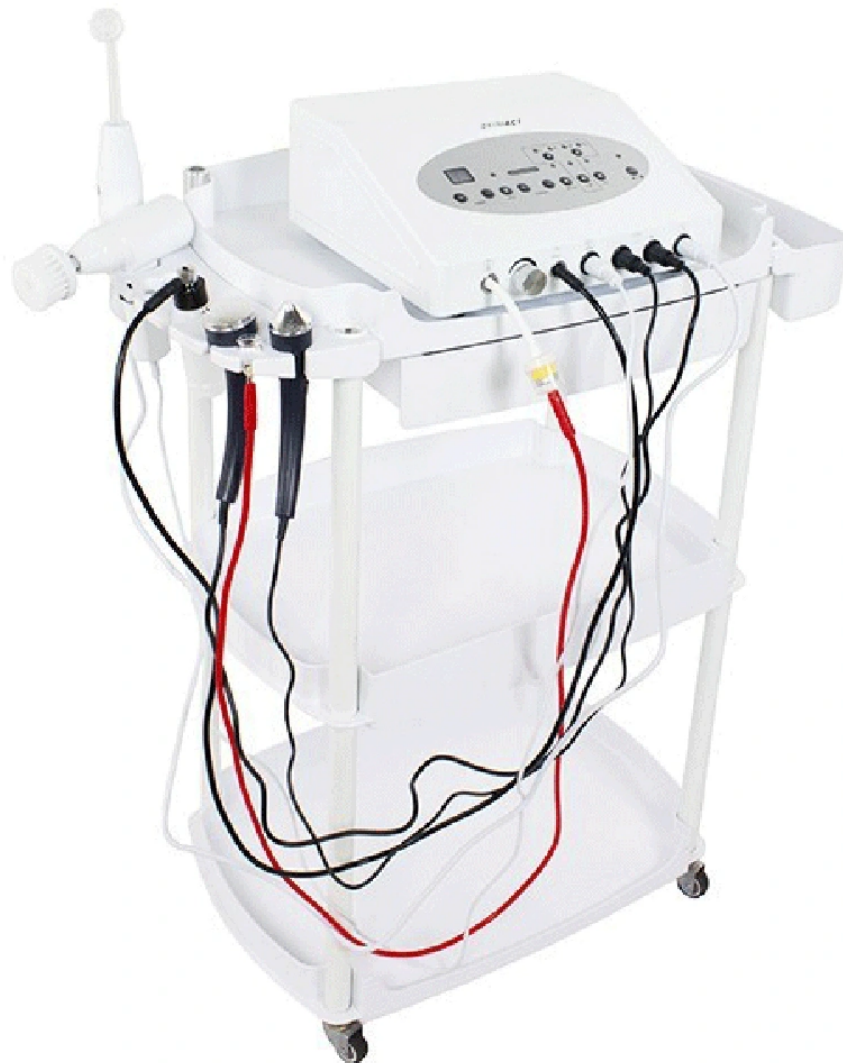

SKIN+ACT

Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

SKIN+ACT

Toll Free: 1.800.418.0830	www.skinact.com www.spaandequipment.com	International: 818.834.6640
---------------------------	--	-----------------------------

User Manual

Vacuum

The vacuum function is helpful in deep cleansing of the skin. It works like a miniature suction device to remove deeply embedded dirt, grease, and other impurities. It also promotes circulation by drawing blood closer to the surface of the skin.

How it works:

- Glass suction cups create suction when the finger hole is pressed.
- When the finger is released, suction breaks.
- A thin layer of cleansing lotion should be applied to allow the cup to glide over the skin smoothly.

Notes:

- Use light suction on dry or aging skin.
- Stronger suction can be used on oily skin.
- After use, wash suction cups with soap and water, disinfect, and dry before storage.
- Always use a filter to prevent impurities from entering the machine.

SPRAY

The spray function provides cooling, soothing, and toning benefits during facials. It is also effective for softening debris, loosening blackheads, and calming skin after treatments.

Preparation:

- Fill a spray bottle with 2 parts distilled water and 1 part astringent or toner.
- Fill only two-thirds full.

How to operate the spray:

1. Select the vent-hole and insert into the vacuum hose.
2. Turn on the switch and adjust intensity with the dial.
3. Press the finger hole on top of the spray head to release mist.
4. Turn switch "OFF" after use.

HIGH FREQUENCY

Benefits:

1. Aids in elimination and absorption.
2. Stimulates glandular activity.
3. Improves blood circulation.
4. Provides germicidal action.
5. Increases product penetration.

Caution:

- Clients must not wear metal jewelry during treatment.
- Do not use on pregnant clients or those with pacemakers.
- Always use high frequency after applying lotion or treatment product.

Operation:

1. Insert the appropriate electrode into the holder until it locks.
2. Switch on and adjust intensity.
3. After use, return the intensity to zero before removing the electrode.

PEELING (Brushing)

The peeling function exfoliates dead skin cells, removes dirt and grime, and smooths skin texture.

Operation:

- 1.Insert brush head until it clicks in place.
- 2.Switch on and adjust speed according to skin type.
- 3.To change brushes, switch off, remove, and insert new head.
- 4.After treatment, cleanse brushes with alcohol and disinfect.

Note:

- Use slower speeds for thin/sensitive skin.
- Use faster speeds for thick, coarse skin.
- On broken capillaries, use very gently or avoid.

ULTRASONIC WAVE THERAPY

Main Functions:

- 1.Soften thrombus and clean “red face”
 - Improves microcirculation, softens thrombus, expands blood vessels, and reduces redness.
- 2.Remove speckles
 - Helps fade pigmentation caused by poor treatment or blocked circulation.
- 3.Remove abnormal skin pigment
 - Gradually breaks down stubborn pigment deposits (e.g., freckles, sun damage).
- 4.Differentiate coloring matter & eliminate speckles
 - Destroys and melts speckle pigment membranes, reducing visible spots.

5.Prevent and remove wrinkles

- Stimulates metabolism, increases elasticity, and improves skin tone.

6.Reduce eye pouch & dark circles

- Increases circulation, breaks down fat deposits, and reduces swelling.

7.Dispel scars

- Reduces hypertrophic scars and softens tissue.

Treatment Notes:

- Clean the treatment area before use.
- Apply cream or lotion before ultrasonic therapy.
- Use low intensity for face and sensitive skin.
- Typical treatment time: 8–10 minutes.

Dosage Guidelines:

- Intensity: 0.5–1.25 W/cm² for face.
- Duration: 5–15 minutes (depending on skin condition).

Treatment course:

- Acne: 3–5 treatments per week.
- Chronic pigmentation/scars: 10+ treatments per course.

Modes:

- Successive wave – for inflammation, swelling, and vessel expansion.
- Pulse wave – for reducing heat and soothing irritation.

Dimensions:

Length: 18" L x 13.4" W x 6.5" H