

charli chair



Instruction manual

**READ ALL INSTRUCTIONS BEFORE ASSEMBLY
AND USE OF CHAIR. KEEP INSTRUCTIONS FOR
FUTURE USE.**

charli chair

Congratulations on Purchasing your new charlichair shower chair. Within no time you should be able to have this chair ready to meet your beautiful child :)




parts

IMPORTANT: Please make sure you have all of the parts shown on this page before beginning assembly. If you are missing any parts, please DO NOT return this product to the store. Please call our customer service department at 1300 558 056 or contact us at enquiry@charlichair.com

Tools required for assembly: Phillips screwdriver
(not included)

Please note this is a breakdown of all pieces most of these are already pre-Assembled.

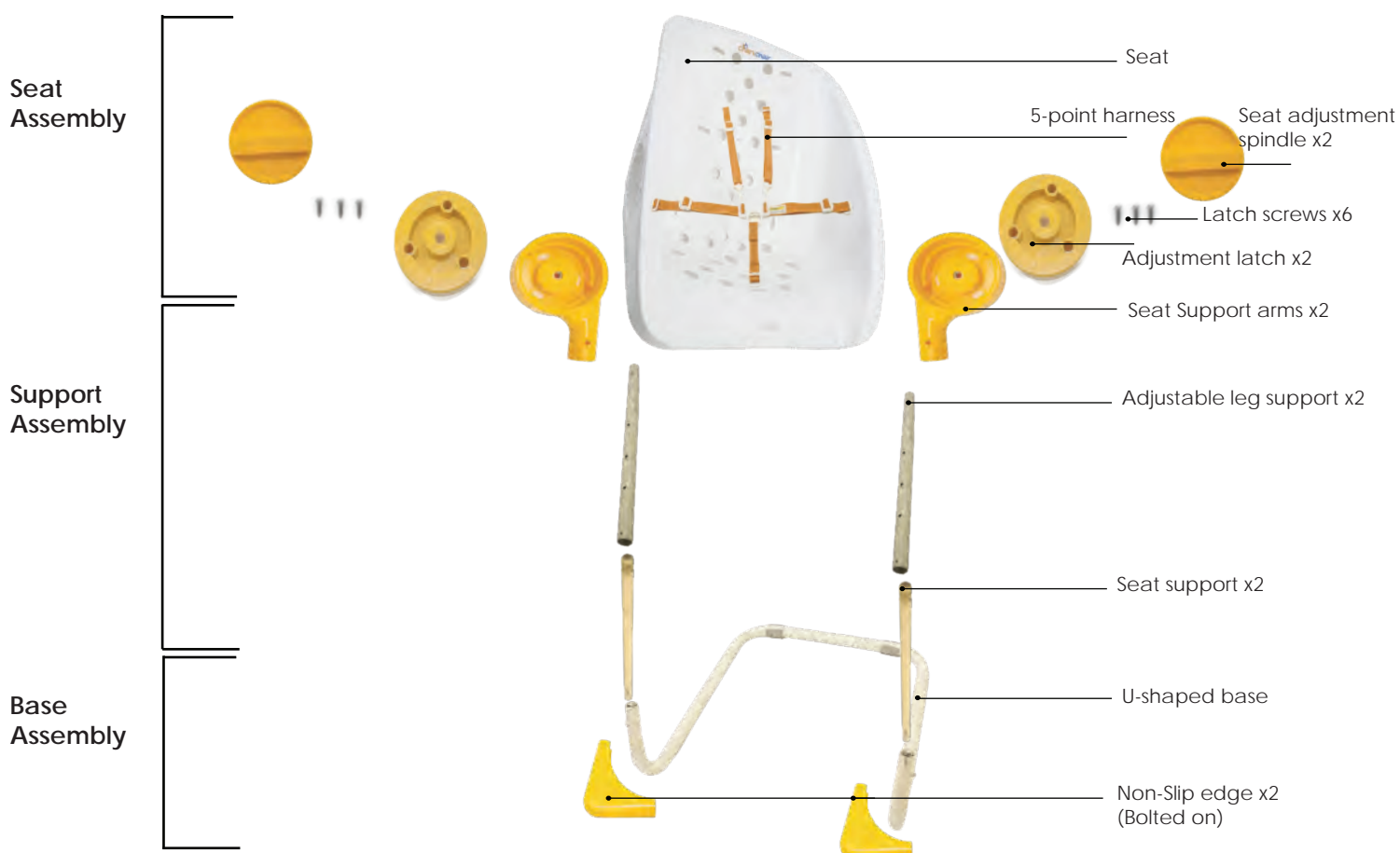

6 x Latch screws

charli chair

SAFETY GUIDELINES AND INSTRUCTIONS MUST ALWAYS BE FOLLOWED BEFORE MAKING USE OF THIS PRODUCT. PLEASE RETAIN THESE INSTRUCTIONS FOR FUTURE REFERENCE.

WARNING


- Prevent serious injury or death from falls or sliding out.
- ALWAYS secure child using the waist, crotch and shoulder harness restraint system. The child should be secured in the highchair at all times by the restraint system.
- NEVER allow child to use footrest to climb in or out of highchair.
- Follow the manufacturer's instructions.
- Do not leave children unattended.
- It is unsafe to use accessories other than those supplied by charlichair. Never modify the product in any way. Failure to follow instructions may put the user or carer at risk and will invalidate the warranty on the product.
- The seat is suitable for newborns to 17kg.
- The product will become unstable if the manufacturer's recommended load is exceeded.
- If in any doubt to the continued safe use of your Charlichair product or if any parts should fail, please cease using the product and contact our customer services department as soon as possible.
- Carry out all positional adjustments and ensure that they are securely fastened before you put the user into the product.
- Never leave the product on a sloping surface, only on flat surfaces.
- NEVER allow other children to substitute for adult supervision.
- NEVER lift this product with baby in it.
- NEVER place the child directly under the shower head or flowing water.
- When not in use disconnect shoulder straps from the waist strap and/or buckle.
- The harness can form loops which may be a strangulation hazard.
- Never leave a harness connected when not fitted to a child in the chair.
- To avoid serious injury always use the harness provided to secure the child. (adjust the straps to suit the size of your child's body)
- Always use the crotch strap in combination with the waist strap and shoulder strap.
- This chair should not be used near an open fire or an exposed flame. Position the product so that the child is far from dangers such as electrical wiring, heat sources and flames.
- Do not leave the high chair exposed to sunlight for long periods of time. Wait until it has cooled down before sitting the child in the chair.

⚠️ WARNING ⚠️

do's and dont's



Do not place toddler in seat directly under shower head

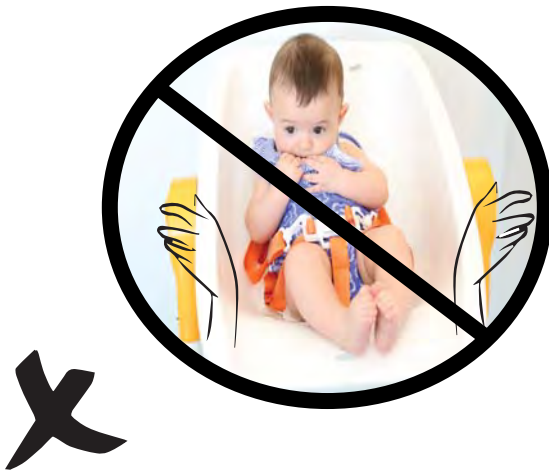

Do not adjust the seat position with the child in the seat



Do not adjust the height position with the child in the seat

assembly

Please note that most steps may have already come pre-assembled. This is a guide if you need to reassemble everything.



- 1** Place the leg support onto the holder on the U-shaped base. Push in the button to slide it onto the holder. Move it into place until the button comes out of the leg support



- 2** Repeat for the other side. The Assembled leg support on the base should look like this.



- 3** Place the adjustable leg support onto the holder on the leg support bar. Select the height required by the bar and push in the button to slide it onto the holder. Move it into place until the button comes out of the leg support



- 4** Repeat for the other side. The Assembled Adjustable leg supports on the base should look like this.

assembly



5 Place the seat support onto the holder on the arm support. Push in the button to slide it onto the holder. Move it into place until the button comes out of the seat support



6 Repeat for the other side. The Assembled seat support on the leg support should look like this.



7 Place the adjustment latch over the seat support and align with the holes.



8 Align the 3 screws with the holes from the seat. Tighten the screws with a philips head screwdriver. Repeat for the other side

assembly



9 The assembled adjustment latches should look like this.



10 Move the seat into the position that you require.
Note: there are 2 positions of recline.



11 Place the seat adjustment spindle over the adjustment latch and turn it in a clockwise way until it is tightened.



12 The assembled seat should look like this.

assembly



13 Position of the 5 Point harness assembled.



14 Squeeze the strap tabs together and push through opening. Repeat for the strap on the other side. ALWAYS use the same height slots for both sides of the shoulder straps. Use the lower shoulder strap position for newborns.

assembly



15 Inspect strap tabs and ensure tabs are flat against back of seat. Pull straps on BOTH sides to ensure that they are secure.



16 Squeeze the strap tabs together and push through opening. Repeat for the waist strap on the other side. Inspect strap tabs and ensure tabs are flat against back of seat. Pull straps on BOTH sides to ensure that they are secure.



17 Squeeze the strap tabs together and push through opening. Repeat for the strap on the other side. Use the lower crotch strap position for newborns.



18 Assembled 5 Point Harness.

using the seat

WARNING

- Prevent serious injury or death from falls or sliding out.
- ALWAYS secure child using the waist, crotch and shoulder harness restraint system. The child should be secured in the highchair at all times by the restraint system.
- NEVER allow child to use footrest to climb in or out of highchair.

WARNING

USE THIS HARNESS AT ALL TIMES

- (A) Waist Strap buckle prong and holder
- (B) Shoulder strap connector
- (C) Buckle housing
- (D) Harness Sliders

A five point harness is provided to restrain your child.

To secure your child in the harness please follow the steps below carefully.


Make sure the connector (B) is connected with the Buckle prongs holder on the waist straps (A)

Push buckle (A) into the buckle housing (C) until the buckle prongs snap into position.

Carefully adjust the harness by pulling the slider (D) until the desired fit is achieved in the lap and shoulder straps. Always adjust the harness when changing the seating position of the child.

To remove the child, release the buckle by squeezing the top and bottom prongs, remove the shoulder straps and lift the child out of the highchair.

Use of Harness

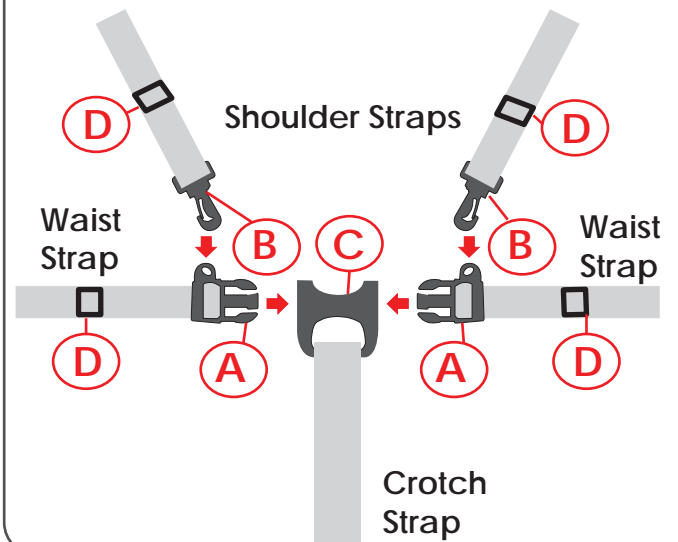


WARNING

WHEN NOT IN USE DISCONNECT SHOULDER STRAPS FROM THE WAIST STRAP AND/OR BUCKLE



Remove the child steps

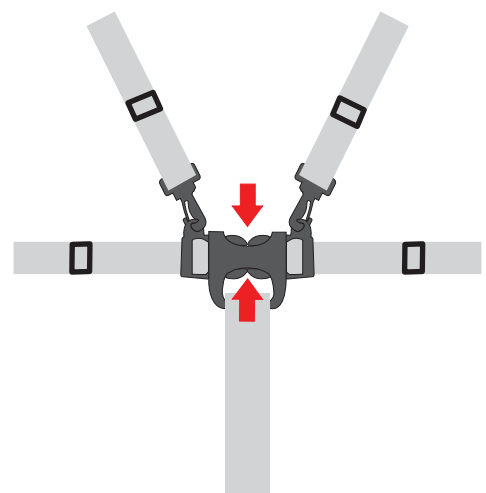

using the seat

Adjust recline position



Stabilize recline position



1. To adjust the recline position first make sure the child is not in the seat. Turn the seat adjustment spindle Anti-clockwise and push the seat upwards or downwards according to the desired position needed. The seat has 2 positions.
2. To stabilize the position then turn the seat adjustment spindle in a clockwise position. pull down on it slightly to make sure the seat is firm.



using the seat



1. To adjust the height position first make sure the child is not in the seat. Push the button located on the seat support on both legs and move the seat upwards or downwards according to the desired position required. The seat has 3 height positions.
2. To make sure the seat is stabilized you should hear the button pop out of the hole, pull down on it slightly to make sure the seat is firm.



charli chair

care and maintenance


The shower seat with attached straps and belts may be wiped clean using a mild cleaning solution and a damp cloth. Do not use bleach. Do not use harsh or abrasive cleaners. Rinse clean with water to remove residue.

From time to time check your high chair for loose screws, worn parts, torn material or stitching. Replace or repair the parts as needed. Excessive exposure to the sun or heat could cause fading, yellowing or warping of parts.

CLEANING THE CHASSIS:


Check all nuts and bolts for tightness and tighten when required. Check all coverings for bolts and sharp edges and replace when required. Replace when required in accordance with the manufacturer's instructions.

DO NOT place any of the parts into a dishwasher or washing machine to clean them. Clean restraint system straps and buckles using a soft cloth with warm water and mild soap. Clean seat with a soft cloth using warm water and mild soap. Disinfectant wipes can also be used. To clean the straps - wash the strap in warm soapy water, rinse and dry in the sun. If storing away store in airtight bag to avoid mould build up.

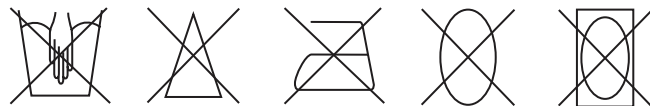

WARNING

Children have **DROWNED** while using bath aids. This is **NOT** a safety device. **ALWAYS** keep baby within arm's reach. **NEVER** leave baby in care of children.

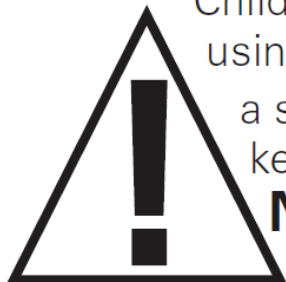

charli chair

Warranty against defects

CharliChair is guaranteed for 12 months from the date of purchase. Our goods come with guarantees that cannot be excluded under the Australian consumer law.

You are entitled to a replacement or refund for a major failure and for compensation for any other reasonable foreseeable loss or damage.

You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

This covers against faults which develop through faulty materials and workmanship. Misuse, accidental damages, normal wear and tear or neglect are not covered.

The CharliChair must be inspected that there was not any physical damage as the cause.

If there are any issues please contact us at enquiry@charlichair.com or call us on 1300 558 056.

Head Office:

CharliChair

6 Inglis Road Ingleburn

NSW 2565 Australia

1300 558 056

enquiry@charlichair.com

www.charlichair.com

charli chair

Contact Us



<http://charlichair.com/>



<https://www.facebook.com/Charlichair>



<https://twitter.com/Charlichair>



<http://youtu.be/1KQQWuqTdog>



<http://au.linkedin.com/pub/charlichair-baby-shower-chair/8a/b03/a56>



<https://plus.google.com/+Charlichair/posts>



<http://www.pinterest.com/babyshowerchair/>



http://instagram.com/charlichair_shower_chair



100 % AUSTRALIAN OWNED