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TABLE OF CONTENT

Content of starter pack	17
Tips from the Dryly team	18
Installation	19
Receiver functions	21
Changing the Wi-Fi network	22
The evening ritual	23
Dryly webshop	24
Error messages	25
Warranty	26
Help and support	26

THANK YOU FOR CHOOSING THE DRYLY BEDWETTING ALARM!

We are going to help you in the fight against bedwetting. Our goal is to help you as much as possible in the adventure to dry nights. We want your child to become motivated and enjoy taking on bedwetting.

In this manual we explain how to get started with the Dryly bedwetting alarm.

Dryly was founded because the son of co-founder Clarien had bedwetting problems for years. After years of trying all kinds of methods, her son came up with the idea of a fun, positive method. Why couldn't being potty-trained be fun? Why wasn't there a fun and motivating app? During Saturday morning breakfast the idea of Dryly was born.

Our vision is that we want to help both parents and children in the fight against bedwetting. We are going on an adventure with you. An adventure is exciting, has a goal, but never has a straight track. We are at your side, with positive and fun products, but also with help, advice and tips.

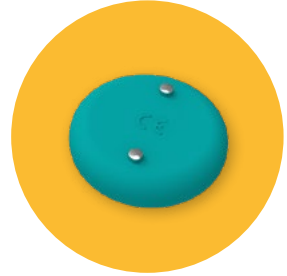
CONTENT OF STARTER PACK

Underwear

Wizzu

Transmitter

Receiver

**Mobile App
(App Store or
Googleplay Store)**

TIPS FROM THE DRYLY TEAM

- Note; You have to charge the set with a USB-C Cable, this is not included in the package.

- Keep in mind that this method takes an average of 3 months.

- Use the bedwetting alarm until your child has stayed dry for 14 nights.

- It is crucial to support your child during this period. Get out of bed too when your child wakes up from the alarm clock at night. Motivation and support are very important!

- Try to put as little pressure on your child as possible. Make it fun! Also make sure to reward your child differently besides app gadgets!

- The alarm stops going off by shaking Wizzu back and forth, so your child really needs to wake up.

- You should always charge the set with the transmitter clicked onto the receiver. Make sure you always put the set on the charger for 4 hours from day to day and turn it off during the day.

- Don't wake up your child to pee in the evening.

- Save the bedwetting alarm; your child may have a setback.

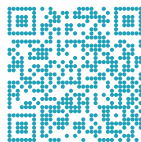
- Be careful when washing the shorts, make sure you take the transmitter off. Transmitters cannot be put in the washing machine!

- After an accident, make sure to remove the transmitter from the shorts first and only then shake the Wizzu.

INSTALLATION


Before you get started:

1. Put the transmitter & receiver on the charger. The light gives colour during charging. It must be charged for at least 5 hours before it can be put into use.
 2. Collect all parts from the starter pack.
 3. Make sure you have your Wi-Fi password available.
 4. Download the Dryly app in the Apple store or Google Play Store.
 5. Install the Dryly app.
 6. Create an account in the app.
 7. Enter your child's information.
- The installation will take a few minutes, we will help you through it! It is important to follow the steps of the app.


1 The beginning

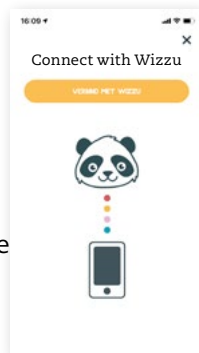- Select the Version B of the receiver in the app.
- Make sure you have the transmitter, receiver & wifi name and password.
- Click the transmitter on the receiver and put it on the charger, the light is on.
- After 1:30 Minutes, the receiver says "transmitter connected" the transmitter is now connected to the receiver. This may have already been done during the earlier 5h charging.
- Remove the set from the charger and place the transmitter next to the receiver.

2 Connect the receiver to the Wi-Fi network

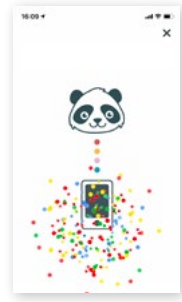
- Please note; during the various steps in the pairing process, this may occasionally take longer!
- Turn on the receiver only when the app says to do this!
- You hear a noise, this is the sign that it is working and the battery is charging.
- Connect to the “Wizzu” Wi-Fi network in the app.
- The app creates an overview of available networks.
- Select your home network and enter the password.
- When the receiver is connected correctly, you will hear a success sound.
- As soon as you see confetti in the app, the receiver has been successfully added!
- If connecting doesn't work please check if the permission settings regarding “location” are correct. Phone's settings › Dryly app › location › Make sure the check mark is set to “always”.

3 Dressing Wizzu

- Now that the bedwetting alarm has been installed successfully, your child can dress up Wizzu with the first points.

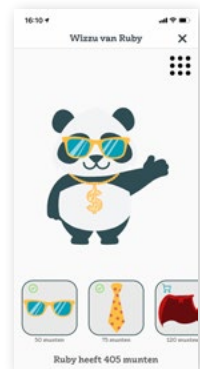Something not working out? We will gladly take you by the hand through the chat on our website or by phone.

RECEIVER FUNCTIONS

● Off

Pressing the black button on the receiver for 3 seconds turns the receiver off.

● On

By pressing the black button, you turn the receiver on.

● Charging

You can charge the set by clicking the transmitter onto the receiver and then placing it on the charger. The set should charge from day to day for a minimum of 4 hours. Turn the set off during the day. When the battery runs out, the set will turn itself off.

● Receiver battery light

● Green = 40-100% | ● Red = 0-40%

Note! The light only reflects the battery of the receiver. To check the exact battery percentage of transmitter & receiver, please check the app.

● Sound volume:

When that the set is on, you can change the volume of the receiver to different modes by briefly pressing the black button once.

The set always starts with a volume of 100%

1x press = 75% / press again = 50% / press again = silent mode / press again = 100% again. Make sure you wait 3 seconds each time between pressing.

In silent mode, only the app gives an alarm (and vibrator should you have this variant) Handy for silent moments like a sleepover!

● Offline mode

When you pair the set by charging it. You can start the installation process to set up the set on the home network with the phone app.

Do you prefer to use only the transmitter & receiver? There is an offline mode with only connection between the transmitter & receiver that requires no home network or app.

You can install the offline mode by leaving the set on for 5 minutes after pairing (which happens while charging) if you lay the transmitter alongside receiver. The set will automatically say: “Wizzu in *offline mode*” after 5 minutes. If this doesn’t happen, perform reset 1 (page 25) and try again.

REMOVE OR CHANGE THE RECEIVER FROM THE WIFI NETWORK

Disconnecting the receiver

- Go to ‘My Wizzu’ through the burger menu in the top left corner of the app.
- Select the receiver of which you want to change the Wi-Fi network and press ‘delete’.

Selecting a new Wi-Fi network

- Make sure you have your Wi-Fi password available.
- Make sure you have first performed reset 1 and put the set on the charger again for 1:30 minutes until it indicates “transmitter connected”.
- Press ‘pair’ Wizzu’ on the app.
- Turn the receiver off.
- Only turn on the receiver when the app says to do so.
- You hear a noise, this is the sign that it is working and the battery is charged.
- Connect to the “Wizzu” Wi-Fi network in the app.
- The app searches for nearby Wi-Fi networks.
- Select your home network and enter the password.
- When the receiver is connected correctly, you will hear a success sound.

THE EVENING RITUAL

The moment before bedtime is very important while using the bedwetting alarm. A moment of rest and awareness.

1. Let your child hear the alarm sound a few times every night before going to sleep. This way your child knows when to wake up to. You can do this through the app!
2. Let your child go to the toilet before he/she goes to sleep.
3. Let your child put on the Dryly pants and press the transmitter in place.
4. Check the app with your child.
5. Read a book or let your child read for at least 20 minutes.
6. Let your child go to the toilet again.
7. Wish your child good night and put Wizzu about 1 meter away. This way your child has to get up and really wake up to turn off the alarm.
8. Never wake your child when you go to sleep yourself to go to the toilet one more time. By doing so, you will delay the training.

Sweet dreams!

DRYLY WEBSHOP

Pants

The Dryly pants are made of organic cotton and have a perfect fit. We developed the cheerful prints with children.

The bottoms can be washed at 60 degrees Celsius; they can also be put in the dryer. Avoid the use of fabric softener or chlorine. We also have neutral bottoms for children who are a bit older or for sleepovers.

Waterproof mattress protector

Place the mattress protectors under the fitted sheet to protect your mattress. This prevents stains and unpleasant odors.

Wizzu

Wizzu is the sweet panda, a great friend for your child. You can always order a Wizzu for a brother or sister.

The transmitter (also available with vibrating version!)

The transmitter detects the moisture in the Dryly underpants. As soon as an accident occurs, a signal is sent to the receiver. Do not wash the transmitter because it will not work.

The receiver

The receiver is placed in Wizzu and makes a wake-up sound in the event of an accident.

The receiver immediately sends a message to the mobile app so that you can help your child. Parental support is crucial to the success of this method. In the app you can see the state of the battery. Experience shows that you need to charge the receiver with transmitter attached every from day to day, and that you have to switch it off during the day.

ERROR MESSAGES

“Transmitter out of range”

It can sometimes happen that you hear the message ‘transmitter out of range’. This means that the receiver has been searching for the transmitter’s signal for a while, but has not been able to detect it.

Causes

Your child sleeps on his or her stomach

The body blocks signals from the transmitter. If your child sleeps on his or her stomach, this can cause the signals not to get as far.

You improve the connection by placing the receiver at the hip of your child’s body. For example, you can put a chair next to the bed with Wizzu on it. Whenever the alarm loses the signal. The transmitter will make the sound: “*transmitter disconnected*” Whenever the alarm is back in range again, it will tell “*transmitter connected*”.

The transmitter is not yet connected to the receiver

When you receive our bedwetting alarm, pairing the transmitter is part of the process. You pair the transmitter with the receiver by charging them together. After 1:30 minutes, the set says “*transmitter connected*” You only have to do this once and you will hear it once. Do you not hear this? Perform reset 1 and try again. If this step did not go right. Please consult the customer service.

SOLUTIONS

Reset 1

Press and hold the receiver’s black button for 20 seconds. Make sure that the set is turned ON. After this, you will hear “*Transmitter out of range*”. This resets the set back to factory settings (The wifi network, transmitter & app pairing will be erased) After this, you need to pair the transmitter and receiver again while charging.

Reset 2

Press a pin/ paperclip into the hole made in the letter “D” of the receiver. This will restart the receiver. You do not need to pair the set again.

WARRANTY

Dryly is committed to good quality products on a daily basis. It may happen that your Dryly bedwetting alarm does not work properly. Dryly b.v. complies with the law on market practices and consumer protection.

Dryly has a warranty period of 6 months for electronics and 3 months for underwear upon receipt of the bedwetting alarm by the purchaser. The warranty covers manufacturing and/or material defects and defects caused by normal use. This is at Dryly's discretion.

If your bedwetting alarm does not work as it should, follow these steps; Report your complaint, preferably by email to info@dryly.com, with the subject; 'complaint' with your order number. We will respond as soon as possible and do everything we can to solve your problem.

Damage caused by careless handling or by not following the instructions as stated in the user manual is excluded. The warranty can be invoked upon presentation of the invoice. The warranty is invalidated if repairs have been carried out by third parties, if the user has worn a nappy over the pants with the transmitter, if the bedwetting alarm has been purchased through a reseller (e.g. Marktplaats or eBay) and if water damage has occurred.

HELP AND SUPPORT

Are there any uncertainties? Be sure to always check our frequently asked questions (FAQ) on the website too! Here you will find a lot of information about usage and training!

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