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Symbols	Description
	Batch number
	Warning and Precautions
	Use before date
	Manufacturer address
	CE Symbol
	Non sterile
	Keep out of sunlight
	Medical device
	Consult user manual
	Production date
	Catalog Number
	Keep the product dry
	Model number

Lumbar Brace with Adjustflex™ tension support

Bynatic

Lower Back Brace

User manual

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Anweisungen

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SUPPORT TO
MOVE

3ynatic

Our
mission

Bynatic helps people to maintain the freedom of movement by providing high-quality products and supporting them during their recovery journey.

English

User manual

T

Thank you for purchasing the **Bynatic back brace**. Please read this user manual carefully, and in case of any questions regarding your personal situation always contact your treating doctor or therapist.

Safety regulations

1. Only use the back brace in consultation with your treating doctor or therapist.
2. Follow the instructions in the user manual at all times and only use the back brace as described under intended use.
3. In case of inexpert use, the proper effect of the back brace cannot be guaranteed.
4. Do not make changes to the back brace yourself, this can negatively impact the effect of the brace and damage the health of the user.
5. If the symptoms get worse, always contact your treating doctor or therapist.
6. Use the back brace in the correct size in order to guarantee optimal effect and wearing comfort.
7. Do not use the back brace in case of damage or deformation. Safe operation can only be guaranteed if the back brace is undamaged.
8. Using the back brace in combination with other products must always be discussed with your treating doctor or therapist first.
9. The back brace must only be used by one patient.
10. The back brace is not suitable to use while sleeping.

Washing instructions

- Wash by hand in warm water with soap.
- Wash in water no warmer than 40 °C.
- Do not iron, do not dry-clean, do not use chloride, do not put in the dryer.
- Do not dry the back brace near heat sources.

Intended use

The Bynatic back brace is a medical aid, providing stabilisation and support of the lumbar spine.

Not suitable for

- People below 18.
- People with psychological disorders, without proper support.
- Use near open fire or hot surfaces.

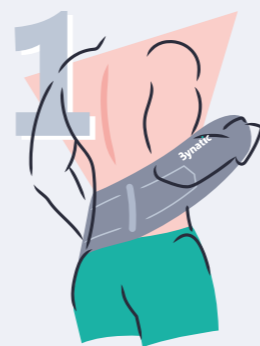
In the following cases, we advise you to contact your treating doctor or therapist:

- Malignancy
- Oedema
- Thrombosis
- Varicose veins
- Sensory disorders
- Ankylosing spondylitis
- Skin conditions or wounds
- Cardiovascular or respiratory conditions
- Serious form of canal stenosis or spondylolisthesis
- Scars that are red and/or swollen and warm to the touch
- A condition relating to a (osteoporotic) vertebral fracture

Reporting requirement

In case of serious incidents caused by the use of this back brace, the manufacturer must be notified immediately.

User manual


Undress down to your underwear. Loosen the velcro and make sure that the Bynatic logo is facing outwards.

Pull the sides of the brace forward, fold the big flaps of the brace over each other and close the velcro.

Materials

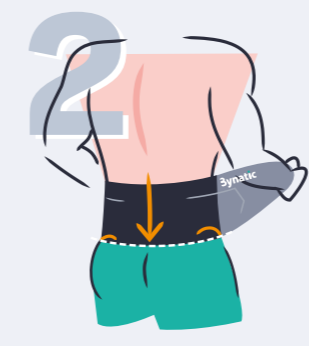Polyester (55%), Neoprene (30%), Nylon (5%), Polyvinyl Chloride (5%), Rubber (5%)

In a standing position, put the middle of the brace on your lower back. The bottom of the brace should barely cover the top of your buttocks (at the level of your pelvic brim).

Next, tighten the two small flaps of the brace until you feel sufficient support. It is important that you are still able to breathe through your stomach. If this is no longer possible, you must loosen the brace.

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