

# InBody770

User's Manual **USA**

Please note the important information below before reading this manual.

 **Warning**

Failure to comply with safety warnings and regulations can cause serious injury or death.

 **Caution**

Failure to comply with safety cautions and regulations can cause injury or property damage.

---

For any problems with the InBody770 or any clinical questions, contact:

**InBody**

13850 Cerritos Corporate Dr., Unit C, Cerritos, CA 90703, USA

TEL: +1-323-932-6503

FAX: +1-323-952-5009

Website: <http://www.inbodyusa.com>

E-mail: [info@inbodyusa.com](mailto:info@inbodyusa.com)

©2014 InBody Co., Ltd. All rights reserved.

Reproduction, adaptation, or translation of this manual is prohibited without prior written consent from InBody Co., Ltd under the copyright laws. This manual might have typographical errors, and its content can be changed without prior notice. InBody Co., Ltd shall not be liable for any errors, incidental, or consequential damages that occurred by not complying with the content of the User's Manual.

InBody Co., Ltd reserves the right to modify the appearance, specifications, and etc. of the InBody770 to improve the quality of the product, without prior notice for reasons of performance improvement.

# InBody770

## User's Manual Contents

### **I. InBody770 Installation**

A. Product Components	4
B. Operating Environment	5
C. Installation Instructions	5
D. Initial Setup	10
E. Connecting Printer and Stadiometer	14
F. Maintenance	15

### **II. InBody Test**

A. Precautionary Steps	16
B. Test Instructions	17
C. Test Posture	18

### **III. Transportation and Storage**

A. Cautions during Transportation	19
B. Repacking Instructions	19
C. Transportation and Storage Environment	21

### **IV. Frequently Asked Questions (FAQ)**

A. Regarding the InBody	22
B. Regarding the InBody Test	24

### **V. Others**

A. Exterior and Functions	25
B. Safety Information	28
C. Classification	29
D. Specifications	29





# I. InBody770 Installation

## A. Product Components

The InBody770 consists of the following components. Please make sure all of the following components are present.

\* Please inspect each component of the InBody770 for defects prior to installation.

- InBody770
- Adapter (DC 12V, 3.4A) 1 EA
- Power cord 1 EA
- User's Manual 1EA
- Hexagonal Wrench
- Hinge Cover

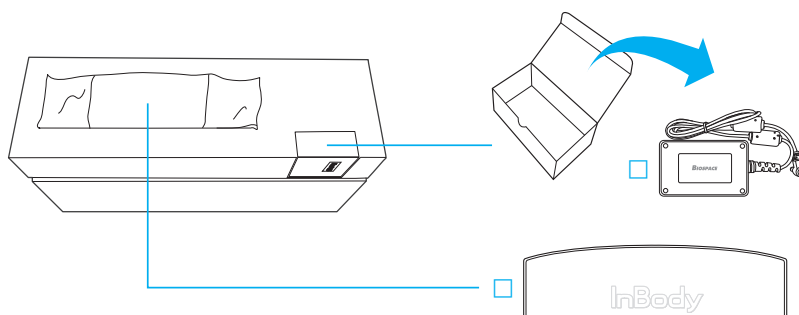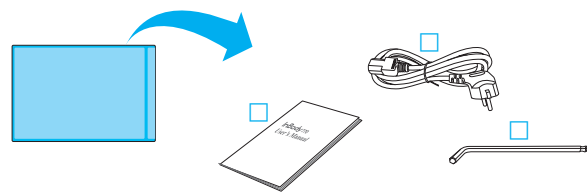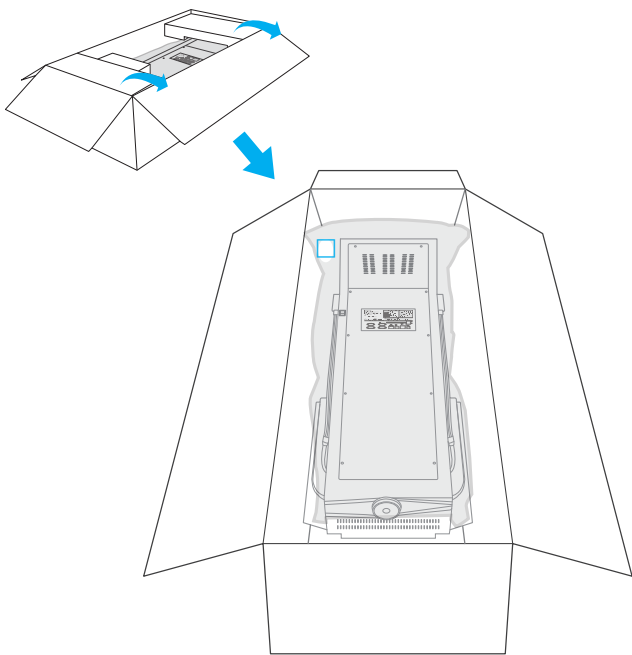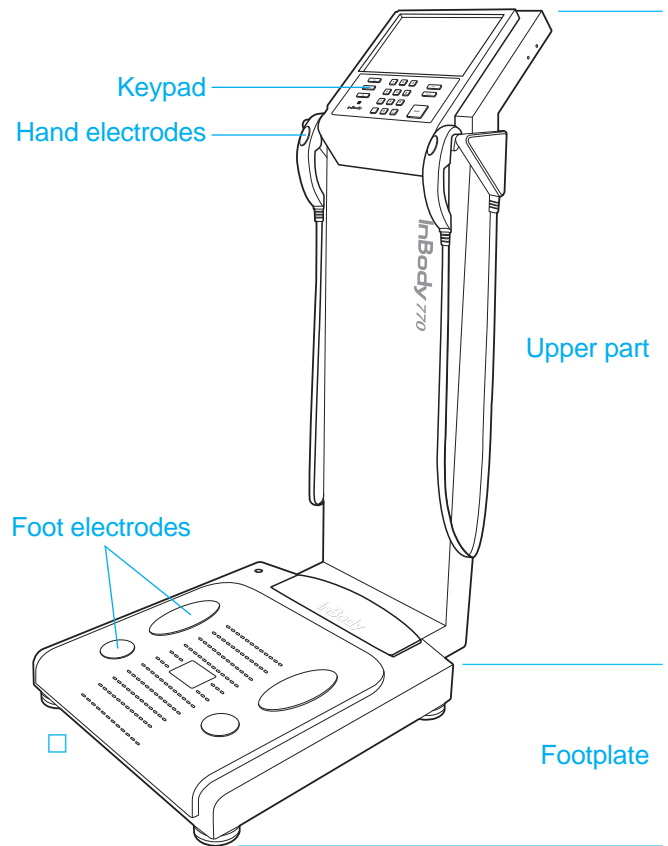


## B. Operating Environment

Please make sure that the environment is adequate for the InBody770 installation. This equipment is designed for indoor use. If installing outdoors, the following requirements must be fulfilled.

Temperature range	50 ~ 104 (10 ~ 40 )
Relative humidity	30 ~ 75% RH
Atmospheric pressure	70 ~ 106kPa

## C. Installation Instructions


1. Open the packing box of the InBody770 and remove the packing pads. Then take the InBody770 out of the box.

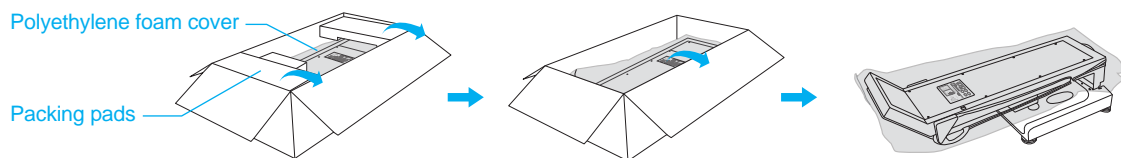

### ⚠ Caution

- If you have any problems installing your InBody770, please contact InBody for assistance.
- Do not transport the equipment by holding the screen portion or the joints of the hand electrodes.
- Keep the packing materials provided for repacking the equipment in the future. Other wastes should be disposed of according to relevant laws and regulations.

\* Please refer to the following illustrations to properly transport the equipment.

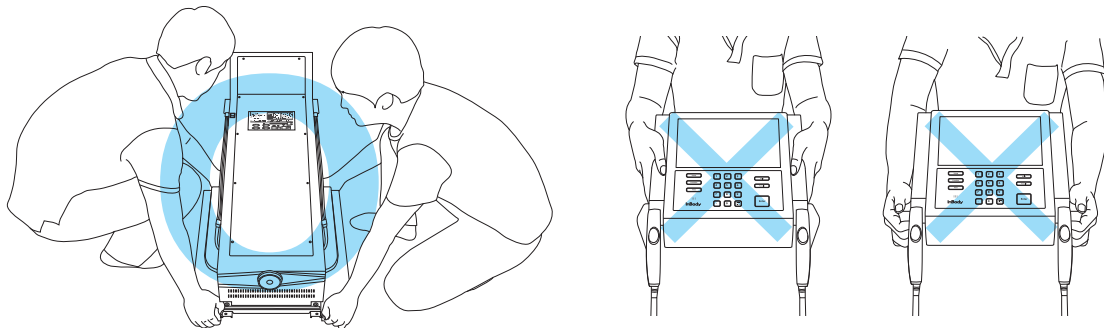


### ⚠ Caution


- Using the InBody770 on carpet may cause static electricity, which could damage the equipment. If installing the InBody770 on carpet is unavoidable, please use an antistatic mat.
- Install the InBody770 on a leveled, non-vibrating surface. Installing the equipment on an uneven surface may cause the examinee to fall down. Test results may also be inaccurate.
- Never clean the hand and foot electrodes with liquid spray or detergent directly. The equipment may corrode and/or malfunction if the liquid or detergent leaks inside. Use the InBody Tissue when cleaning the InBody770.

\* For inquiries regarding the InBody Tissue, please contact InBody.

2. After completely raising the upper part of the InBody770, remove the polyethylene foam cover. Then remove the packing material from the footplate.

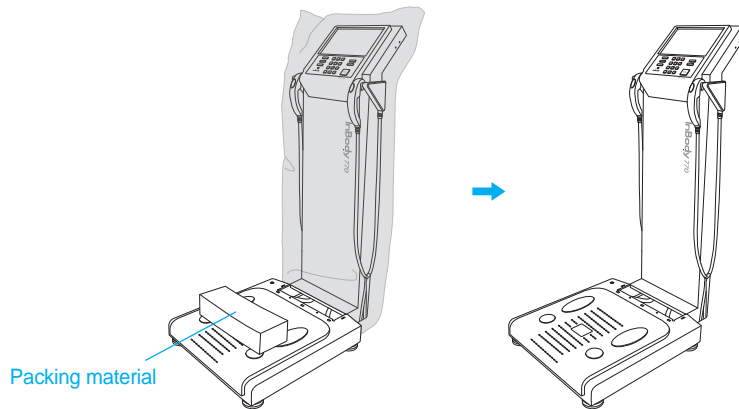


3. Slightly lower the upper part of the InBody770 and remove the packing bands, which are used to connect the leveling screws and the hand electrode cables.

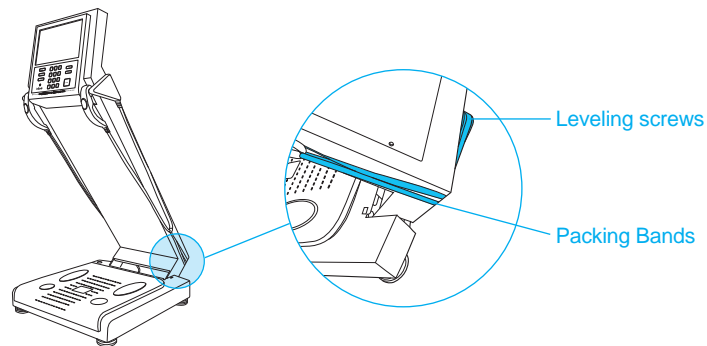

**!** **Caution**


- Do not use a knife or scissors to remove the packing bands. Keep packing bands for repacking the InBody in the future.

4. Please refer to the following illustrations to remove the Packing Bands from both hand electrode cables.

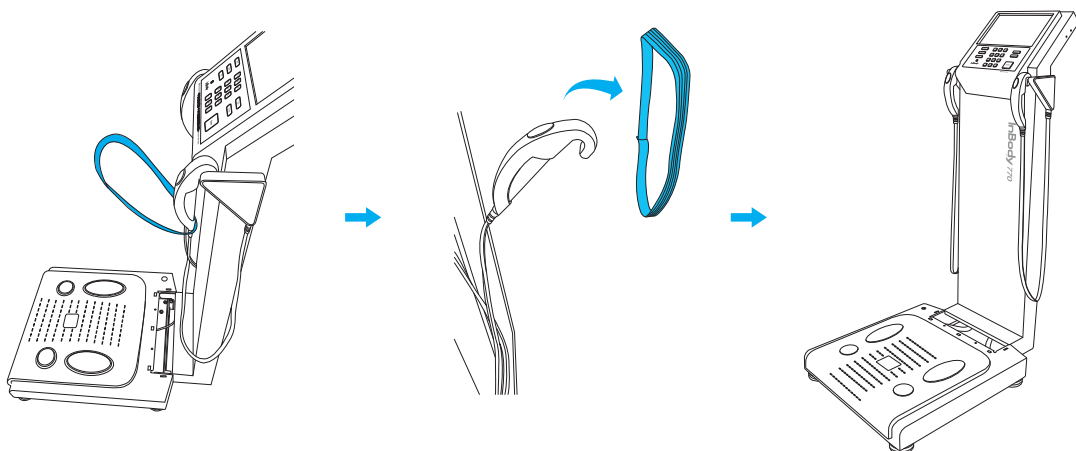


5. Completely raise the upper part of the InBody770.

6. Tighten the hexagonal screws, on the InBody770 joint, by rotating the Hexagonal Wrench clockwise.

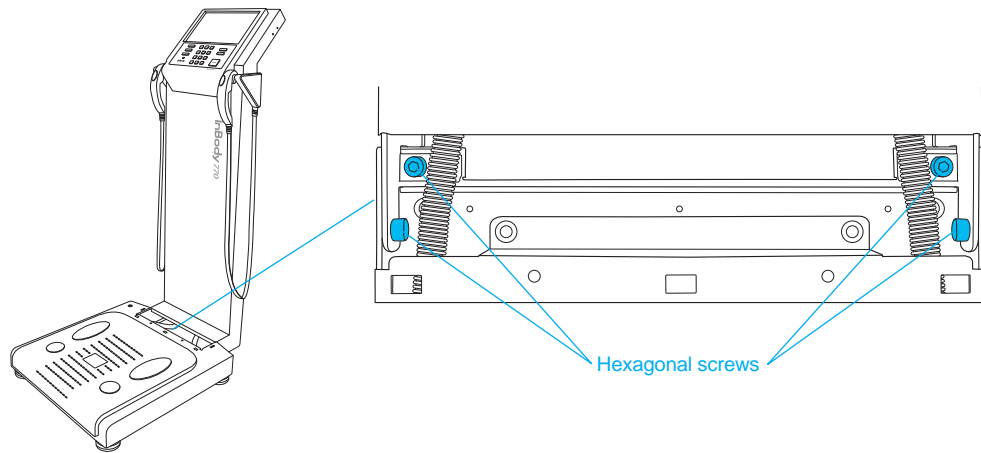


7. Please refer to the following illustrations to insert the Hinge Cover.

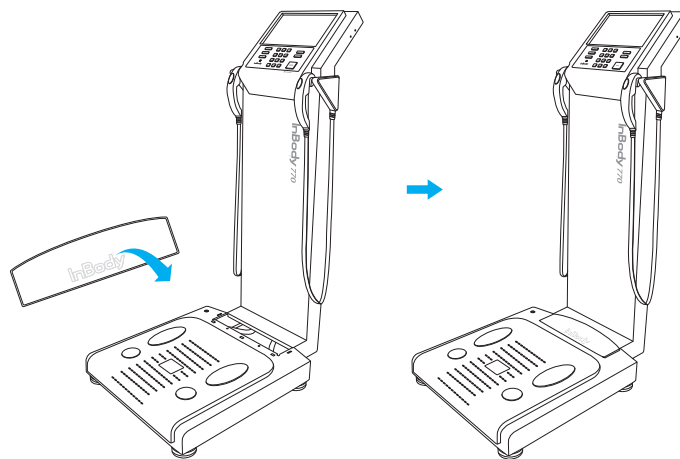

8. Level the InBody770 by rotating the leveling screws under the footplate to the left and right so that the air bubble is centered.

\* Leveling the equipment is necessary for accurate measurement of weight. There are a total of 5 leveling screws.

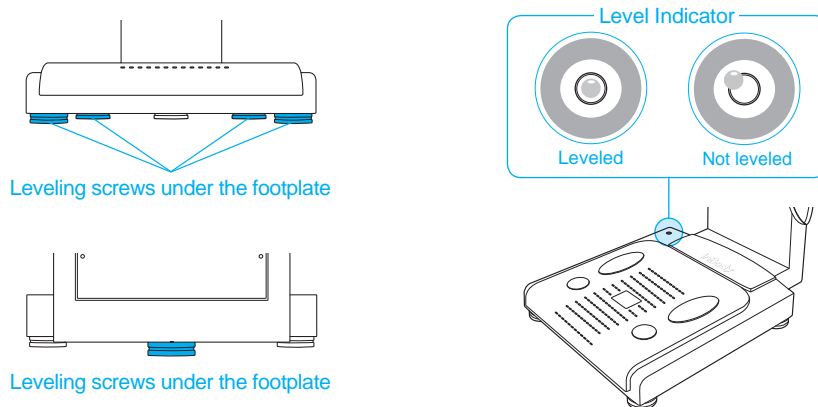


 **Caution**

- Avoid injuring your hands when rotating the leveling screws under the footplate.

9. Connect the adapter (□) to the power input port, which is located on the rear panel (□). Connect the adapter (□) to the power cord (□). Plug the power cord (□) into a grounded 3-socket outlet.

\* The InBody770 can be used in connection with other test equipment such as a stadiometer or data management software called Lookin'Body120. For more information, please refer to 'A. Exterior and Functions 3. Rear Panel' in section 'V. Others' in this User's Manual.

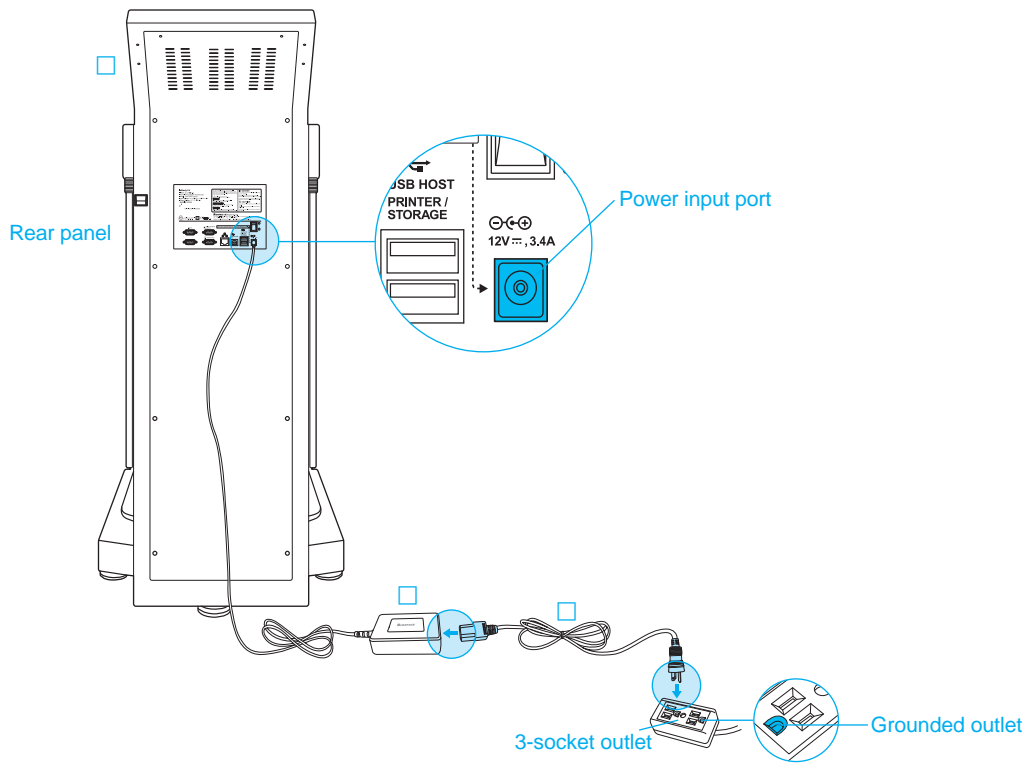

### Warning


- Do not place the InBody770 in a location making it difficult to disconnect the power cord.
- Do not plug in or pull out the power cord with wet hands. There is a risk of an electric shock.
- Always use an outlet connected to the rated power (AC 100 - 240 V). Using other power rated outlets may result in fire or malfunction.
- When using a power surge protector, make sure that the outlet or the extension cable has adequate power capacity.
- Do not disassemble or modify the equipment including internal parts without written consent from the manufacturer. This may cause electric shock or injury, product malfunction, inaccurate results, and will void the manufacturer's warranty.
- Do not directly contact the InBody770 with any other electronic device when the InBody770 is on. This may result in an electric shock.



 **Caution**

- If the InBody770 is not plugged into a grounded outlet, it may cause damage through electric surges or product malfunction. This may affect the test results.
- The test results may be inaccurate if the InBody770 is under electrical interference. Do not install the InBody770 near products that generate electrical interference such as fluorescent lights, large AC motor equipment (treadmill, vibration plate, refrigerator, air-conditioner, compressor, etc.), high-frequency thermal therapy equipments, or heating appliances. Do not share the power source of the InBody770 with other electrical devices. This may affect the test results.
- When connecting the InBody770 with other test equipment, turn on the other equipment first. When turning off other equipment, turn off the InBody770 first. This is necessary to minimize electrical surges on the InBody770.
- Always use the specified adapter provided by InBody as it is a part of the InBody770. Using other adapters may result in malfunction of the InBody770.
- Operation of the InBody770 6,500 feet above sea level may affect the weight measurement.

10. Flip the power switch to turn on the InBody770.

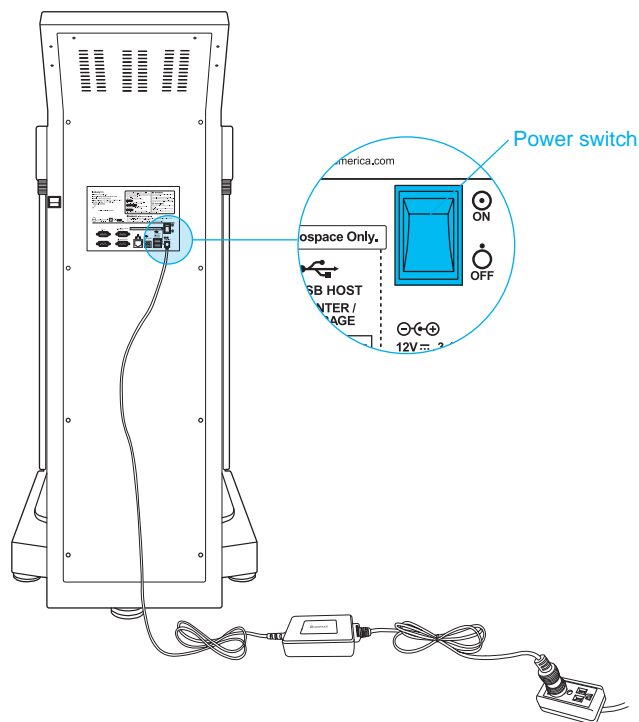

## D. Required Settings

- The InBody770 automatically starts booting when it is turned on. While booting, it performs a self weight calibration.
  - \* While booting (about 5 minutes), make sure there is nothing on top of the footplate. Please do not stand on the footplate, or place objects on the footplate.



- Press the [Administrator Menu] button on the screen, which appears when no one is on the footplate.



Administrator Menu

- Input the password (default password: 0000) to have access the Administrator Menu.



- The Administrator Menu will give you access to 'Setup' and 'Troubleshooting'.



Setup



Troubleshooting

- 1) Setup: Configure settings and manage data according to the test environment.
  01. Date/Time/Units/Password/Volume  
: Change or modify the InBody's basic settings.
  02. Self Mode/Professional Mode
    - Self Mode: The examinee takes the InBody Test by entering only his/her height.  
Throughout the test, instructions and the InBody Information will be shown on screen.
    - Professional Mode: An examiner is present and guiding the examinee through the InBody Test.
  03. N/A
  04. N/A
  05. Bypass Age/Gender  
: The examinees can bypass inputting their age or gender if the test environment is designed for testing only adults or a specific gender.
  06. View/Print/Delete Data  
: The administrator can manage test results using ID or mobile phone number.
  07. Export Data as Excel  
: You can export test results as an excel file on a USB Thumb Drive. Exported test results can be viewed as an excel file on a computer.
  08. Data Backup/Restoration  
: Back up InBody Test results to a USB Thumb Drive or restore test results using a backup file on a USB Thumb Drive.
  09. Printer Setup  
: Connect the printer to the InBody. A connected printer will allow for printing results sheets after testing.
  10. Results Sheet Types  
: Select which results to utilize with the InBody770.  
(Body Composition Results Sheet and Body Water Results Sheet).
  11. Automatic Printing Options  
: Print applicable results sheets automatically after each completed test. The InBody can print up to 2 copies after every test.
  12. N/A
  13. Outputs/Interpretations for Results Sheet  
: Select outputs or interpretations that will appear on the right side of the Body Composition Results Sheet and Body Water Results Sheet.

14. Results Sheet Custom Logo

: Insert a logo on upper right corner of the printed results sheet.

\* Please contact InBody for help with uploading or modifying a logo.

15. Printing Alignment

: Adjust the alignment of where the results will be printed on the results sheets.

16. Internet Options

: You can connect the InBody to your PC through WiFi. Once connected to your WiFi network, you can utilize the data management software Lookin'Body120.

17. Bluetooth

: Connect the InBody770 to data management software Lookin'Body120 via Bluetooth.

18. Manual/Automatic Weight

: Select whether to have weight automatically weighed or manually entered before testing.

19. Adjust Weight

: Adjust measured weight by a fixed value on the InBody. (Example: Workout clothes at the gym are approximately 0.44 lbs most examinees are assumed to be wearing workout clothes, so the examiner may adjust the set value to -0.44 lbs.)

20. Reference Range

: Set the reference range for BMI and Percent Body Fat.

\* The ideal value for BMI may also be set.

21. N/A

22. N/A

23. Touchscreen Alignment

: Adjust the alignment of the touchscreen.

24. Customer Service Information

: View the contact information for InBody. Please contact InBody if you have any inquiries regarding the InBody Test, or problems that cannot be resolved through the 'Troubleshooting' menu.

25. Auto-Lock

: Set the password and wait time for auto-lock on the InBody770.

2) Troubleshooting: See additional information on how to use the InBody. Refer to the troubleshooting checklist when there are problems that occur during the InBody use/test.

01. Customer Service Information

: See the contact information of InBody. Please contact InBody if your problem cannot be resolved through the 'Troubleshooting' or if you have further inquiries regarding the InBody Test.

02. Results Sheet does not print.

: View the troubleshooting checklist when the Results Sheet does not print.

03. Weight is not being measured.

: View the troubleshooting checklist when weight is not being measured.

04. Weight measurement seems to be inaccurate.

: View the troubleshooting checklist when the weight measurement seems to be inaccurate.

05. The InBody Test has stopped.

: View the troubleshooting checklist when the InBody Test has stopped.

06. Test results seem to be inaccurate.

: View the troubleshooting checklist when the test results seem to be inaccurate.

## E. Connecting Printer and Stadiometer

### 1. Printer


In order to print InBody Results Sheets, an InBody770 compatible printer is required.

\* A list of printers compatible with the InBody770 can be found at <http://www.inbodyservice.com>.

1) First turn off the InBody770 and then the printer.

\* You may experience connection issues in connecting the printer to the InBody770 if the InBody is turned on.

2) Plug the USB cable provided with the printer into the USB HOST port on the rear panel of the InBody770 and plug the other end of the USB cable into the printer.

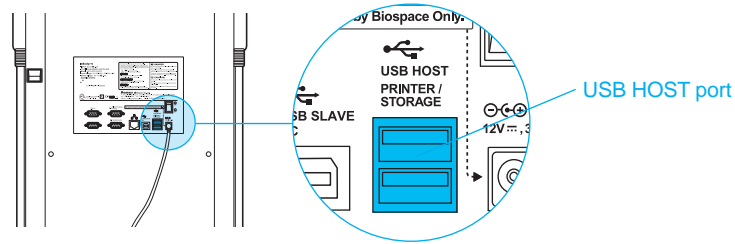

3) Turn on the printer.

4) Turn on the InBody770 and setup your printer under Setup of the Administrator Menu '09. Printer Setup'.

5) You can edit your printing settings under Setup of the Administrator Menu from '11. Automatic Printing Options' through '15. Printing Alignment'.

### 2. Stadiometer


If a stadiometer is connected to the InBody770, the height values measured by the stadiometer will be sent directly to the InBody770.

\* Always connect a stadiometer from InBody.


1) First turn off the InBody770 and then the stadiometer.

\* You may experience connection issues in connecting the stadiometer to the InBody770 if the InBody is turned on.

2) Plug the serial cable provided with the stadiometer to the 9-pin stadiometer serial port on the rear panel of the InBody770.

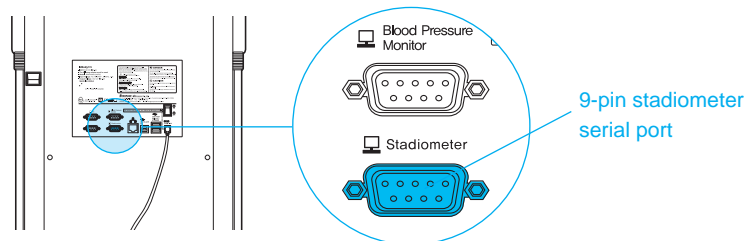

3) Turn on the stadiometer.

4) Turn on the InBody770. If the stadiometer is connected to the InBody770, the stadiometer icon (  ) will appear on the top left corner of the screen when no one is on the footplate.

## F. Maintenance



### Caution

- Place the hand electrodes on the hand electrode holder to prevent the electrodes from falling down. Dropped hand electrodes may cause malfunctions.
- Do not place any objects on the footplate.
- Do not apply excessive force on the equipment.
- Turn off the equipment if you are not using it for a day or longer.
- Do not allow any liquid substances to contact the equipment directly. Keep food and drinks away from the equipment. Substances getting inside the equipment can cause critical damage to the electronic components.
- Use a lint-free cloth to gently wipe the external surface of the equipment about once every week. Be careful not to scratch the LCD screen.

## II. InBody Test

### A. Precautionary Steps

#### Warning

- Individuals with medical implant devices such as pacemakers or essential support devices such as patient monitoring systems, must not use this equipment. Safe, low-level currents will flow through the body during the test, which may cause malfunctioning of the device or endangering of lives.
- Bioelectrical Impedance Analysis (BIA) uses safe low level currents which are not harmful to the body. However, we do not recommend pregnant women to test.
- Children and people with limited mobility should be supervised or assisted when attempting to test on the InBody.
- After an individual with any kind of contagious disease or infection tests on the InBody, use an InBody Tissue to clean the equipment.

#### Caution

- Stand upright for about 5 minutes before testing. Taking the test immediately after lying in bed or sitting for a long period of time might result in a slight change in the test results. This is because body water tends to move to the lower body as soon as the person stands or gets up.
- Do not eat before testing. In cases where the examinee has already eaten, the test should be put off for at least two hours after the meal. This is because food mass is included in the examinee's weight and thus, may result in measurement errors.
- Use the bathroom before testing. Waste is not included in the body's compositional elements, but the volume of urine and excrement is included in the weight measurement and may affect the accuracy of the test results.
- Do not exercise before testing. Strenuous exercise or sharp movements can cause temporary changes in body composition. Even light exercise can change your body composition temporarily.
- Take the test in the morning, if possible. Body water tends to gravitate towards the lower body throughout the day, affecting accuracy of the test results.
- Thoroughly wipe the palms and soles with the InBody Tissue before testing. Testing may be difficult if the examinee's palms and soles are too dry or if the examinee has too many calluses.  
\* For inquires regarding the InBody Tissue, please contact InBody.
- Avoid physical contact with the examinee during testing. Contact may lead to interference affecting test results.




## B. Test Instructions

1. Step on the footplate when the screen below is shown.

- \* The screens vary according to the Setup of the Administrator Menu '02. Self Mode/Professional Mode'.  
Professional Mode: An examiner is present and guiding the examinee through the InBody Test.  
Self Mode: The examinee takes the InBody Test following the instructions that are displayed on screen.



2. Weight measurement begins.

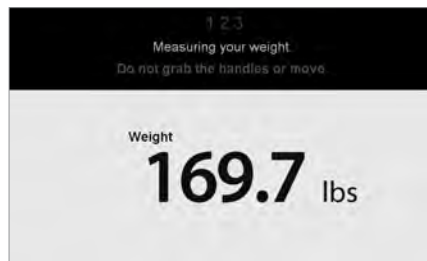

3. Input personal information.

- \* Input height only if using Self Mode.



Professional Mode



Self Mode


4. Maintain proper posture to take the test.

- \* Refer to 'C. Test Posture' for the proper posture.




5. The InBody Test begins.

- \* InBody Information is shown if using Self Mode.

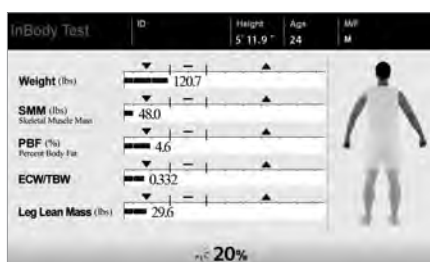


Professional Mode

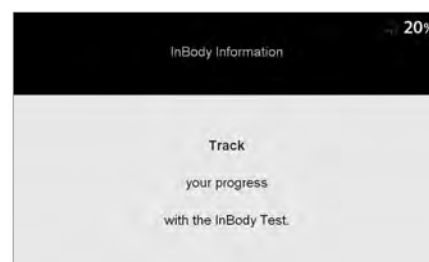

Self Mode

6. When the test is completed, the results will be shown on screen.

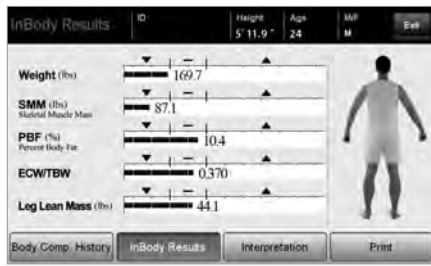

Professional Mode




Self Mode

## C. Test Posture

The examinee must maintain proper posture to have accurate test results.


\* The test will proceed when there is good electrical contact.

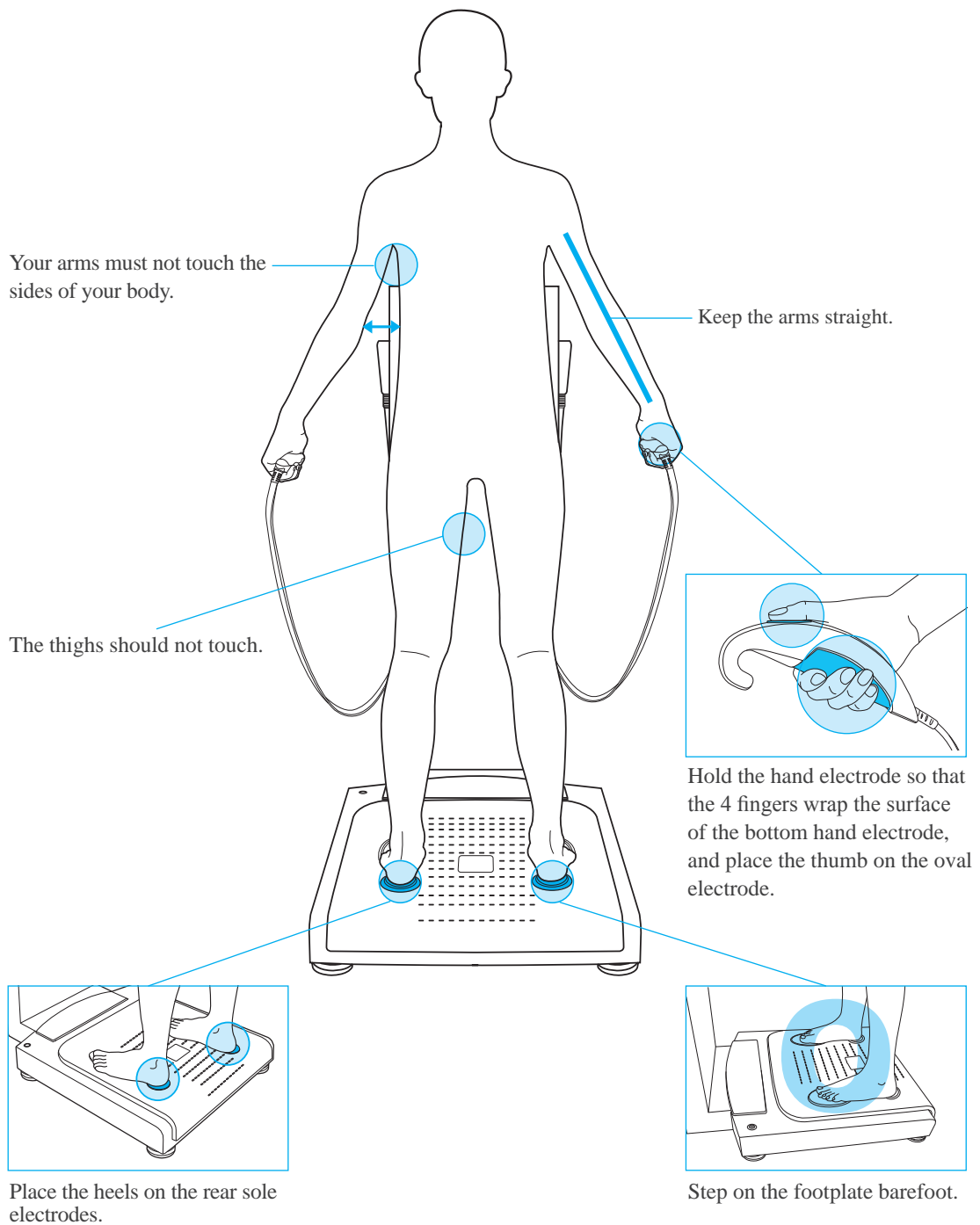

### III. Transportation and Storage

#### A. Cautions during Transportation


When transporting, have two people keep the InBody770 parallel to the ground.

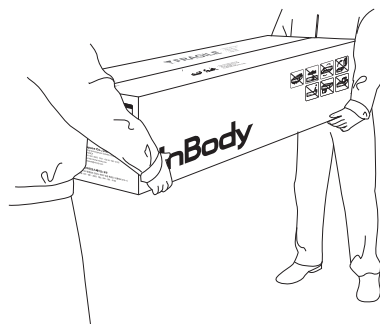


#### B. Repacking Instructions

Once the InBody770 is installed, avoid transporting the equipment. If it must be transported, repack it in the following sequence.


1. Turn off the InBody770.
2. Separate the connected adapter, cords and cables from the equipment. Place both hand electrodes on the hand electrode holders.
3. Loosen the hexagonal screws which are located on the joints of the InBody770 to counter-clockwise using the hexagonal wrench.

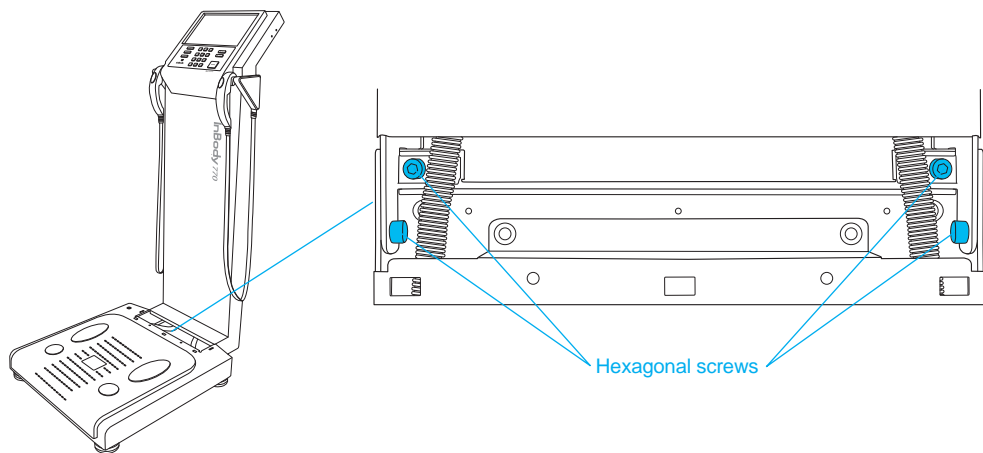


4. Please refer to the following illustrations to properly hang the packing bands on the hand electrodes.

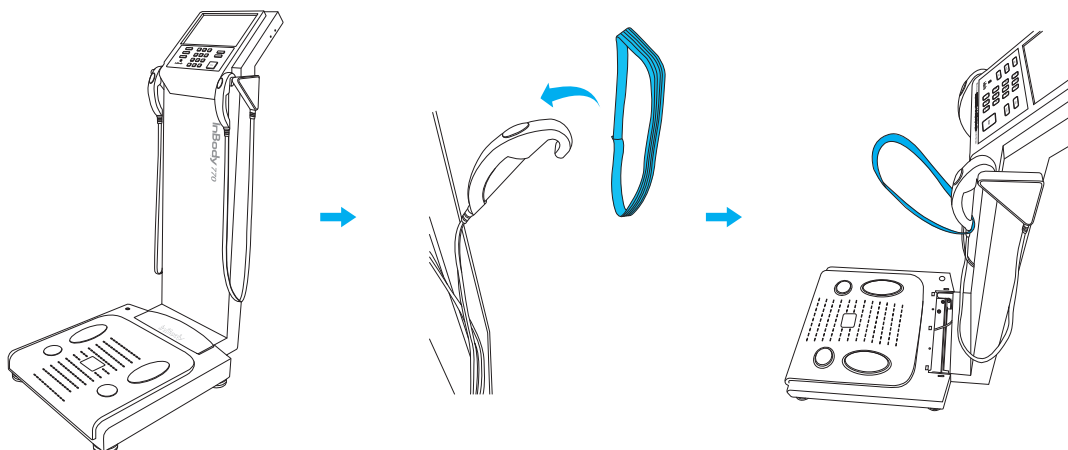

5. Slightly lower the upper part of the InBody770 and hook the packing band to the leveling screws as illustrated below.

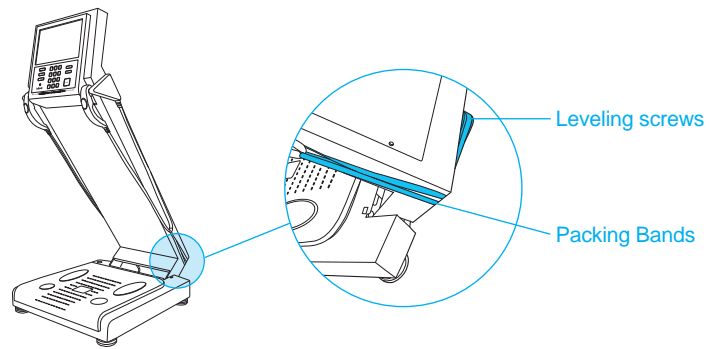


6. Place the packing material on the footplate of the InBody770. Cover the InBody770 with the polyethylene foam cover then fold down lower the upper part.

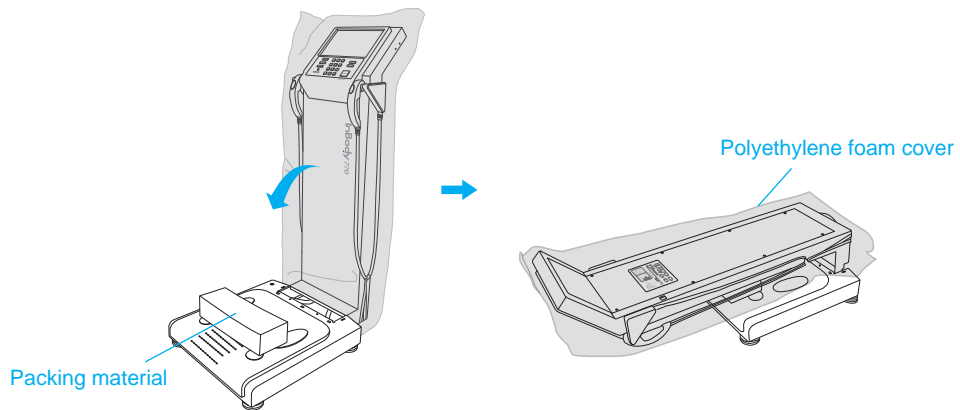


 **Caution**

- Always use the protective packing materials provided by InBody when repacking.


7. Place the InBody into the packing box.

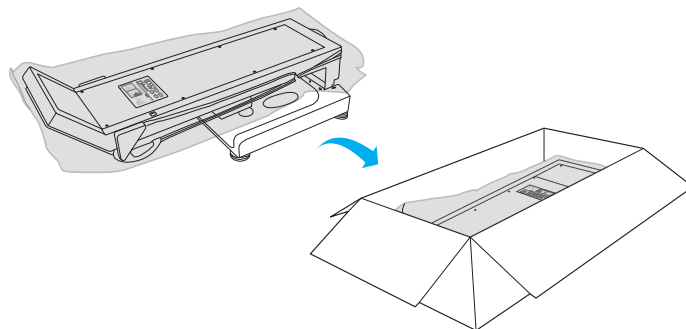

8. Place the adapter into the adapter box as illustrated below. Insert the adapter box and hinge cover into the packing material.

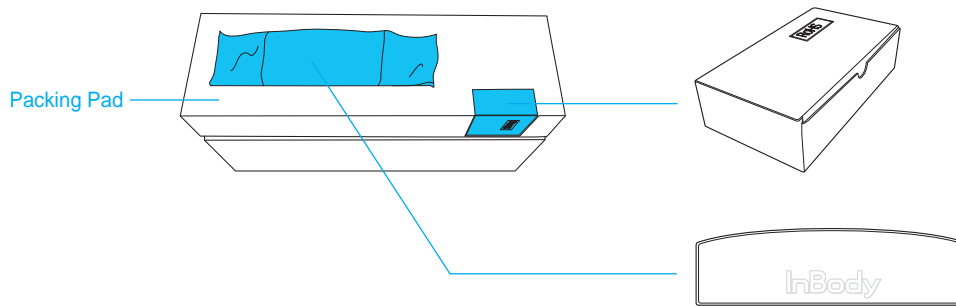

9. Place the packing pad over the equipment and tape up the packing box.

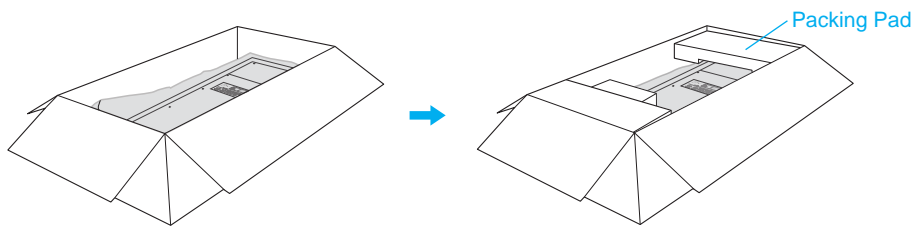

### C. Transportation and Storage Environment

The InBody770 should be transported or stored under the following conditions.

Temperature range	-4 ~ 158 (-20 ~ 70 )
Relative humidity	10 ~ 95% RH (No Condensation)
Atmospheric pressure	50 ~ 106kPa

## IV. Frequently Asked Questions (FAQ)

Even if no problems arise from the equipment, users may still have many questions, especially regarding clinical procedures. Few common questions and answers are listed below. If your questions are not answered here, please contact InBody.

\* Customer contact information can be found under Setup of the Administrator Menu '24. Customer Service Information'.


### A. Regarding the InBody

If a problem arises with the InBody770, you may first attempt to check the 'Troubleshooting' in the Administrator Menu. The InBody770 can help you diagnose and solve some problems. If your problem cannot be resolved through the 'Troubleshooting', please refer to the possible solutions below.


Question	Answer
----------	--------

• My InBody770 does not turn on.


• Insert the power plug completely into a grounded 3-socket outlet.

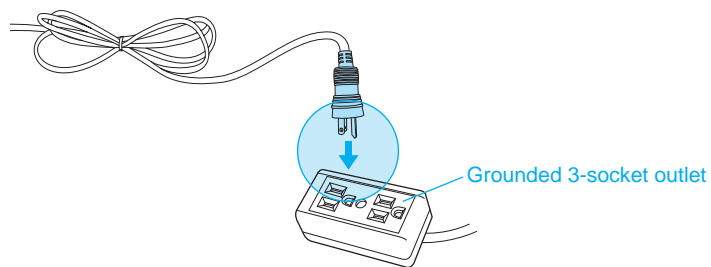


• When using a power surge protector, the equipment may not power on if the power switch on the power surge protector is turned off. Check the power surge protector which the power plug is connected to.

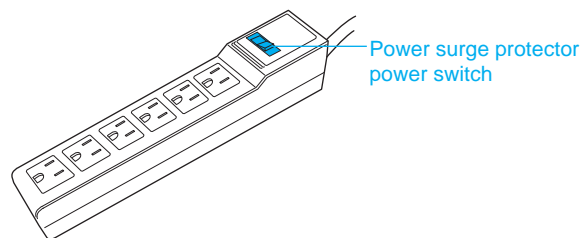


• The problem may occur if the power cord is not completely plugged into the adapter. Insert the power cord completely into the adapter.

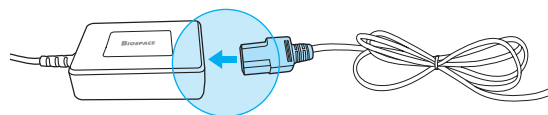

• The problem may occur if you are using a power adapter that was not provided by InBody. Always connect a power adapter (DC 12V, 3.4 A) provided by InBody.

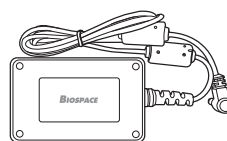

Power adapter

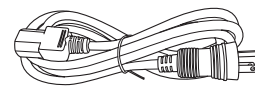

Power cord

Question	Answer
<ul style="list-style-type: none"> <li>• My touchscreen is inaccurate or not responsive.</li> </ul>	<ul style="list-style-type: none"> <li>• Calibrate the touchscreen under Setup of the Administrator Menu '23. Touchscreen Alignment'.               <ul style="list-style-type: none"> <li>* Press firmly to optimize touchscreen response.</li> </ul> </li> <li>• If you cannot enter the Administrator Menu due to touchscreen problems, please restart the InBody. The InBody stores the last touchscreen alignment and will automatically recall the previous touchscreen settings. The InBody can also recognize if its touchscreen alignment is off screen and will automatically take the user to the calibration screen after restarting.</li> </ul>
<ul style="list-style-type: none"> <li>• I would like to connect other equipment to the InBody770.</li> </ul>	<ul style="list-style-type: none"> <li>• Please refer to 'E. Connecting Printer and Stadiometer' in section 'I. InBody770 Installation' in this User's Manual.</li> </ul>

## B. Regarding the InBody Test

Some of the more common clinical questions are answered below. If additional questions or more clarification is desired, please contact InBody.

Question	Answer
<ul style="list-style-type: none"> <li>• Must socks or stockings be removed for the InBody Test?</li> </ul>	<ul style="list-style-type: none"> <li>• Bare skin contact is essential in the analysis using the BIA method. Socks or stockings may cause a varying degree of distortion in the results. Socks or stockings must be removed to obtain accurate data.</li> </ul>
<ul style="list-style-type: none"> <li>• Is it okay to wear accessories (jewelry, watch, rings, etc) or metal objects while taking the InBody Test?</li> </ul>	<ul style="list-style-type: none"> <li>• The ideal condition for the analysis is simply standing with no clothes and wearing no accessories. However, this may not always be possible. Therefore, we recommend that the examinee remove as many clothing items and accessories as possible as they may affect the weight.</li> </ul>
<ul style="list-style-type: none"> <li>• Who cannot take the InBody Test or will have difficulties taking the InBody Test?</li> </ul>	<ul style="list-style-type: none"> <li>• Individuals with medical implant devices such as pacemakers, or essential support devices such as patient monitoring systems, must not use this equipment. The currents will flow through the body during the test, which may cause malfunctioning of the device or endangering of lives.</li> <li>• Children, amputees, or the elderly, may have trouble testing if they cannot hold the hand electrodes or stand still on the foot electrodes.</li> </ul>
<ul style="list-style-type: none"> <li>• Can a person with metal implants in their body take the InBody Test?</li> </ul>	<ul style="list-style-type: none"> <li>• The ideal test methodology is where the examinee does not wear anything metallic. Individuals with metallic implants may have skewed test results due to the conductivity of the metal.</li> <li>• As the weight of clothes and other wear affects the results of the body composition analysis, it is strongly recommended to take off any heavy clothing or metallic wear. Except for the weight, jewelry does not effect the body composition analysis, as the contact point with the InBody770 are the hands and feet.</li> </ul>
<ul style="list-style-type: none"> <li>• I have limited mobility and cannot maintain proper posture for the InBody Test. How can I still be tested?</li> </ul>	<ul style="list-style-type: none"> <li>• It is impossible to test if an individual cannot maintain contact with the hand or foot electrodes. InBody has a line of products that conducts body composition analysis on bed-ridden examinees that allow the patients to connect to the electrodes via adhesive or clip.. For more information, please contact InBody.</li> </ul>
<ul style="list-style-type: none"> <li>• Is the electric current harmful to the body?</li> </ul>	<ul style="list-style-type: none"> <li>• The physiological electric impedance method uses safe, low-level currents that are not harmful to the body. All InBody products have been approved for medical use with CE certification. Many medical institutions around the world are actively using the InBody.</li> </ul>
<ul style="list-style-type: none"> <li>• How often should I take the InBody Test?</li> </ul>	<ul style="list-style-type: none"> <li>• Individuals who are undergoing any programs that may affect their body composition are strongly recommended to have the InBody Test done every two to four weeks.</li> <li>• Consistent testing will allow individuals to track and monitor their progress over time.</li> </ul>
<ul style="list-style-type: none"> <li>• What are the precautionary steps to ensure accuracy of the InBody Test?</li> </ul>	<ul style="list-style-type: none"> <li>• Please refer to 'A. Precautionary Steps' in section 'II. InBody Test' in this User's Manual.</li> </ul>



## V. Others

\* The InBody770 is manufactured according to the quality management procedure of InBody. InBody complies with the ISO9001 and ISO13485 which are international quality management systems.

\* This equipment satisfies the IEC60601-1 (EN60601-1), an international safety standard for electronic medical equipment. This equipment also satisfies the IEC60601-1-2 (EN60601-1-2), an international standard for electromagnetic conformity.


### A. Exterior and Functions

The following are the names and functions of each part of the InBody.

\* Please inspect each component of the InBody770 for damage prior to installation.


#### 1. Upper Part

- Hand electrode: Examinee holds the hand electrode so that the 4 fingers wrap the surface of the bottom hand electrode while the thumb is placed on the oval electrode.
- Hand electrode cable: Supports the hand electrode and encloses the wiring for the electrode.
- Body: Connects the upper part of the equipment to the lower part.
- LCD screen: Shows each stage of the test, instructions, test results, etc. You can touch the screen to input the data required for the test, configure settings, or view test results.
- Number keypad: Used for inputting age, height, and other number-based data.
- Setup button: Used for entering 'Setup' under the Administrator Menu when no one is on the footplate.
- Function button: Used for entering 'Troubleshooting' under the Administrator Menu when no one is on the footplate.
- Print button: Used for printing the test results.
- Speaker: Provides audible indication for test in progress, test complete, and successfully saved setting changes.
- Decimal point button: Used for inputting the decimal point in ID, height, age, or weight.
- Gender buttons: Used for selecting gender (Male or Female).
- Enter button: Used to finish inputting data or to save changes in Administrator Menu.
- Delete button: Used for deleting inputted data.

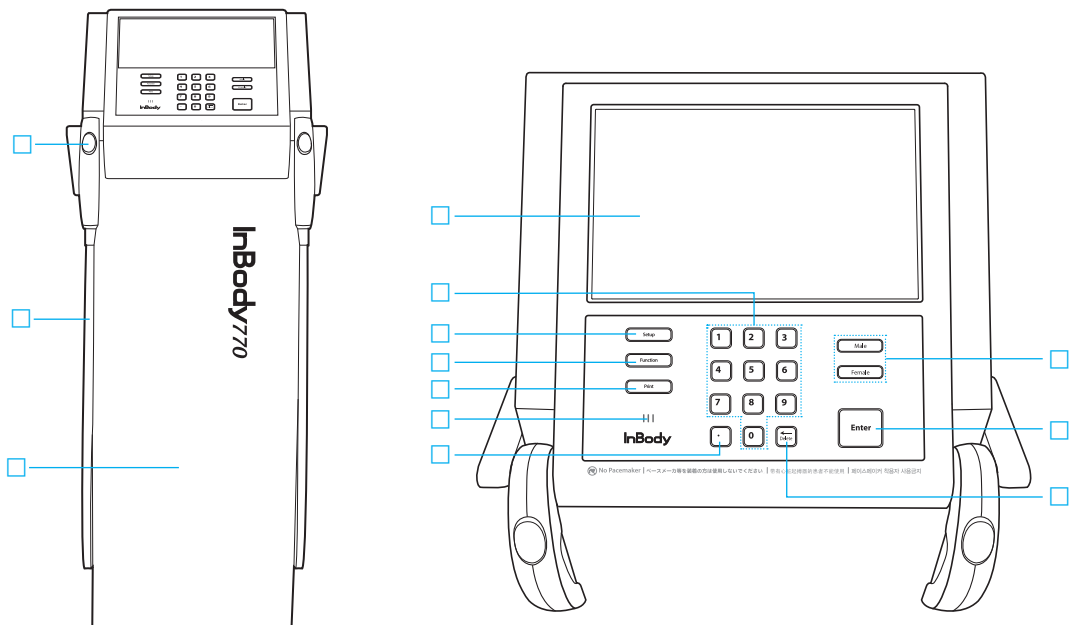


## 2. Footplate

- Front sole electrode: The examinee makes contact with this electrode by stepping with the front part of their foot.
- Rear sole electrode: The examinee makes contact with this electrode by stepping with the heel of their foot.
- Footplate: This is connected to the scale, which measures the examinee's weight.
- Level indicator: Indicates the current horizontal level of the InBody770.
- Hinge Cover: Joins the upper part and lower part of the equipment together.
- Leveling screws: Used for adjusting the horizontal level of the equipment.

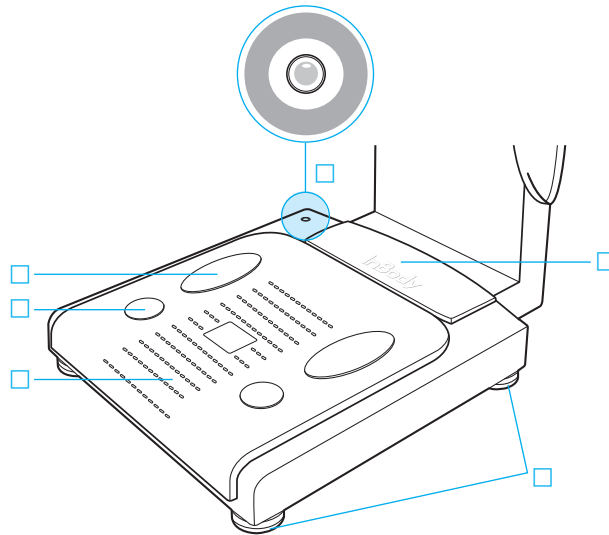




### 3. Rear Panel

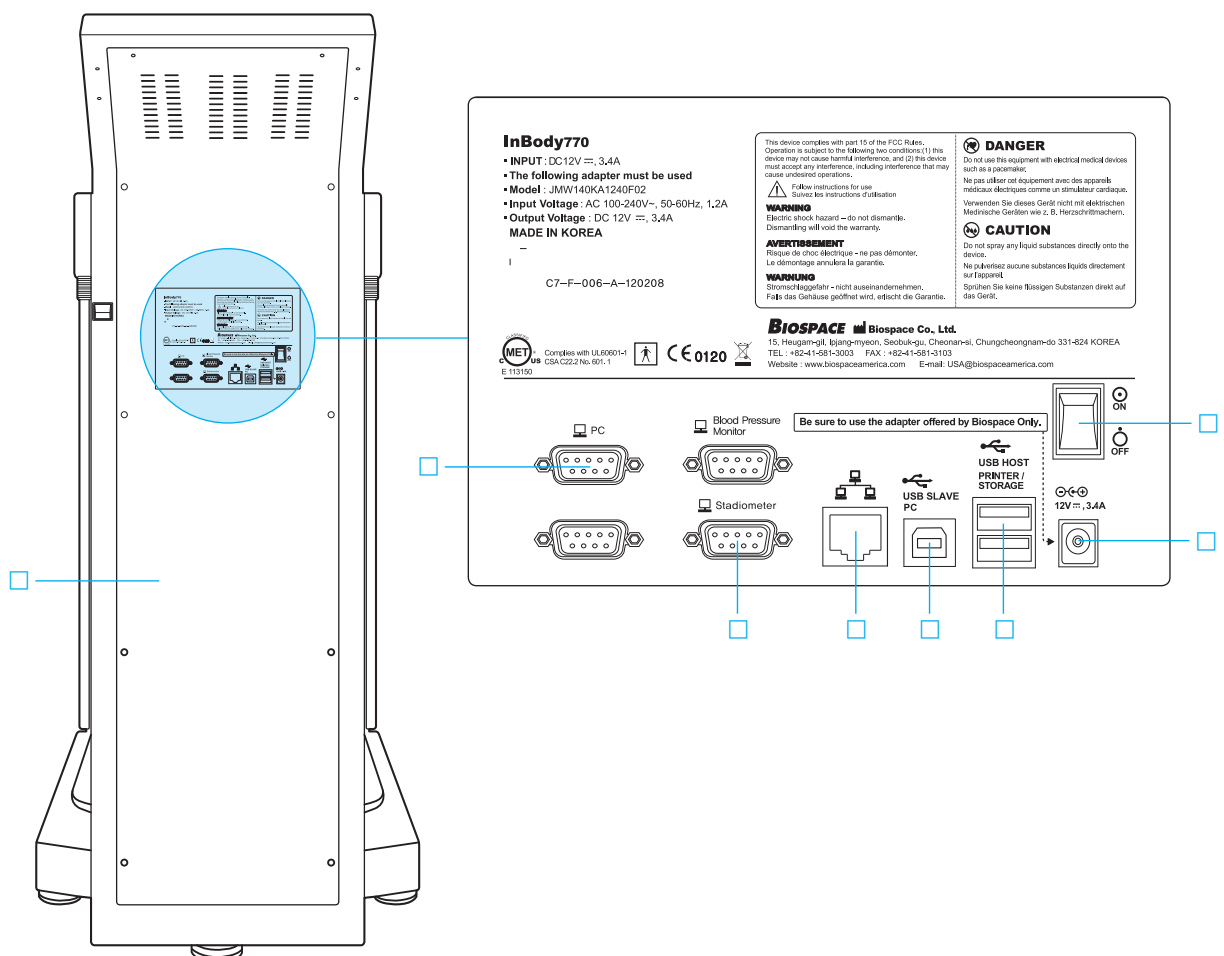- Rear cover: Designed to be opened only by InBody service personnel for checking internal circuits, etc.
- 9-pin PC serial port (Female, RS-232C): Used for connecting the InBody770 to Lookin'Body120 installed on the computer.
  - \* The InBody770 can be connected to Lookin'Body120 installed on a computer using one of the ports □, □, or □.
- 9-pin stadiometer serial port (Female, RS-232C): Used for connecting the InBody770 to a stadiometer.
  - \* Only compatible with a InBody stadiometer.
- LAN port (10T Base): Used for connecting the InBody770 to Lookin'Body120 installed on a computer.
  - \* The InBody770 can be connected to Lookin'Body120 installed on a computer using one of the ports □, □, or □.
- USB SLAVE port: Used for connecting the InBody770 to Lookin'Body120 installed on a computer.
  - \* The InBody770 can be connected to Lookin'Body120 installed on a computer using one of the ports □, □, or □.
- USB HOST port: Used for connecting to a printer or a USB Thumb Drive.
- Power input port: Used for connecting to the power adapter.
  - \* Always use the specified adapter provided by InBody as it is a part of the InBody770. Using other adapters may result in malfunction of the InBody770.
- Power switch: Used for turning on/off the equipment.




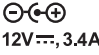




## B. Safety Information

### Indicators

	9-pin serial port (Female, RS-232C)
	LAN port (Ethernet, 10T Base)
	USB port

### Safety Symbols

	Dangerous High Voltage
	Warning, Caution
	BF Type Equipment
	Adapter
	Power On
	Power Off



Follow instructions for use  
Suivez les instructions d'utilisation

### WARNING

Electric shock hazard – do not dismantle.  
Dismantling will void the warranty.

### AVERTISSEMENT

Risque de choc électrique - ne pas démonter.  
Le démontage annulera la garantie.



### DANGER

Do not use this equipment with electrical medical device such as a pacemaker.  
Ne pas utiliser cet équipement avec des appareils médicaux électriques comme un stimulateur cardiaque.



### CAUTION

Do not spray any liquid substance directly onto the device.  
Ne pulvérisez aucune substances liquids directement sur l'appareil.



### CAUTION

No excessive force on shoulder joint  
Ne pas appliquer de force excessive sur les bars articulés.

## C. Classification

Body Composition Analyzer of Direct Segmental Multi-frequency Bioelectrical Impedance Analysis Method

- Types of protection against electric shock: Class I
- Type of the applied parts: BF Type
- EMC Immunity: Level A
- EMC Emission: CLASS A
- Degree of protection against water infiltration: IPX0

## D. Specifications

Bioelectrical Impedance Analysis (BIA) Measurement Items	Bioelectrical Impedance (Z)	30 Impedance Measurements by Using 6 Different Frequencies (1kHz, 5kHz, 50kHz, 250kHz, 500kHz, 1000kHz) at Each of 5 Segments (Right Arm, Left Arm, Trunk, Right Leg and Left Leg)
	Reactance (Xc)	15 Impedance Measurements by Using 3 Different Frequencies (5kHz, 50kHz, 250kHz,) at Each of 5 Segments (Right Arm, Left Arm, Trunk, Right Leg, and Left Leg)
Electrode Method	Tetrapolar 8-Point Tactile Electrodes	
Measurement Method	Direct Segmental Multi-frequency Bioelectrical Impedance Analysis Method, DSM-BIA Simultaneous Multi-frequency Impedance Measurement (SMFIM)	
Body Composition Calculation Method	No Empirical Estimation	
Outputs (Body Composition Results Sheet)	<p>Intracellular Water, Extracellular Water, Total Body Water, Dry Lean Mass, Lean Body Mass, Body Fat Mass, Weight, Skeletal Muscle Mass, Body Mass Index, Percent Body Fat, Segmental Lean Analysis (Right Arm, Left Arm, Trunk, Right Leg, Left Leg), Segmental ECW/TBW Analysis (Right Arm, Left Arm, Trunk, Right Leg, Left Leg), ECW/TBW, Body Composition History (Weight, Skeletal Muscle Mass, Percent Body Fat, ECW/TBW), Visceral Fat Area, Body Fat-Lean Body Mass Control, Segmental Fat Analysis (Right Arm, Left Arm, Trunk, Right Leg, Left Leg), Segmental Body Water Analysis (Right Arm, Left Arm, Trunk, Right Leg, Left Leg), Segmental ICW Analysis (Right Arm, Left Arm, Trunk, Right Leg, Left Leg), Segmental ECW Analysis (Right Arm, Left Arm, Trunk, Right Leg, Left Leg), Basal Metabolic Rate, Leg Lean Mass, Visceral Fat Level, Results Interpretation QR Code, TBW/LBM, Reactance (5kHz, 50kHz, 250kHz), Whole Body Phase Angle (50kHz), Impedance at each segment/frequency</p> <p>* Indications for use: For use only in healthy subjects for measurement of:</p> <ul style="list-style-type: none"> <li>• Estimated: Skeletal Muscle Mass, Extracellular Water (ECW), Intracellular Water (ICW), Total Body Water (TBW), ECW/TBW, Body Fat, Percentage of Body Fat (PBF), Body Lean + Dry Lean, Metabolic Rates(Basal Metabolic Rates), Segmental Lean Mass, Segmental Fat Mass, % Segmental Body Fat, Visceral Fat Area (VFA), Visceral Fat Level, Segmental Body Water, Percent Body Water, Weight Control, Fat Control, Muscle Control, Segmental ECW/TBW, Segmental ECW, Segmental ICW, TBW/LBM, Leg Lean Mass</li> <li>• Actual: Weight, Body Mass Index (BMI) and Impedance Values, Height, Resistance Values, Reactance Values, Phase Angle</li> </ul>	
Outputs (Body Water Results Sheet)	<p>Intracellular Water, Extracellular Water, Total Body Water, ECW/TBW, Segmental Body Water Analysis (Right Arm, Left Arm, Trunk, Right Leg, Left Leg), Segmental ECW/TBW Analysis (Right Arm, Left Arm, Trunk, Right Leg, Left Leg), Body Water Composition History (Weight, Total Body Water, Intracellular Water, Extracellular Water, ECW/TBW), Visceral Fat Area, Segmental ICW Analysis (Right Arm, Left Arm, Trunk, Right Leg, Left Leg),</p>	

Outputs (Body Water Results Sheet)	Segmental ECW Analysis (Right Arm, Left Arm, Trunk, Right Leg, Left Leg), Dry Lean Mass, Lean Body Mass, Body Fat Mass, Weight, Skeletal Muscle Mass, Body Mass Index, Percent Body Fat, Basal Metabolic Rate, Visceral Fat Level, Body Cell Mass, TBW/LBM, Results Interpretation QR Code, Reactance (5kHz, 50kHz, 250kHz), Whole Body Phase Angle (50kHz), Impedance at each segment/frequency	
Optional Equipment	Stadiometers from InBody	
Logo	Name, Address, and Contact Information can be shown on the InBody Results Sheet.	
Digital Results	LCD Monitor, Data management software Lookin'Body120	
Results Sheet	Body Composition Results Sheet, Body Water Results Sheet	
Voice Guidance	Provides audible indication for test in progress, test complete, and successfully saved settings changes.	
Database	Test results can be saved if the member ID is utilized. The InBody can save up to 100,000 results.	
Test Mode	Self Mode, Professional Mode	
Administrator Menu	Settings: Configure settings and manage data Troubleshooting: Additional information to help use the InBody770	
USB Thumb Drive	Copy, backup, or restore the InBody770 data (data can be viewed on Excel or Lookin'Body120 data management software)	
Barcode Reader	The member ID will be automatically inputted when the barcode ID is scanned.	
Applied Rating Current	80 $\mu$ A( $\pm$ 10 $\mu$ A)	
Adapter	Power Input	AC 100 ~ 240V, 50/60Hz, 1.2A
	Power Output	DC 12V, 3.4A
Display Type	800 × 480 10.2 inch Color TFT LCD	
Internal Interface	Touchscreen, Keypad	
External Interface	RS-232C 4EA, USB HOST 2EA, USB SLAVE 1EA, LAN (Ethernet, 10T) 1EA, Bluetooth 1EA, Wi-Fi 1EA	
Compatible Printer	Laser/Inkjet Printers * A list of printers compatible with the InBody770 can be found at <a href="http://www.inbodyservice.com">http://www.inbodyservice.com</a>	
Dimension	20.7(W) × 33.6(L) × 46.3(H): inch 526(W) × 854(L) × 1175(H): mm	
Equipment Weight	83.8 lbs (38 kg)	
Test Time	About 50 seconds	
Operation Environment	50 ~ 104°F (10 ~ 40°C), 30 ~ 75% RH, 70 ~ 106 kPa	
Storage Environment	4 ~ 158°F (-10 ~ 70°C), 10 ~ 80% RH, 50 ~ 106 kPa (No Condensation)	
Testing Weight Range	22.0 ~ 595 lbs (10 ~ 270 kg)	
Testing Age Range	3 ~ 99 years	
Testing Height Range	3 ft. 1.4 in. ~ 7 ft. 2.6 in. (95 ~ 220 cm)	

\* Specifications can be changed without a prior notice.

[www.inbodyusa.com](http://www.inbodyusa.com)