

Prestan Professional Manikins Replacing the Prestan Adult CPR Rate Monitor

To replace the CPR Rate Monitor (Monitor) on the Adult Manikin, you will need a small flat-head screwdriver and a small phillips-head screwdriver. To facilitate the instruction procedure, please note the three parts shown at right:





Remove the Wire Cover by gently inserting the flat-head screwdriver into the side slots and lifting upward until the cover easily pops off. Disconnect the Monitor from the Clicker by squeezing, then pulling apart the two wires at the connection.









Turn the manikin over and open the clamshell torso to expose the Monitor's lights. Using the phillipshead screwdriver, remove the front screw holding the Monitor in place. The Monitor will drop out of the backside of the manikin.





















