

## The California Bander

The "California Bander" is our castration tool. It is a simple, clean, and a cost-effective tool for cattlemen and veterinarians across the nation.

### How to Use the California Bander

Roll up a towel to simulate the scrotal sack. The bands arrive pre-assembled and are easy to use. To use the California Bander:



Begin By Slipping the Left Hand In the Tool.



With the Right Hand, Place the Clip Of the Band Into the Slot Of the Tool.



Hold the Free End of the Band Between the First and Second Fingers of the Left Hand. Grasp the Scrotal Sac and Hold the Tool Firmly Against It.



With Your Right Hand, Reach Around the Scrotal Sac and Grab the Free End of the Rubber Tubing. Stretch the Band a Little Downward and Away From Yourself With About 14 to 17 Pounds Of Pull Or Until There Is No More Free Stretch.



Pull the Stretched Band Around the Scrotal Sac.



Lock the Band Into the Clip By Pulling the Stretched Band Down the Center Slot All the Way To the Bottom. When the Clip Is Down, Pull the Band to the Right Corner Of the "T" Slot On the Clip to Lock It. Be Sure to Keep the Band Fully Stretched As You Pull It Around.



Release the Clip From the Tool By Slightly Tipping It Backward. This Will Leave the Band With the Clip Attached On the Scrotal Sac. The Rubber Tube Does Not Need to Come Off. If You Can Fit Your Finger In Between the Band And the Scrotal Sac, You Do Not Have the Band Tight Enough.