

Premium Dental Instrumentation



HESIRA_M

HesiraMed.com | info@hesiramed.com | 844-HESIRA-1





DURABLE SILICONE RAILS:

safeguard instruments from

Specifically designed to

STERILIZATION CASSETTES

Sterilization and infection control prevention are optimized when using instrument sterilization cassettes that have been designed for safety and improved efficiency. Additionally, instrument integrity is preserved as clinicians and staff can move instruments from cleaning through chairside without touching or risking damage to the instruments.

Chairside: Patient Preview & Presentation

- Systematically organizes instruments according to procedure type, allowing for more focus on patients and less time looking for missing instruments
- Neatly organized instruments present a more professional appearance and can enhance referrals

Cleaning: Processed & Prepped

 Instruments are kept together throughout cleaning, rinsing and drying, reducing the potential for breakage or loss

 Allows for less handling of instruments reducing the chance for sharps injuries

Sterilization: Wrapped & Ready

 Cassettes with instruments are packaged and placed in autoclave together to help reduce consumable usage

 Reduced handling of instruments provides safer protocol and minimizes liability

Storage: Neat & Complete

 Ready for use or storage until needed for increased time savings

 Complete procedural set-ups for easy prep and teardown

> FUNCTIONAL ACCESSORY AREA: Separate accessory area to accommodate additional items on CAS6001 and CAS8001

> > SMOOTH ROUND CORNERS AND SLOTTED EDGES: Increases drainage and reduces drying time

EFFICIENT HOLE PATTERN: Protects instruments from protrusion while allowing for enhanced cleaning and sterilization POSITIVE LOCKING MECHANISM: Provides a visual indication that cassette is locked for safety



ELECTROPOLISH:

Provides a protective sheet

against corrosion and a



CAS2001

Small Exam Cassette - Holds 5 instruments Exterior Dimensions 3" x 8" x 1.25" (76mm x 203mm x 32mm)



CAS4001

Medium Exam Cassette - Holds 10 instruments Exterior Dimensions $5.5'' \times 8'' \times 1.25''$ (140mm \times 203mm \times 32mm)



Large Cassette - Holds 16 instruments, 2 hinged instruments in the lid and a separate accessory area Exterior Dimensions 8" \times 11" \times 1.25" (203mm \times 280mm \times 32mm)

Extra Large Oral Surgery Cassette - Holds 24 instruments
Easily accommodates 12 instruments, 4 elevators, 3 hinged instruments
in the lid and a separate accessory area
Exterior Dimensions 8" x 14.5 x 1.25" (203mm x 368mm x 32mm)
Fits most 10" chambers or larger sterilizers, most instrument
washers and larger ultrasonics



PROCEDURAL KITS

- Offers legendary Karl Schumacher instrument quality and consistency in a comprehensive procedural kit at a lower cost than if purchased individually
- Based on recommendations of leading dentists
- Organized in our new line of custom stainless steel sterilization cassettes
- Complete listing of components in each kit is available at HesiraMed.com/collections/instruments
- Save 15-30% on when purchasing a complete Procedural Kit instead of as individual instruments

PEL0715

The Micro Surgery Kit #402 contains 7 of the most popular micro surgery instruments, which are safely and conveniently stored in a medium cassette to help prevent damage to their precision working ends.

BCH0013L	Kirkland Perio Chisel #13L
BCH3637	Rhodes Perio Chisel
CAS4001	Medium Cassette
MTF0001	Micro Cooley Forceps
NEE0012T	Twist Joint Castro-Viejo Needle Holder
PEL0622	Bruno/Gargiulo Periosteal Elevator
SCI0917	Castro-Viejo Scissors

Molt 9 Periosteal Elevator

RAL SURGERY

MICRO SURGERY



The Oral Surgery Kit #801 contains 24 of the most important instruments to successfully perform most any oral surgery procedure all in an extra large cassette.

KIT0801

ASP0017T 3.0mm Aspiration Tip BCU0709S Serrated Lucas Surgical Curette, Small BCU0711S Serrated Lucas Surgical Curette, Large Miller #1 Bone File BFI0021 Extra Large Cassette CAS8001 CPL0561 Cotton Pliers #561 **ELE0034S** Medium Straight Elevator #77R Back Action Elevator ELEO077R ELE0301 Small Straight Elevator EXE1174 Lower Universal Apical Forceps, Ash Pattern Modified Lower Cowhorn Forceps EXF0023CR EXF0150AB AB Upper Universal Forceps EXF0151AB AB Lower Universal Forceps HDL0490C Cone Socket Mirror Handle #490 Bard-Parker Scalpel Blade Handle Curved Kelly Haemostat HDL0605 HEM0611 #5 Cone Socket Mirror MIRO005CF Crile-Wood Needle Holder NEE0690 Seldin Flap Retractor PEL0623D Molt 9 Periosteal Elevator PEL0715 PPR0815 UNC Perio-Probe / #23 Explorer RET0808 Minnesota Retractor RON0003 Blumenthal Rongeur SCI0659SC Super-Cut Kelly Scissors TIF0804T Adson/Brown Tissue Forceps, 2x8 Row Teeth

TUNNELING

All the instruments needed for optimum gingival enhancement are contained in the Tunneling Kit #605. The large cassette contains 9 instruments, with additional room for other preferred instrumentation.

CAS6001 Large Cassette

HDL0509 Micro Scalpel Blade Handle

NEE0012T Twist Joint Castro-Viejo Needle Holder

PEL0644 Buser Periosteal Elevator SCI0677SC Super-Cut LaGrange Scissors TIF0016 Suture Forceps, 1.6mm

TIF0804 Adson/Brown Tissue Forceps, 2x8 Teeth
TUN0001 Maxillary Tunneling Instrument
TUN0002 Mandibular Tunneling Instrument
TUN0012 Combo Tunneling Instrument



BASIC SINUS LIFT



The Basic Sinus Lift Kit #607 includes a selection of 7 sinus lift curettes and grafting instruments housed in a large cassette, with room for additional specialty instrumentation.

41.848.01 Sinus Lift Currette #1 41.848.02 Sinus Lift Currette #2 41.848.03 Sinus Lift Currette #3 41.848.05 Sinus Lift Currette #5 Sinus Lift Currette #51 41.848.51 3.5mm Graft Syringe BGS0713 CAS6001 Large Cassette Bone Condenser #918 ICS918

SOFT TISSUE

Soft tissue surgeries can easily be performed with the collection of 12 instruments contained in the Soft Tissue Kit #602. Housed in our large cassette, there is additional space available to include some of your preferred instruments.

BCH0013 Kirkland Perio Chisel #13

CAS6001 Large Cassette

CPL0018L Locking Cotton Pliers #18L
HDL0428 Drop Blade Scalpel Handle
HDL8007C Cone Socket EFM Mirror Handle

MIR0005CF #5 Cone Socket Mirror MOC0024 Molt Curette 2/4

NEE0690 Crile-Wood Needle Holder
PF10940 Sugarman 3/4 Perio File
PKN0799 Goldman-Fox #11 Perio Knife
PPR6806 Schumacher Perio-Probe/#23 Explorer

SCA0267 Younger-Good 7/8 Curette SCI0908 Goldman-Fox Scissors



POST OPERATIVE KIT

Quickly remove sutures and evaluate your surgical results with the 6 instruments in the Post Operative Kit #401, conveniently organized in a medium cassette.

CAS4001 Medium Cassette CPL0561 Cotton Pliers #561 EXP0006 Explorer #6

EXP1305N Explorer 05N (#23/#17)

Suture Scissors SCI0909

Adson/Brown Tissue Forceps, Serrated TIF0804S

TIF0963 Tissue Forceps w/TC Insert



KIT0401

ORAL PLASTIC SURGERY



The Oral Plastic Surgery Kit #606 features a large cassette storing 15 of the most common soft tissue instruments needed for aesthetically focused periodontal surgery.

85.251.00 Mixing Cup w/Lid CAR0601A IPC Carver #601 Large Cassette CAS6001

HDL0509 Micro Scalpel Blade Handle HDL8007C Cone Socket EFM Mirror Handle #5 Cone Socket Mirror MIR0005CF

MTF0001 Micro Cooley Forceps

Twist Joint Castro-Viejo Needle Holder Pritchard Periosteal Elevator NEE0012T PEL0624

Mini Buser Periosteal Elevator PEL0643 PKN0929 Orban 1/2 Perio Knife PPR0807 Schumacher Perio-Probe, Color Younger-Good 7/8 Curette SCA0267 Super-Cut Goldman-Fox Scissors SCI0908SC

SYR0001 Aspirating Syringe TIF0005DP Corn Suture Forceps

IMPLANT ESSENTIALS

Dr. Jay Reznick's Essentials of Implantology Kit #608 is a setup of 15 "must have" surgical instruments for anyone just starting to place implants in a practice.

85.251.00 Mixing Cup w/Lid 3.0mm Aspiration Tip Large Cassette ASP0007 CAS6001 Bard-Parker Scalpel Blade Handle Molt Curette 2/4 HDL0605 MOC0024 MOP0000K Silicone Bite Block Set Mosquito Forceps, 1x2 Teeth Crile-Wood Needle Holder MOS0607 NEE0690 PEL0715 Molt 9 Periosteal Elevator RET0808 Minnesota Retractor Curved Wieder Tongue Retractor RET0820C Kelly Scissors, Straight Metzenbaum Scissors, Pointed Aspirating Syringe SCI0375 SCI0667S SYR0001 TIF0805 Adson/Brown Tissue Forceps, 1x2 Teeth



3RD MOLAR EXTRACTION KIT



Dr. Nekky Jamal's 3rd Molar Extraction Kit #614 is for those beginning to extract third molars.

NEE0791	Mayo-Hegar Needle Holder
SCI0659	Kelly Scissors
RET0808	Minnesota Retractor
BCU0711S	Medium Serrated Lucas
HDL0428	Drop Blade Scalpel Handle
PELO715	Molt #9 Periosteal Elevator
ELE7660K	Dr. Koerner Modified Cogswell B Elevator
BFI0021	Miller #1 Bone File
LUX0003CT	Luxating Elevator, #3C, 3mm Curved
ELEO001P	T-Bar Potts Elevator, Angled Right
HEM0611	Kelly Hemostat
ELEO002P	T-Bar Potts Elevator, Angled Left
RET0820A	Large Wieder Retractor
CAS6002	Large Cassette for Dr. Jamal 3rd Molar Extraction Kit

IMPLANT SITE DEVELOPMENT

Dr. Suheil Boutros' Implant Site Development Kit #603 is a collection of 11 instruments to help atraumatically extract and then graft a site in preparation for the placement of an implant.

17.008.01 Small Distal Proximator® 17.008.02 Small Mesial Proximator® BCU0709S Serrated Lucas Surgical Curette, Small 3.5mm Graft Syringe Large Cassette BGS0713 CAS6001 Upper Universal Deep Grip Forceps Lower Universal Deep Grip Forceps EXX0307 EXX0321 Bone Condenser #12 IDR0012 NEE0690 Crile-Wood Needle Holder PEL0643 Mini Buser Periosteal Elevator SCI0665SC Super-Cur Iris Scissors

Suture Forceps, 1.6mm

TIF0016



BONE GRAFTING



The Bone Grafting Kit #601 is the ultimate collection of 16 instruments specific to preparing and placing graft material within the oral cavity, anywhere from single extraction sites to bilateral sinus lift procedures.

41.868.07 BCH0035 BCU0008 CAS6001 CPL0002 HDL0428 ICS918 MTF0001 NEE0690 PEL0624 PEL0644 PFI0039T PKN0799 PPR0807 SCA0270 85.251.00	Kirsch Sinus Plugger / Spoon Buser Bone Harvesting Chisel Bone Curette #8 Large Cassette Cotton Pliers #2 Drop Blade Scalpel Handle Bone Condenser #918 Micro Cooley Forceps Crile-Wood Needle Holder Pritchard Periosteal Elevator Buser Periosteal Elevator Composite Instrument #39T Goldman-Fox #11 Perio Knife Schumacher Perio-Probe, Color Columbia 13/14 Curette Mixing Cup w/ Lid
SCI0908	Goldman-Fox Scissors

AETRANOX™ PROXIMATORS®

Proximators are used for severing the ligaments in the periodontal space. This instrument combines the sharp, thin working-edge of the Periotome and the strength of the Luxating Elevator. The result is the creation of the ultimate atraumatic extraction instrument, limiting the need for traditional elevators that could lead to unwanted forces on the surrounding bone structure.

The Aetranox™ version features proprietary Aluminum Titanium Nitride coating that is significantly harder, scratch resistant and non-reflective for long lasting performance.

- Used at the beginning of any extraction
- Faster and easier to utilize than traditional Periotomes
- Inserted vertically into the PDL space, parallel to the long axis of the root
- Sever the periodontal ligament and expand the alveolus laterally towards the mesial or distal
- Eliminate the need for traditional elevation and greatly reduce rotational luxation with extraction forceps
- Create a larger path of delivery for the root structure while at the same time protecting the fragile buccal wall
- Feature a razor thin leading edge to aid in insertion where a root is broken down at or below the gumline
- Also available in Stainless Steel

PROXSPADEAE

Set of 5 Aetranox Edition Spade Proximators





The pointed tip of the Spade Proximators easily enters the PDL space to incise the fibers.



Unique designs like the distally angled versions allow for superior access throughout the mouth.



In many cases the root can be loosened to such an extent that it can be retrieved with haemostats or similar instruments.

Surgical photos courtesy of Dr. Jay Reznick, DMD., M.D., Tarzana, CA



PROXIMATORS®



Proximator Sharpening Stone, 1cm x 10cm Long

PROX7

Full Set of 7 Proximators incl. Small Straight, Small Curved, Sharpened Spade, Small Mesial, Small Distal, Large Mesial, Large Distal, Sharpening Stone and Cassette





Small Mesially Angled Proximator, 2.5mm, Purple

PROX4

Small Set of 4 Proximators incl. Small Straight, Small Curved, Small Mesial, Small Distal, Sharpening Stone and Cassette





PROXSPADE

Set of 5 Spade Proximators in Stainless Steel

PERIOTOMES

Periotomes are the original, patented instruments for severing the ligaments in the periodontal space. This instrument features a very thin and flexible blade that is sharpened along the entire edge.

The Periotomes are the finest and least traumatic extraction instrument available. When applied properly, they can completely sever the periodontal ligament without the need for further instrumentation.

- Used at the beginning of any extraction
- Flexible to follow the curvature of the root
- Feature a sharpened edge along the entire radius
- Thin enough to apply along the fragile buccal wall
- Most conservative ligament incising instruments available
- Sever the periodontal ligament without expansion to the alveolus
- Available in three configurations; Anterior, Posterior and Perpendicular
- Inserted vertically into the PDL space, parallel to the long axis of the root
- Titanium nitride coated for durability and edge retention; they never need sharpening
- Reduce the need for traditional elevation and reduce rotational luxation with extraction forceps



The conservative design of the Periotomes yield the most atraumatic result of any instrument.





Periotomes thin design allows them to be applied on the fragile facial aspect of anteriors without damage.

ROOT EJECTORS



Handform Root Ejector #634, Mesial/Distal Curved Ends

HEIDBRINK ELEVATORS



Heidbrink Root Tip Elevator #760-2, Angled Right, Curved, Pointed



Heidbrink Root Tip Elevator #760-3, Angled Left, Curved Tip, Pointed



ELE7601

Heidbrink Root Tip Elevator #7601, Straight, Flat Tip, Pointed



ELE7602N

Heidbrink Root Tip Elevator #7602N, Angled Right, Flat Tip, Pointed



ELE7603N

Heidbrink Root Tip Elevator #7603N, Angled Left, Flat Tip, Pointed

ROOT ELEVATORS

SPADE ROOT ELEVATORS



ELEO060M

Bernard Mesial Spade Root Tip Elevator #60M



ELEO060R

Bernard Right Spade Root Tip Elevator #60R



ELE0063

Bernard Curved Spade Root Tip Elevator #63



ELE0060L


Bernard Left Spade Root Tip Elevator #60L



ELEO060D

Bernard Distal Spade Root Tip Elevator #60D



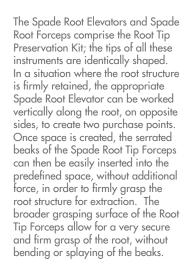



SPADE ROOT FORCEPS





EXF0145E Spade Lower Root Forceps #145E, Serrated

STANDARD ROOT TIP FORCEPS

Narrow, Serrated Tips Reach to the Apex



Upper Root Tip Forceps #144S, Curved Tip, Serrated



Lower Root Tip Forceps #145G, Serrated

STRAIGHT ELEVATORS



3.5mm Straight Elevator #34S





ELE0034

4.5mm Straight Elevator #34



2.5mm Straight Elevator #301



ELE0010

2.75mm Straight Elevator #10, Serrated



ELE0012

3mm Straight Elevator #12, Pointed Tip



Any of our Elevators come serrated at no additional charge. Just be sure to ask when you place your order.

POSTERIOR FLARED-TIP ELEVATORS



ELEO046R

2.5mm Bayonet Elevator #46R




ELE0077

 $3.75 \mathrm{mm}$ Flared Tip Distal Angled Elevator #77, Serrated

There's a good reason why the ELEO077R is the most popular Elevator we carry. The thin, serrated tip engages securely onto the mesial aspects of posterior teeth without stretching the patient's cheek. During application the tip 'pushes' the root structures up and out of the socket along the natural distally curving arc of the roots.

STRAIGHT FLARED-TIP ELEVATORS



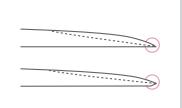

All Karl Schumacher Elevators are sharpened by finishing the gouge 'from the top down,' resulting in a very durable, thin and sharp leading edge. Most other manufacturers sharpen their Elevators by bevelling the bottom of the leading edge, resulting in a very sudden taper into a thick tip. It takes more skill to finish our elevators this way, but we know it makes a difference.

CRANE PICK AND COGSWELL ELEVATORS






BECK MODIFIED ELEVATORS




3.5mm Straight Beck Elevator #202, Tri-Bevel Back Surface



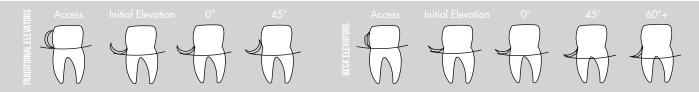

Traditional elevators have a rounded backside, which allows the tip to rotate like a wheel. This only allows engagement and lift for the first 45° of elevation, at which point the edge will lose purchase from the tooth. Each bevel on the back of the Beck Elevators acts as its own fulcrum point, allowing the Beck Elevators continuous engagement past the first 45° of rotation. This special sharpening, coupled with a thin tip for access into tight interproximal areas, make the Beck Elevators extremely effective instruments for elevation, but they must only be used by rotating, not prying them.

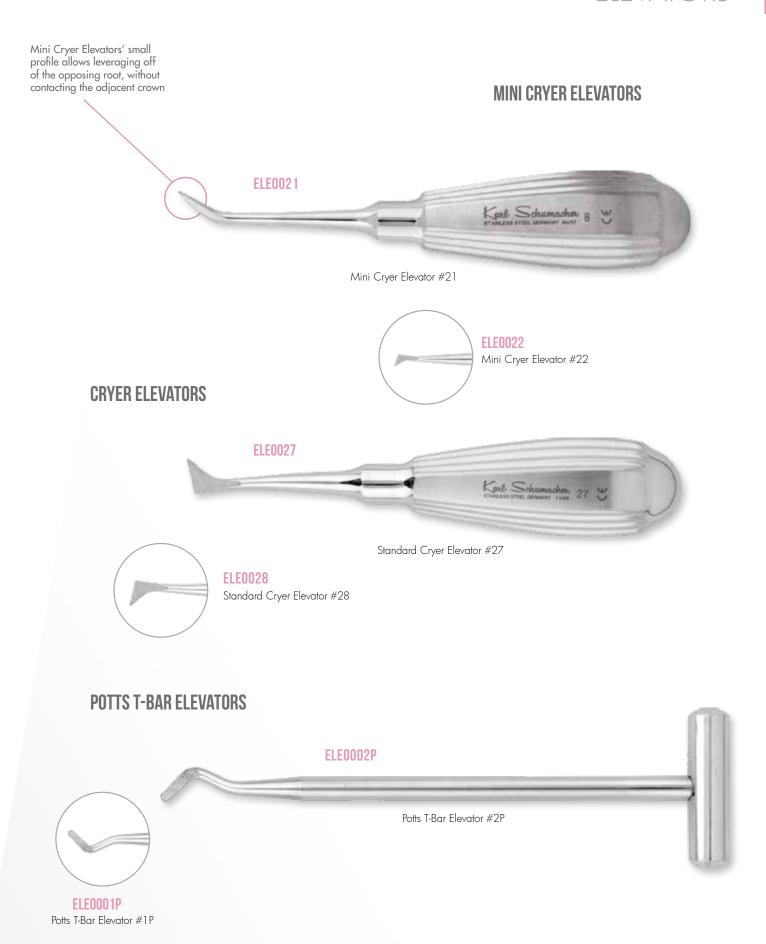

LUXATING ELEVATORS

TINI COATED LUXATING ELEVATORS

• Hardened TiNi coating eliminates the need to sharpen



LUX0005ST

5.0mm TiNi Coated Straight Luxating Elevator #5ST



LUX0005CT

5.0mm TiNi Coated Curved Luxating Elevator #5CT

LUXKIT1T

Set of 4 TiNi Coated Luxating Elevators



LUX0003CT

3.0mm TiNi Coated Curved Luxating Elevator #3CT



LUX0003ST

3.0mm TiNi Coated Straight Luxating Elevator #3ST

STANDARD LUXATING ELEVATORS

- Hone and refine the edges as needed
- Will become thinner with continued sharpening



LUX003S

3.0mm Straight Luxating Elevator #3S



LUX003C

3.0mm Curved Luxating Elevator #3C



Set of 4 Luxating Elevators w/Arkansas Sharpening Stone



LUX005S

5.0mm Straight Luxating Elevator #5S 5.0mm Curved Luxating Elevator #5C

BEIN STYLE LUXATING ELEVATORS

- Hone and refine the edges as needed
- Shorter length handles give better control



LUX0040

2.5mm Curved Bein Handle Luxating Elevator #40



LUX0030

2.5mm Straight Bein Handle Luxating Elevator #30

LUXKIT2

Set of 6 Bein Handle Luxating Elevators w/ Arkansas Sharpening Stone



LUX0041

3.5mm Curved Bein Handle Luxating Elevator #41



3.5mm Straight Bein Handle Luxating Elevator #31



LUX0042

4.5mm Curved Bein Handle Luxating Elevator #42



LUX0032

4.5mm Straight Bein Handle Luxating Elevator #32

LUXATING ELEVATORS

FLEX-EX LIGAMENT KNIFE

- Thin sharpened tip
- Flexible tip follows root curvature without deforming
- Single-ended design allows for greater application of force
- Can be held in modified pen grasp or palm driven

OS STYLE LUXATING ELEVATORS

- Sharpened leading edge
- Left and right configurations for posterior access
- Can be sharpened and honed
- Solid handle design



DIAMOND PrecisGrip FORCEPS

DIAMOND PrecisGrip FORCEPS

The Diamond Edition PrecisGrip Forceps feature custom diamond coated tips that provide a radically superior grip in all applications, including wet conditions.

Increased friction from the diamond particles ensures better engagement and reduces the potential for fracture, shortening operative time.

- Large diamond particles bonded into the beak surface
- A more secure purchase than serrations alone
- Positive engagement with less grip-strength
- Parallel gripping surface spreads forces along the tooth
- Thin leading edges for subgingival engagement
- Patterns for the entire maxillary and mandibular arch
- Spade patterns for fractured roots

After hand finishing the beaks, a precise application of the diamond coating ensures the even distribution of the diamond particles along the gripping surface inside. Whether it is a wet or dry situation, the diamond coating significantly improves the engagement of contacts, and in combination with the PrecisGrip beak design, makes for the best gripping extraction forceps.



PGFSETDI
Full Set of 6 Diamond
PrecisGrip Forceps





Diamond Tipped Lower Anterior PrecisGrip Forceps #2074DI



EXR2002DI

Diamond Tipped Upper Anterior
PrecisGrip Forceps #2002DI

DIAMOND PrecisGrip FORCEPS



EXX0145ED

Diamond Tipped Lower Spade Root PrecisGrip Forceps #145ED



EXR2021DI

Diamond Tipped Lower Universal PrecísGrip Forceps #2021DI



Diamond Tipped Upper Spade Root PrecisGrip Forceps #1251ED



EXR2067DI

Diamond Tipped Universal Upper Molar PrecisGrip Forceps #2067DI



EXR2079DI

Diamond Tipped Lower Molar PrecísGrip Forceps #2079DI



Diamond Tipped Upper Universal PrecisGrip Forceps #2007DI

AETRANOX™ EDITION PrecísGrip FORCEPS

AETRANOX™

The Aetranox Edition PrecisGrip Forceps feature proprietary Aluminum-Titanium-Nitride coating that is significantly harder, scratch-resistant and non-reflective for long-lasting performance.

Increased hardness created by the Aetranox surface treatment ensures long-term integrity of both the beak serrations and forceps overall.

- Specially formulated finish is a surface-reinforcing matrix
- Vapour deposited blend that is covalently bonded to the stainless steel, not plated onto the surface
- Reaction at the molecular level increases the durability and ensuring long-term performance of the serrated beaks
- Extremely smooth surface seals the pores of the stainless steel, minimizing protein adhesion and reducing time to clean
- Matte finish that reduces distracting glare and reflections mid-procedure
- Available in 6 traditional designs and 2 root patterns



EXX0145EAE

Aetranox™ Spade Lower Root PrecísGrip Forceps #145EAE, Serrated



EXX1251EAE

Aetranox™ Spade Upper Root PrecísGrip Forceps #125EAE, Serrated



EXR2074AE

Aetranox™ Lower Anterior PrecísGrip Forceps #2074AE



EXHZUUZAE

Aetranox™ Upper Anterior PrecísGrip Forceps #2002AE

AETRANOX™ EDITION PrecísGrip FORCEPS



Aetranox™ Upper Universal PrecísGrip Forceps #2007AE



Aetranox™ Lower Universal PrecisGrip Forceps #2021AE



EXR2067AEAetranox[™] Universal Upper Molar PrecísGrip Forceps #2067AE



EXR2079AE

Aetranox™ Lower Molar

PrecisGrip Forceps #2079AE



PGFSETAEFull Set of 6 Aetranox™
PrecisGrip Forceps

PrecisGrip EXTRACTION FORCEPS

PrecisGrip FORCEPS

- Anatomically correct
- Prevent root fracture
- Parallel gripping beak
- Super heavy edge serrations
- Available in nine forceps designs
- Contact both the crown and root structure
- Long axis serrations within the beaks
- Sharpened beaks to improve subgingival access
- Used on intact, partially intact or broken down teeth



are the next generation in atraumatic extraction forceps. Combining all the best attributes of the Apical Retention Forceps and the deep, heavy serrations of the Kratzman Elevators, creates the ultimate extraction forceps. The parallel closing and serrated beaks feature a sharpened, tapered outer profile to increase subgingival access and contacts along the entire tooth structure. Once subgingival, the heavy cross serrations apply extra contact to prevent slipping during luxation. Exclusively available on the Xcision Hygine Critical Handles, the PrecisGrip Extraction Forceps comes in a variety of patterns.



EXR2007
PrecisGrip Upper Anterior
Forceps #2002



EXR2074
PrecisGrip Lower Anterior
Forceps #2074



EXR2021
PrecisGrip Lower Universal
Forceps #2021



EXR2067 PrecísGrip Universal Upper Molar Forceps #2067



EXR2079
PrecisGrip Lower Molar
Forceps #2079



PGFSET6 Full Set of 6 PrecisGrip Extraction Forceps



APICAL RETENTION FORCEPS

Apical Retention Forceps™ are the original Karl Schumacher atraumatic extraction forceps. The complete set of six was designed with five major mechanical advantages in order to eliminate the basic problems of minor, supra-gingival contact that traditional forceps provide.

With parallel beaks that are anatomically correct, the Apical Retention Forceps^{tom} are able to achieve greater surface contact along the tooth, while the sharpened and tapered outer profile penetrates deep into the alveolus in order to make secure purchase on the root structure. The precision cut longitudal serrations within the beaks stay engaged during axial rotation, for a confident and secure grip of the tooth and root throughout the procedure.



EXE1174NX

Extra Narrow Lower Anterior Apical Retention Forceps #1174NX

APICAL RETENTION FORCEPS™

- Anatomically correct
- Prevent root fracture
- Parallel gripping beak
- The original atraumatic forceps
- Available in all six standard forceps designs
- Contact both the crown and root structure
- Aggressive long axis serrations within the beaks
- Sharpened beaks to improve subgingival access

• Used on intact, partially intact or broken down teeth



EXF1102

Upper Anterior Apical Retention Forceps #1102



EXF1121

Lower Universal Apical Retention Forceps #1121



EXF1167

Universal Upper Molar Apical Retention Forceps #1167



EXF1179

Lower Molar Apical Retention Forceps #1179







Upper Universal Apical Retention Forceps #1107

DEEP GRIP FORCEPS

DEEP GRIP FORCEPS

- Parallel gripping beak
- Aggressive edge serrations
- Long axis serrations within the beaks
- Thick beaks for superior penetration
- Contact the root and remaining coronal structure

Used on partially intact, broken down or sectioned teeth



EXF0302

Upper Anterior Deep Grip Forceps #302



Upper Universal Deep Grip Forceps #307



EXF0321 Lower Universal Deep Grip Forceps #321



EXF0351 Upper Molar Deep Grip Forceps #351



Deep Grip Forceps are designed to make penetration into the alveolus in the case of severely broken down teeth or multi-rooted teeth which have been sectioned. This is a parallel, cross-serrated beak design with a sharp notch to enter the socket. The Deep Grip beak design is offered on an array of patterns configured to access different areas of the mouth.

AB/AS EXTRACTION FORCEPS

AB/AS FORCEPS

- Parallel gripping beak
- Lateral edge serrations
- Best coronal purchase
- Height of contour relief cut-out
- Heavy beaks for secure purchase
- Long axis serrations within the beaks
- Used on intact or partially intact teeth
- Available in all six standard forceps designs



Lower Molar 'AB/AS' Forceps #222AS



EXF0151AB

Lower Universal 'AB/AS'



superior to the gumline. The open tipped, parallel beaks include cross-serrations on the first 5mm of the tips and a cut-away to adapt to the height of contour, which helps prevent fracture of the often broken down and brittle crown. The AB/AS beak design is offered on an array of patterns, including Anterior, Molar and Pediatric styles, which will address

any extraction presentation.

AB/AS forceps are designed

to make maximum purchase on the remaining crown structure,



Forceps #151AB



EXFO010AS

Upper Molar 'AB/AS' Forceps #10AS



PEDIATRIC SIZE



EXF0151AS

Pediatric Lower Universal 'AB/AS' Forceps #151AS



EXF0150AB

Upper Universal 'AB/AS' Forceps #150AB



EXF0150AS

Pediatric Upper Universal 'AB/AS' Forceps #150AS



707

Forceps #33AS

MOLAR EXTRACTION FORCEPS


Upper Molars



Upper Right Molar Cowhorn Forceps #1267R

PA

Lower Molars

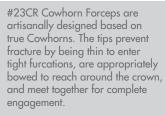



EXF1267L Upper Left Molar Cowhorn Forceps #1 267R

1267's vs. 88's





#1267 Upper Cowhorn
Furcation Forceps is
compact in design
based off the traditional
#88 style forceps.
This allows for deeper
penetration into the
furcation while also
reducing bone loss. In
addition, the #1267's
come with a serrated
palatal beak for a more
secure purchase.



EXF0023CR
'CR' Modified Lower Molar Cowhorn
Forceps #23CR



Lower Molar Modified Cowhorn Forceps #79AS



#79AS Forceps combine Cowhorn and Standard Forceps tip designs. They enter the furcation to lift the tooth from below and provide coronal contact for security and stability during application.

STANDARD EXTRACTION FORCEPS

Karl Schumacher's desire for the providing only the highest quality instrumentation can be found in all of the entire line, including Standard Style Extraction Forceps. Starting with the best materials and hand-finished by the industry's best craftsman, these forceps will meet or exceed your expectations.



EXF0023 Standard Lower Molar Cowhorn Forceps #23



EXF0150 Standard Cryer Upper Universal Forceps #150




EXF0151 Standard Cryer Lower Universal Forceps #151

SURGICAL INSTRUMENTATION

ASPIRATORS

Please go to HesiraMed.com/collections/instruments for our complete selection of aspirators and related products.

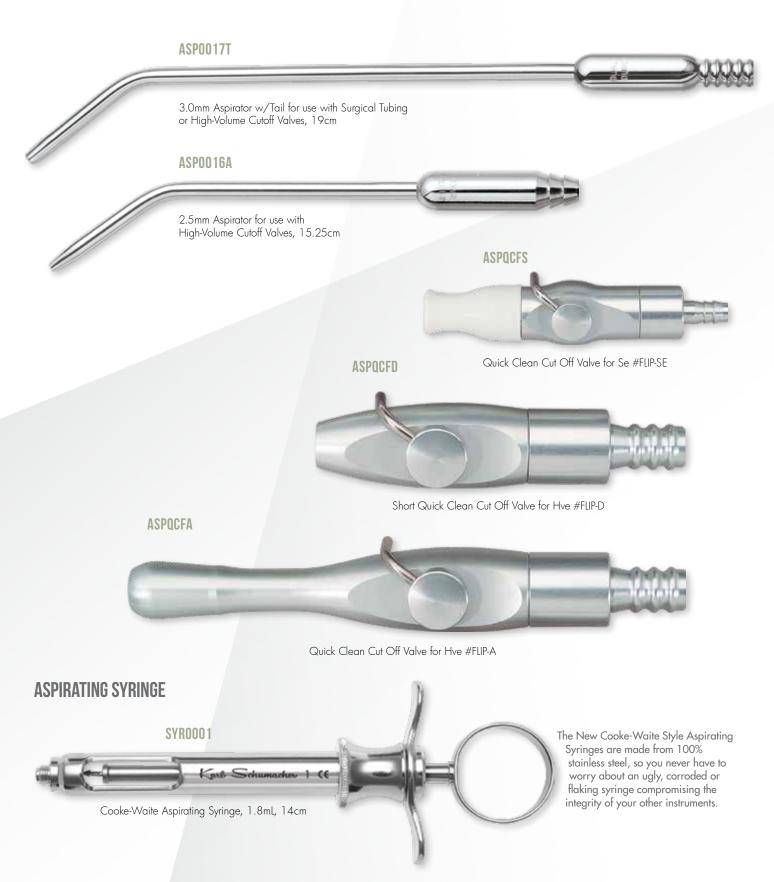

SCALPEL HANDLES



Drop Blade Scalpel Handle #428 Self-Ejecting Blade Design, 15cm



Bard-Parker #3 Scalpel Blade Handle w/60mm Ruler, 12.5cm



Round Scalpel Blade Handle #601, 15.5cm

PERIOSTEAL ELEVATORS

PEL0715



Molt #9 Periosteal Elevator #715, Standard Periosteal/Pointed Periosteal, 18cm Our Molt #9 Periosteal Elevators are available with either wide and thin (above) or narrow and heavy duty tips (below).

PEL0643



Mini Buser Periosteal Elevator #643, 2.8mm Periosteal/2.4mm Pointed Periosteal, 17cm

The sharp, pointed tip of the Micro Buser Periosteal is ideal for delicate manipulation of the papilla.

PLA0536



Our Modified Woodson #1's always feature a sharpened, pointed tip for delicate access and excellent penetration.

PEL0623D



Seldin Periosteal Elevator #23D, Large Flap Retractor/Small Flap Retractor, 20cm

Seldin #23 Retractors are the optimal choice for retraction of the released soft tissue. The broad, smooth sides and tips prevent inadvertent cutting or tearing of the flap.

PEL0724G



Periosteal Elevator #724G, Standard Periosteal/Angled Straight Chisel, 17cm

PEL0644



Buser Periosteal Elevator #644, 2.8mm Pointed Periosteal/4.2mm Periosteal, 17.5cm





The 'Out' curved tip is perfect for application to the facial as it enables a high angle-of-approach to the periosteum, keeping the handle away from the tissue.

PELO715N



Molt #9 Periosteal Elevator #715N, Heavy Standard Periosteal/ Pointed Periosteal, Round Handle, 18cm

PEL0624



Pritchard Periosteal Elevator #624, Retractor/Standard Periosteal, Round Handle, 20cm

PEL0622



Bruno / Gargiulo Periosteal Elevator #622, Thin Flat Periosteal/Standard Periosteal, 17cm

BONE CURETTES & TISSUE FORCEPS



Aggressive serrations on the tips score the socket to promote blood flow and securely engage granuloma for removal.

SERRATED BONE CURETTES




BONE CURETTES




TISSUE FORCEPS




RETRACTORS & HAEMOSTATIC FORCEPS & FILES

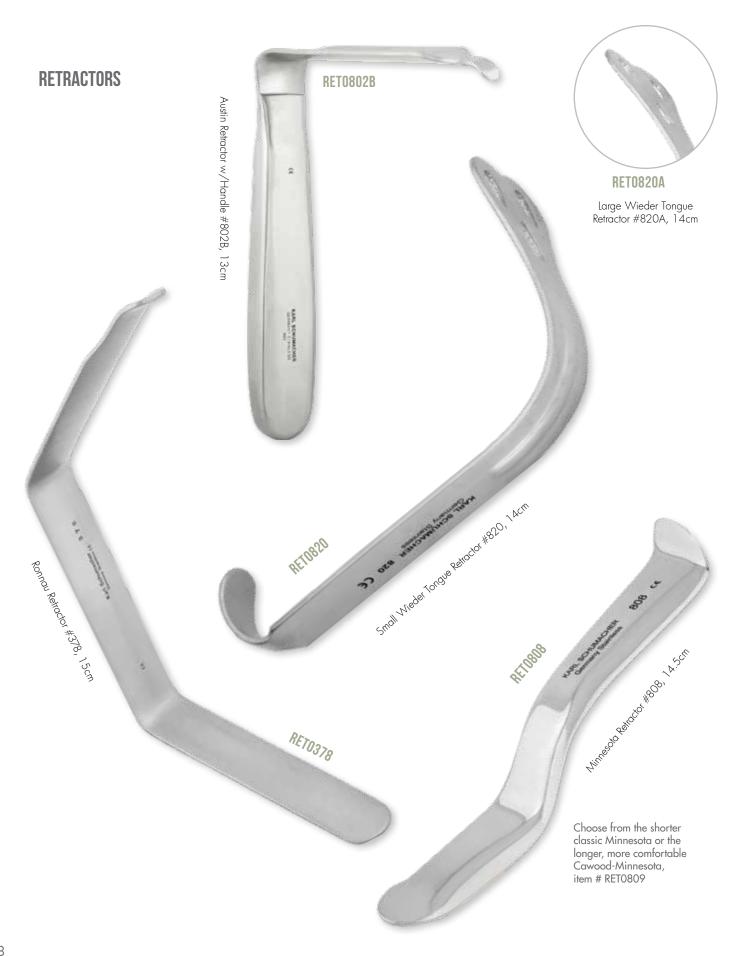

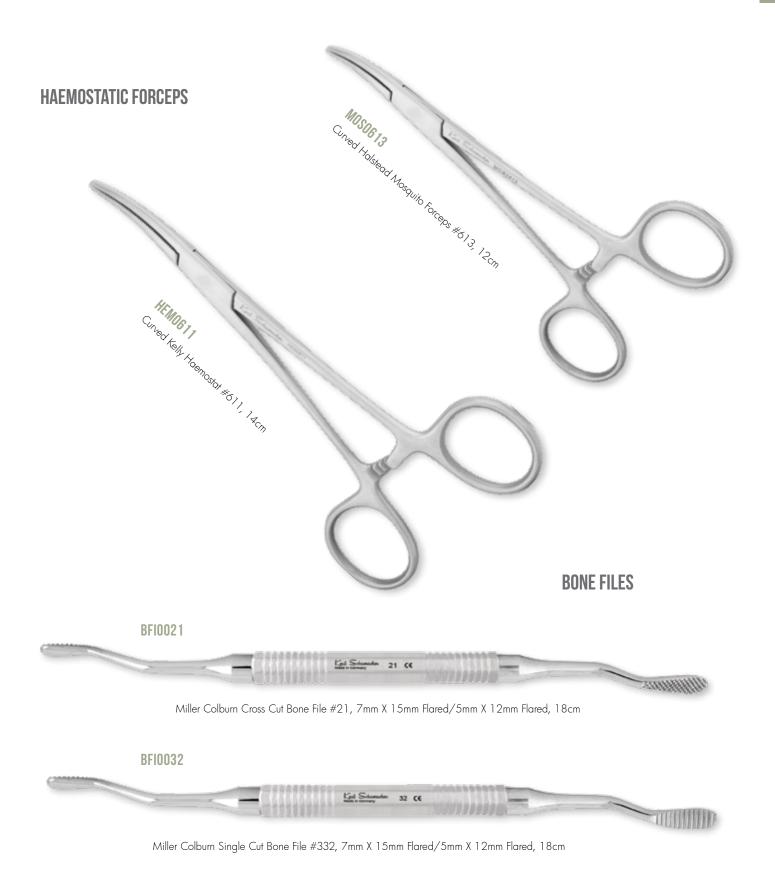


RONGEURS

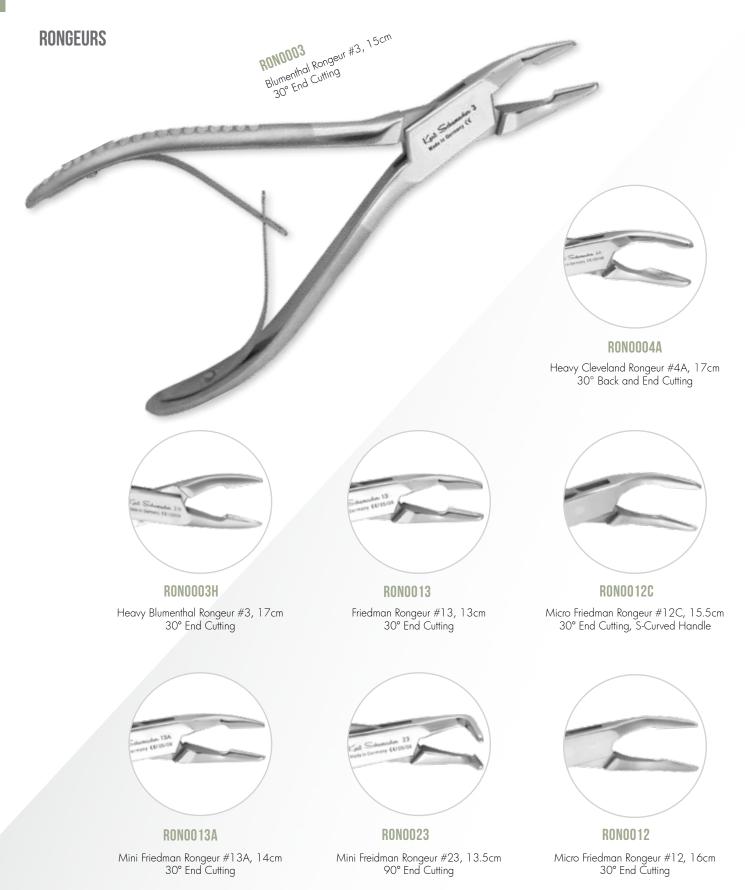

NEEDLE HOLDERS **NEEDLE HOLDERS** We also carry hard-to-find Left-Handed and All of our Needle Holders feature: Curved versions of the most popular designs, as well as Needle Holder / Scissors combinations and durable, hand-finished tips Mathieu styles. precision-cut serrations anti-snare beveling around the box joint For the complete selection please go to: HesiraMed.com/collections/instruments premium WIDIA® Tungsten Carbide inserts NEE0740 Crile-Wood Needle Holders #740, 12cm NEE0690 **NEE0686** Baumgartner Needle Holders #686, 13cm 4-0 Suture Maximum Recommended Crile-Wood Needle Holders #690 15cm 3-0 Suture Maximum Recommended Crile-Wood Needle Holders #790, 18cm NEE0791 Mayo-Hegar Needle Holders #791,18cm 2-0 Suture Maximum Recommended **NEE0691** Mayo-Hegar Needle Holders #691,18cm 2-0 Suture Maximum Recommended

AETRANOX™ SCISSORS

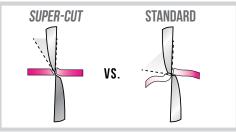


Super-Cut Scissors eliminate the two main causes of scissors failure; smooth, 80° cutting edges 'push' suture and tissue out of the front of the tips and never engage to shear. The anterior blades of our Super-Cut Scissors are hand-sharpened with a fine serration to ensure secure retention within the tips, while the posterior edge is sharpened to a thin, 30° knife edge to cleanly incise suture or tissue without crushing or pushing. Super-Cut scissors are easily identifiable by their single gold ring or AetranoxTM coating and provide the best performance to value ratio.





PERIODONTAL HAND INSTRUMENTS



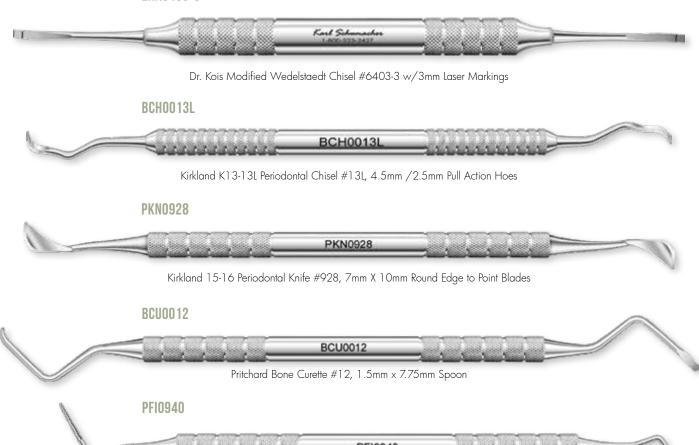

Sugarman 3-4 Periodontal File #940, Perpendicular 11mm x 2-1mm Tapered Safeside Cutting Tips

CASTRO-VIEJO SCISSORS



MICRO TISSUE FORCEPS



MTF0001

Straight Micro Cooley Tissue Forceps #1 , 2×8 Row Teeth, 17.5cm

MTF0004



Curved Micro Tissue Forceps #4, 1 x 2 Teeth, 17.5cm



MTF0002

Angled Micro Cooley Tissue Forceps #2, 2 x 8 Row Teeth, 17.5cm



MTF0005

Straight Micro Dressing Forceps #5, Smooth, 17.5cm



MTF0003

Straight Micro Tissue Forceps #3, 1 x 2 Teeth, 17.5cm



Micro Adson/Brown Tissue Forceps #805, 1 x 2 Teeth, 15cm

CASTRO-VIEJO NEEDLE HOLDERS

CASTRO-VIEJO NEEDLE HOLDERS

Recommended for Suture Sized 6-0 and Smaller



Straight Castro-Viejo Needle Holders #1, Flat Handle, 14cm



NEE0002

Curved Castro-Viejo Needle Holders #2, Flat Handle, 14cm



Straight Castro-Viejo Needle Holders #5, Round Handle, 14cm



NEE0007

Straight Castro-Viejo Needle Holders #7, Round Handle, 18cm

HEAVY BEAK CASTRO-VIEJO NEEDLE HOLDERS

Recommended for Suture Sized 5-0 and Smaller



Straight Heavy Tip Castro-Viejo Needle Holders #9, Flat Handle, 14cm



NEE0010

Straight Heavy Tip Castro-Viejo Needle Holders #10, Flat Handle, 18cm

CASTRO-VIEJO NEEDLE HOLDERS

TWIST JOINT CASTRO-VIEJO NEEDLE HOLDERS

Recommended for Suture Sized 6-0 and Smaller



The unique design of the Twist Joint Castro-Viejo features handles that rotate 90° through the joint to the tip. This rotation eliminates the traditionally machined-flat surfaces at the joint, which can snag sutures if they are not finished and beveled properly, and leaves a consistently smooth instrument surface from the handle to the tip.



Twist Joint Straight Castro-Viejo Needle Holders #7T, 18cm



Twist Joint Curved Castro-Viejo Needle Holders #8T, 18cm



Twist Joint Straight Heavy Tip Castro-Viejo Needle Holders #12T, 18cm


MICRO HANDLE & PERIOSTEALS

MICRO HANDLE AND SCALPELS



MICRO AND MINI PERIOSTEAL ELEVATORS

PEL0639

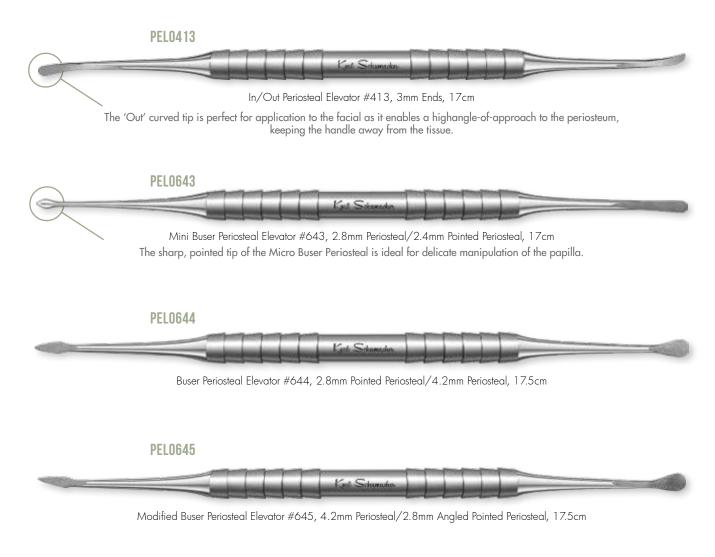

Periosteotome #639, 3.5mm/2.0mm Flat Periosteal Tips, 18cm

639

Made in Germany

Karl Schumacher

TUNNELING INSTRUMENTS







Maxillary Tunneling Instrument #1, 2mm/2.6mm, Magenta! Bionik Handle

TUN0002



Mandibular Tunneling Instrument #2, 2mm/2.6mm, Green Bionik Handle

TUN0012



Combination Tunneling Instrument #3, 2mm/2mm, Blue Bionik Handle

PEL0643




Mini Buser Periosteal Elevator #643, 2.8mm Periosteal/2.4mm Pointed Periosteal, 17cm

HDL0509



Micro Scalpel Blade Handle #509, 13.5cm

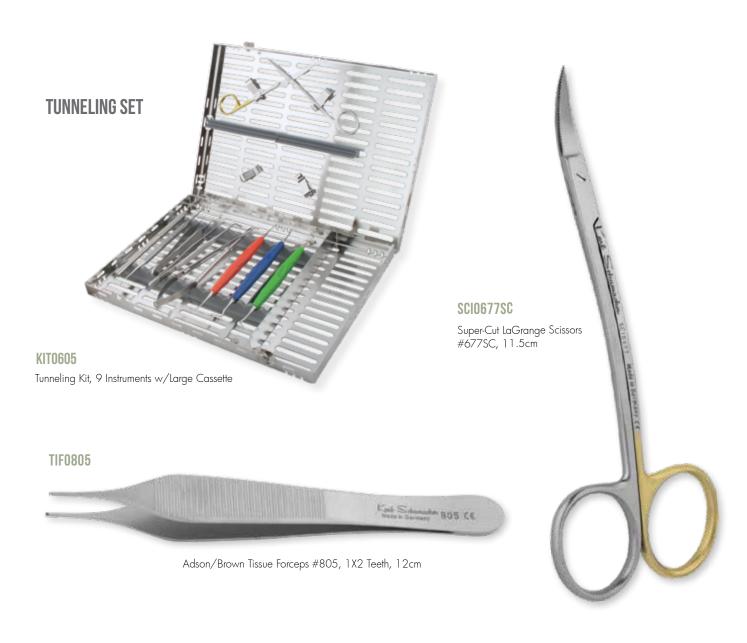




Twist Joint Straight Heavy Tip Castro-Viejo Needle Holders #12T, 18

NEE0012T


BONE MANAGEMENT INSTRUMENTS

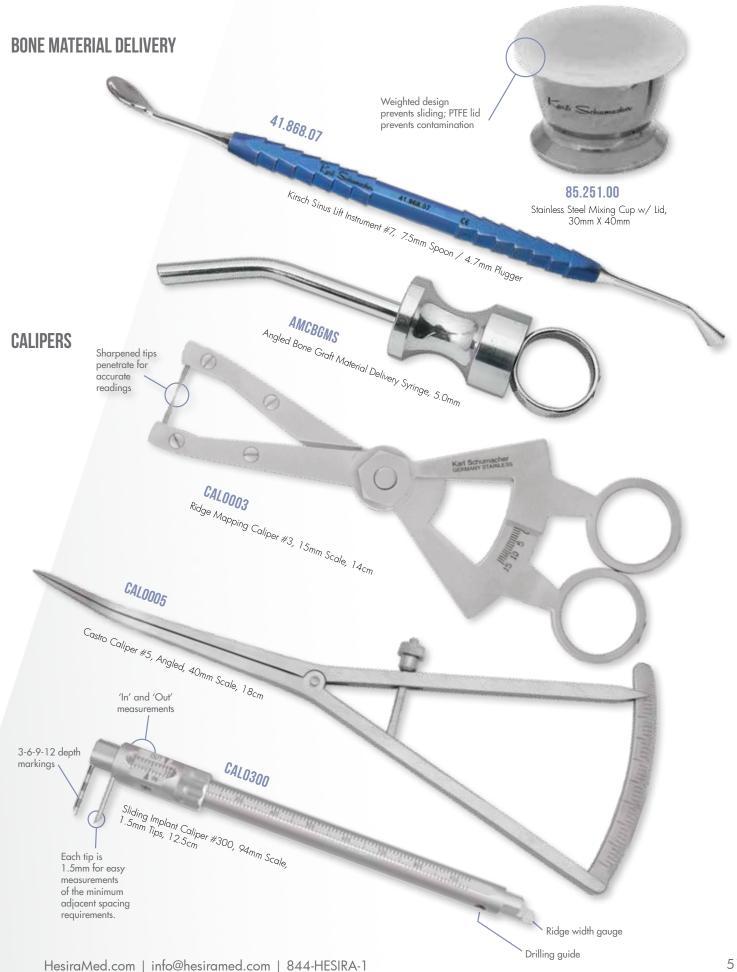

BONE MANAGEMENT INSTRUMENTS



BONE HARVESTING INSTRUMENTS

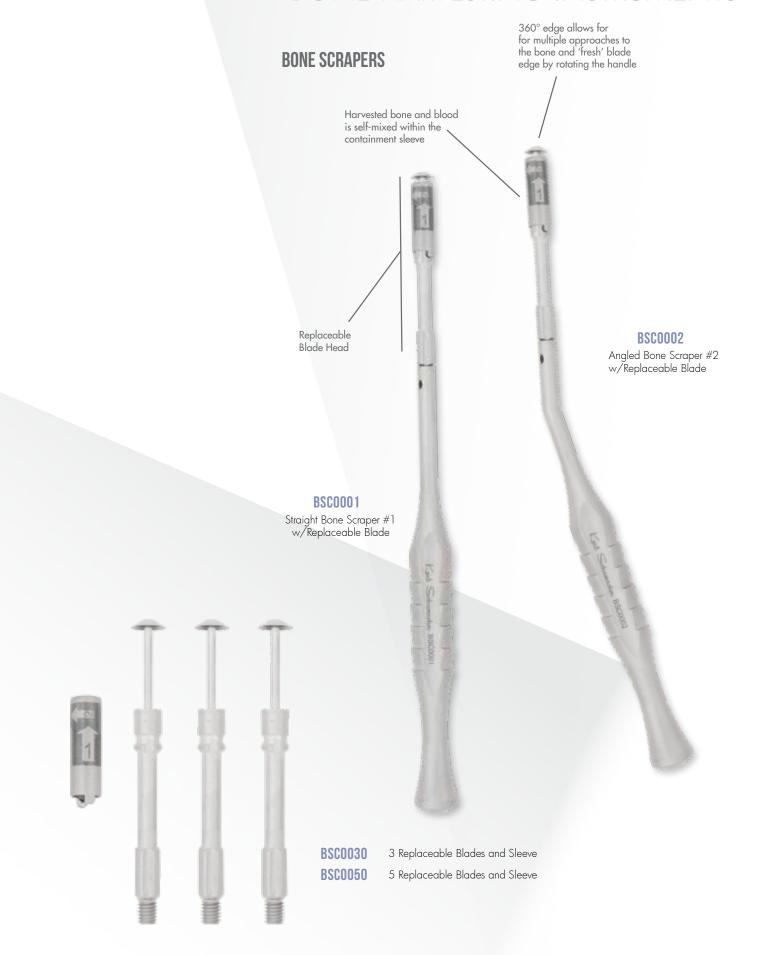

INTERNAL SINUS LIFT OSTEOTOMES

47.943.20



Angled Concave Osteotome, 1.8mm, Silver



47.943.28

Angled Concave Osteotome, 2.2-2.6mm, Silver



47.943.33

Angled Concave Osteotome, 2.6-3.2mm, Silver/Yellow



OSTEOTOMES

OSTSET

Set of 5 Progressively Tapered Angled Concave Osteotomes w/Washtray



47.943.38

Angled Concave Osteotome, 3.2-3.8mm, Purple



47.943.43

Angled Concave Osteotome, 3.8-4.2mm, Blue/Green





Straight Concave Osteotome, 1.8mm, Silver



47.942.28

Straight Concave Osteotome, 2.2-2.6mm, Silver



47.942.33

Straight Concave Osteotome, 2.6-3.2mm, Silver/Yellow



47.942.38

Straight Concave Osteotome, 3.2-3.8mm, Purple



47.942.43

Straight Concave Osteotome, 3.8-4.2mm, Blue/Green

STILLER EXTERNAL SINUS LIFT CURETTES

STILLER SINUS LIFT CURETTES



STILLER EXTERNAL SINUS LIFT CURETTES



STILLER SINUS LIFT INSTRUMENTS





The specialized hardening process of the Stiller Sinus Lift Curettes leaves a softer area in the shank of the curette. This allows you to quickly adjust or angle the shaft of the curette for ideal access through the sinus window while at the same time the fully hardened tips do not distort during instrumentation. The laser etched safe bending zones on the blunt and sharp curettes allow you to quickly see the softer section of the shank.

SLI0995

Stiller Sinus Lift Kit, incl. 8 Curettes, Bone Graft Spoon/Plugger, 2 Titanium Tipped Surgical Aspirators, 2 Mixing Bowls w/Lid, and Washtray



85.251.00 Stainless Steel Mixing Cup w/ Lid, 30mm X 40mm

Replaceable Titanium Tips with Tissue Relief Slot are safe to use around implants and the sinus membrane.

WEDGE RIDGE SPLITTING SET

WEDGE RIDGE SPLITTING SET

BWW0931

Green Bone Wedge, 2mm Wide, 2.2mm Taper

BWW0932

Red Bone Wedge, 2mm Wide, 3.5mm Taper

BWW0933

Yellow Bone Wedge, 3mm Wide, 2.2mm Taper

BWW0934

Blue Bone Wedge, 3mm Wide, 3.5mm Taper



Wedge Ridge Splitting Set, incl. 16 Wedges, Wedge Applicator, 3 Chisels, Mallet and Washtray

BWA0920 Applicator for Bone Wedges MAL0501 KARL SCHUMACHER Mallet #650 w/Replaceable Nylon Faces, 18cm BCH0911 6852 Karl Schumacher 4mm Wide Splitting Chisel #911, Purple **BCH0912** 6 8 8 2 Karl Schumacher 6mm Wide Splitting Chisel #912, Blue **BCH0913** 6802 Karl Schumadur

8mm Wide Splitting Chisel #913, Green

The Wedge Ridge Splitting Set incorporates two progressively tapered sets of wedges to prop open the thin lamellar bone. This allows for controlled ridge expansion and hands-free and precise expansion of the ridge during augmentation. The included Diamond Discs are used to make the initial cut along the crest. Bi-bevel Chisels are then used to advance the split apically with depth markings to ensure the proper depth is attained. Once cut, the wedges are inserted into the ridge to spread it open to the correct width. The areas between the bone can then be filled with grafting material, with additional graft material being placed once the wedges are removed.



Ridge after initial cutting and expansion



Wedges placed, with implant site preparation HesiraMed.com | info@hesiramed.com | 844-HESIRA-1



Wedges removed and additional grafting material being placed

IMPLANT & ABUTMENT DRIVERS/FORCEPS

IMPLANT AND ABUTMENT FORCEPS



Titanium Implant & Abutment Holding Forceps #304, 1 x 2 Curved Tips

ABUTMENT DRIVER SET




ADS0010

Abutment Driving Set #10 incl. 0-40Ncm Wrench, 9 20mm Drivers, 9 30mm Drivers, Insertion Tool, Tray and Washtray



The Abutment Driver System simply and compactly arranges both long and short drivers for 90% of existing abutments on the market. Indispensable when a patient presents with an unfamiliar system, you'll never have to worry if you have the correct driver available. 20mm and 30mm length right angle latch style drivers are arranged numerically. The Adjustable Torque Wrench allows quick selection of the appropriate torque setting, from a fixed, non-breaking setting up to 40 Ncm.



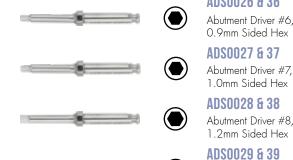







We understand that not all hands are the same; therefore we have a variety of handles available to ensure a proper and secure fit that is correct to you. With three main designs and configurations we are positive that there is a handle option that will enhance your comfort and productivity. Shown within this catalog are the most popular handle and tip configurations. Once you find the right instrument you can always give us a call and ask for a specific handle option if it is not shown.

#6 HANDLE



The Karl Schumacher #6 Handle is the most comfortable and practical handle we carry. The 10mm hollow stainless steel design is one of the lightest handles on the market, which allows for a more relaxed grip and less hand fatigue. Knurled rings enable definitive 360° positioning of the working end. All of our double-ended hand instruments are available in the #6 Handle.

'N' HANDLE

SCHUMACHER

The Karl Schumacher 'N' Handle, is a classic original. Feel the strength in your hands with the 6.5mm solid stainless design. Knurled rings give a secure grip and positive positioning of the tip. All of our hand instruments are available in the 'N' Handle.

BIONIK HANDLE



The Bionik Handle is one of our newest handle options. Originally used exclusively for our Titanium Hygiene instruments and performance surgical hand instruments, we now offer it for our stainless steel scalers. The international award-winning design is formed entirely of thermoplastic PEEK, is ultra lightweight and the best non-metallic material available for use in the medical and dental environments.

ANTERIOR SCALERS



You will find that all Karl Schumacher hygiene instruments feature tips noticeably thinner than the industry standard. This is due to the high grade steel and quality craftsmanship put into each instrument, which allows for deeper interproximal and subgingival access resulting in a better and more thorough cleaning.




Anterior Sickle Scalers are available with different degrees of offset for personal preference and more universal adaptation.

And increased offset allows for more comfortable hand positioning while cleaning both the facial and lingual of the anteriors, with the added benefit of a more posterior reach and adaptation of the blade.

Thin Gets In

291 vs. 288 vs. 208

SICKLE SCALERS

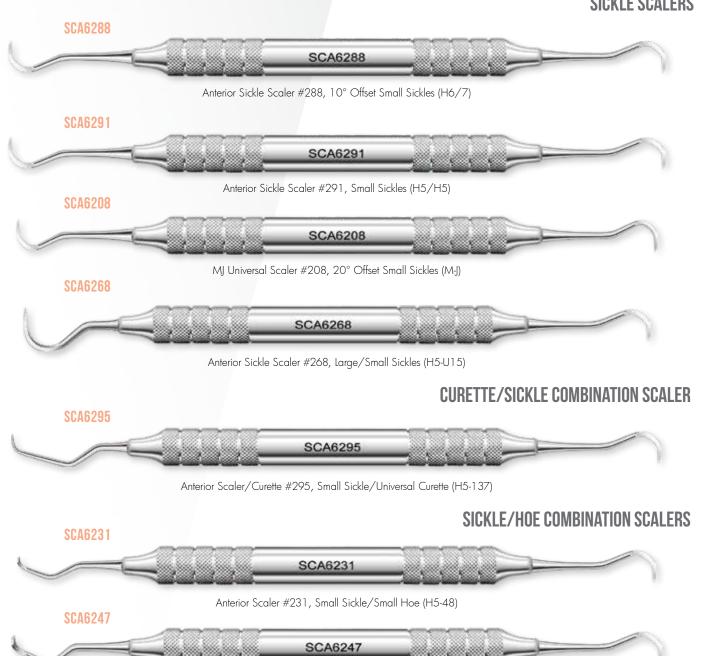


Anterior Scaler #247, Small Sickle/Large Hoe (H5-47)

ANTERIOR SCALERS





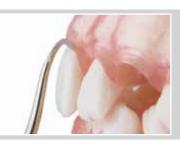

Patina 2mm Spoon

#48 Hoe

N5 Sickle

Karl Schumacher's Patina Lingual Stain Scalers feature a pull-action 2mm sharpened spoon paired with either a N5 Sickle or 48 Hoe tip. The circular edge of the Patina spoon adapts perfectly to the concave lingual anatomy of the anteriors for fast removal of tough stains. Facially, the N5 Sickle is a standard choice for easy and thorough cleaning; or choose the 48 Hoe tip for heavily calcified deposits that require a more aggressive instrument.

PATINA LINGUAL STAIN SCALERS

SCA6228



Patina Lingual Stain Scaler #228, 2mm Spoon/Small Hoe, (SR-48)

SCA6001



Patina Lingual Stain Scaler #1, 2mm Spoon/Small Sickle, (SR-H5)

JAQUETTE COMBINATION SCALERS

SCA6261B



Jaquette Anterior Scaler #261B, Large/Small Jaquette (J1/J1S)

SCA6289



Anterior Scaler #289, Small Sickle/Small Jaquette (H5-J1S)

SCA6281



Anterior Scaler #281, Large Sickle/Small Jaquette (U15/J1S)

ANTERIOR SCALERS

BLADE SCALERS

SCA6299



Anterior Scaler #299, Short/Regular Straight Blade (Morse O-OO),

HEAVY SICKLE COMBINATION SCALERS

SCA6296



Anterior Scaler #296, Small Sickle/Small Half Sickle (H5-YG15)

HOE SCALERS

SCA6276



Anterior Hoe Scaler #237, Small Angled Tip Hoes (45A-46A)

SCA64748



Anterior Hoe Scaler #4748, Standard Hoe/Large Hoe (47-48)

CHISEL SCALERS

SCA6269



Anterior Chisel Scaler #269, Small/Regular Straight Chisel (Zerfing 1-Zerfing 1S)



The Karl Schumacher #6 Handle is the lightest, most comfortable and practical stainless steel handle we carry.

Our Hygiene Kit #403 contains 6 commonly selected Scalers and Curettes, a mirror and explorer/probe for routine prophylaxis and exams, stored securely in a medium cassette.



POSTERIOR SCALERS

204 POSTERIOR SICKLE SCALERS



Small 'Open Angled' Posterior Scaler #204SD (204SD)



204 Scalers are the most popular posterior sickle scaler, and are truly 'universal.' Available in four main tip and shank styles, the 204's angulations orient the thin sickle tips correctly for posterior application, while the handle doesnot interfere with the opposing arch.

The standard 204 is shown at left reaching interproximally, with room to spare.

The 204 has a long terminal shank and tip for excellent reach.

The 204J features a heavier tip design for severe calculus removal.

The 204S has shorter and tighter angulations and a shorter reach for finer access and visualization.

The 204SD is sized similar to the 204S, but with a more 'open' angulation for better distal access.

POSTERIOR SICKLE SCALERS



POSTERIOR SCALERS

REMINGTON POSTERIOR SCALERS

SCA6220



Posterior Sickle Scaler #220, Large, Long Blades (Remington 3/4)

SCA6221



Posterior Sickle Scaler #221, Small, Long Blades (Remington 3S/4S)

SCA6230



Posterior Half Sickle Scaler #230, Large, Long Blades (Remington K)

JAQUETTE POSTERIOR SCALERS

SCA65253



Small Jaquette Posterior Scaler #5253 (J2S/J3S)

HEAVY POSTERIOR SICKLE SCALERS

SCA6293



Heavy Posterior Sickle Scaler #293 (Taylor 2/3)

SINGLE-ENDED SCALERS

SCA0015



Younger-Good Scaler #15 (YG15)



SCA0122

Large Half Sickle Scaler #122 (Whiteside 2)



SCA0140

Heavy Sickle Scaler #140 (CI1)



SCA0005

Straight Blade Scaler #5 (B)



SCA0117H

Large Jaquette Scaler #117H (J1)

UNIVERSAL CURETTES



The face of Universal Curettes are oriented perpendicular to the long axis of the terminal shank, allowing for both edges of the tip to be utilized. Universal Curettes have a wider range of applications and can reduce the amount of instrument switching needed to effectively clean subgingival deposits.

Area Specific Curettes have a face that is oriented 70° in relation to the terminal shank, and allow for the proper application of only one edge. Wider application is reduced, but the tips are more 'specific' to each indicated application.

BARNHART UNIVERSAL CURETTES



Barnhart Universal Posterior Curette #286, Short Blade (Barnhart 5/6)

MCCALL UNIVERSAL CURETTES



McCall Posterior Universal Curette #238, Heavy Blade (McCall 13/14)

GOLDMAN-FOX SCALERS/CURETTES

COLUMBIA UNIVERSAL CURETTES



SCA6270

Columbia Universal Posterior Curette #270, Short Shank, Blades (Columbia 13/14)

YOUNGER-GOOD UNIVERSAL CURETTES

SCA6267 SCA6267


Posterior Universal Curette #267, Regular Blade, Slight Offset (YG 7/8)

GOLDMAN-FOX INSTRUMENTS

SCA6290



SCA6259

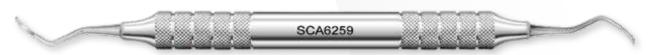

Goldman-Fox Posterior Universal Curette #259 (GF 4)

SCA6279



Goldman-Fox Anterior Scaler #279, Jaquette Small Standard/Short Straight (GF12)


INDIANA UNIVERSITY CURETTE

SCA6275



Indiana University Posterior Universal Curette #275, Large Long Blades (IU 17-18)

ORIGINAL GRACEY CONFIGURATIONS

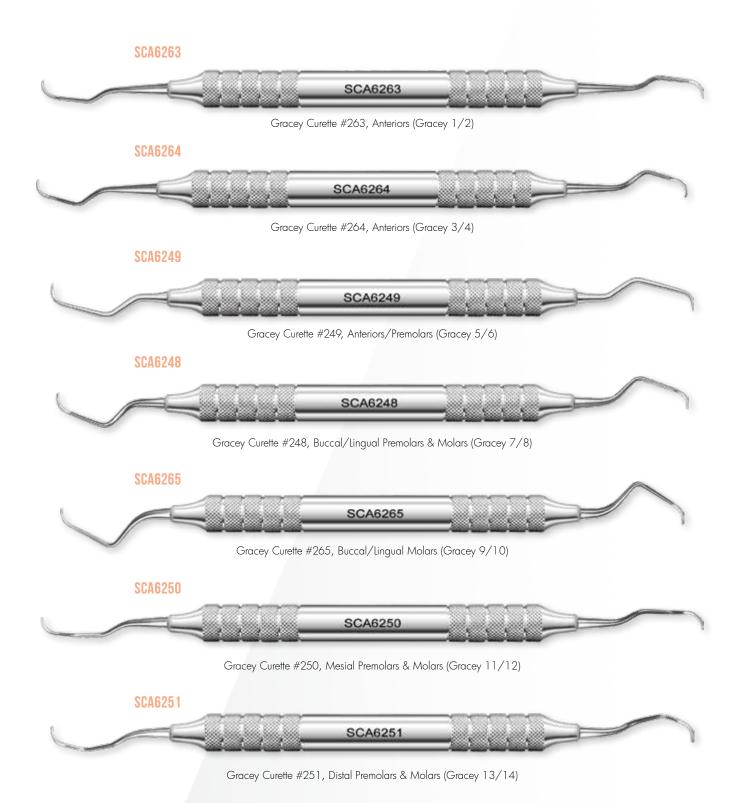

GRACEY CURETTES




Karl Schumacher Gracey Curettes feature Truform Edge Finishing $^{\text{TM}}$ for anatomically correct adaptation to the tooth surface.

Each tip has been precisely formed and curved in order to provide complete contact of the blade edge along the tooth.

Truform Edge Finishing™ results in a faster and more efficient debridement while minimizing tissue trauma by requiring fewer strokes and less pressure compared to a conventionally produced and sharpened Gracey Curette.

SCA61516

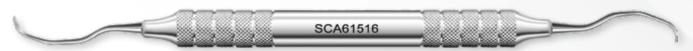

Gracey Curette #1516, Mesial Premolars & Molars (Gracey 15/16)

SCA61718



Gracey Curette #1718, Distal Premolars & Molars (Gracey 17/18)

MESIAL/DISTAL GRACEY CONFIGURATIONS





Gracey Curette #1213, Mesial/Distal (Gracey 12/13)

BIONIKT IMPLANT HYGIENE INSTRUMENTS


BIONIK^{TI}

Bionik Scaler and Curette tips are produced from the same titanium alloy used to manufacture implants and implant components. They are turned and formed from a solid piece of titanium, unlike other instruments made with a thin coating of titanium over a stainless steel core. Titanium is more effective at removal of deposits and can be machined to a much finer tip that will not leave surface debris unlike plastic or graphite instruments. The natural flexibility and compatibility of titanium is rigid enough to withstand scaling pressure, but will not damage the surface of the implant. The complete line of scaler and curette configurations are available exclusively on the lightweight, ergonomic and easy to identify color coded Bionik handles.

UNIVERSAL CURETTES



BIONIKTI IMPLANT HYGIENE INSTRUMENTS

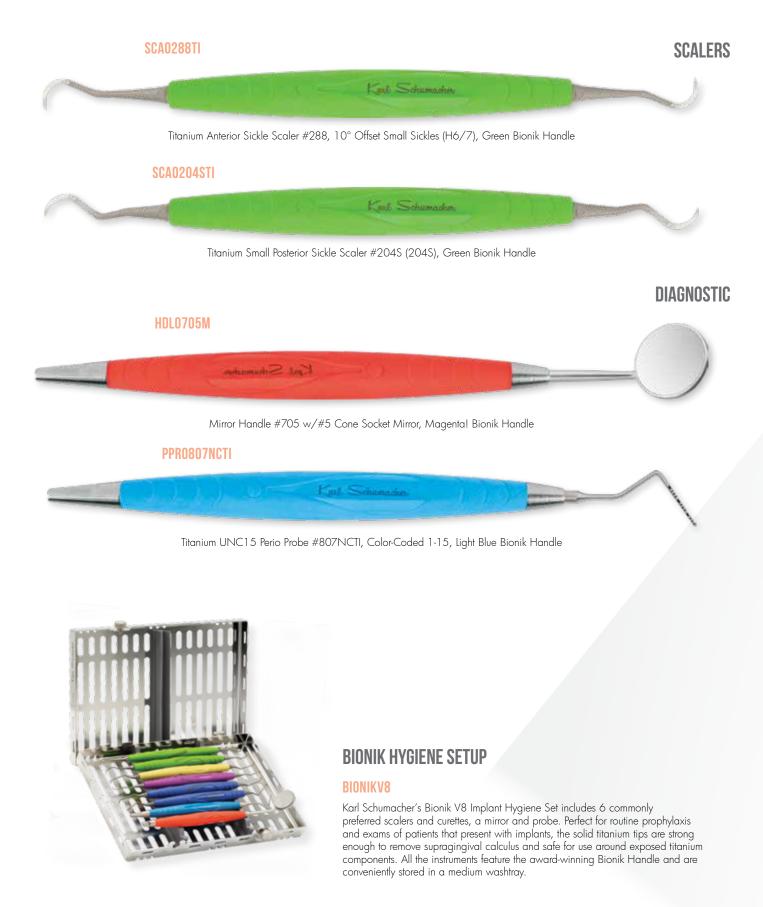

COTTON/COLLEGE PLIERS

COTTON PLIERS





Locking Cotton Pliers #18L, Back Angled, 16cm

MIRRORS & HANDLES





CONE SOCKET MIRROR HANDLES



Small Round Mirror Handle #494, Cone Socket, 12cm

HDL0494CM



Small Round Mirror Handle #494, Cone Socket, w/60mm Scale, 12cm

CONE SOCKET OR SIMPLE STEM MIRROR HANDLES

HDL0490S - Simple Stem HDL0490C - Cone Socket

Small Octagonal Mirror Handle #490, 12.5cm

HDL0493S

- Simple Stem

HDL0483

HDL0493C

- Cone Socket

Kat Samuela

Handform Mirror Handle #493, 13.5cm

HDL8007C

CE Germany

- Cone Socket



EFM Mirror Handle #8007C, w/Endodontic File Measuring Slide, 13.5cm

MIRRORS

Cone Socket Mirrors					
ITEM No.	SIZE				
MIRO004CF	#4 (22mm)				
MIR0005CF	#5 (24mm)				
Single Stem	Mirrors				
MIRO004SF	#4 (22mm)				
MIRO005SF	#5 (24mm)				
MIRO004DS	#4 (22mm)				
MIRO005DS	#5 (24mm)				
MIRO004LS	#4 (22mm)				

BASIC EXAM




KIT0201

The Basic Exam Kit #201 organizes and stores a mirror, Cotton Plier and explorer/probe in a small cassette for fast and easy setup.


EXPLORERS & PERIO-PROBES

EXPLORERS

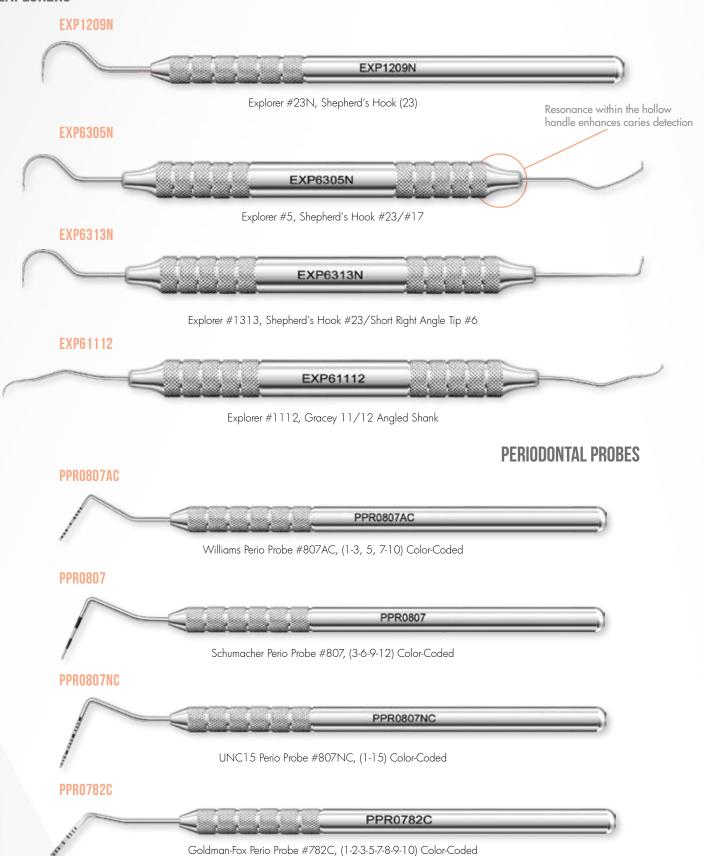

PERIO-PROBE/EXPLORER COMBINATIONS

PROBE/EXPLORER COMBINATIONS

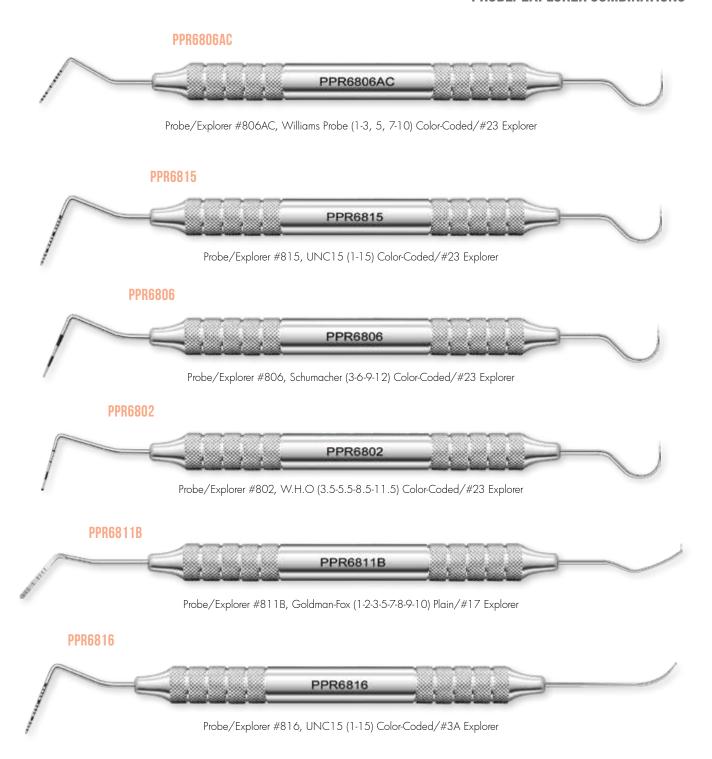

CORD PACKERS, MATRIX & SPATULAS

CORD PACKER

Our Cord Packers are also available with finely serrated tips upon request.



Cord Packer #122, Perpendicular Angled, 1.75mm X 10.75mm

MATRIX BANDS AND RETAINERS





Tofflemire Matrix Bands #1, 12/Pkg

MRT0010



Tofflemire Matrix Retainer #10, Adult, Universal

MRT0011



Tofflemire Matrix Retainer #11, Adult, Angled


SPATULAS

SPA0024A




Cement Spatula #24A, 6.5mm Wide X 42.5mm Long (24A)

SPA0313

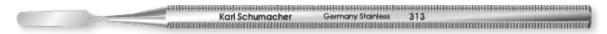

Cement Spatula #313, 3.5mm Wide X 19.5mm Long

EXCAVATORS

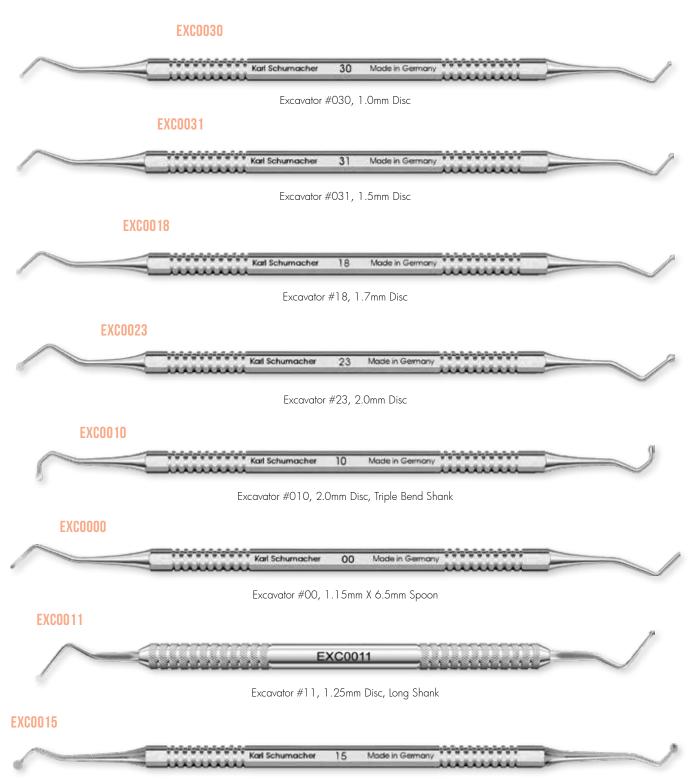

Excavator #15, 2.5mm Disc

NON-STICK COMPOSITE INSTRUMENTS

PADDLES

PFI6039T



TiNi Coated Composite Instrument #39-T, 2.0mm X 9.75mm Long Thin Perpendicular Paddles

PFI6040T



TiNi Coated Composite Instrument #40-T, 2.0mm X 8.0mm Long Thin Perpendicular Paddles

PFI6004T



TiNi Coated Composite Instrument #4-T, Perpendicular 11.15mm Long X 2.15mm Wide, Flared

PFI6007T



TiNi Coated Composite Instrument #7-T, Perpendicular, 12.5mm Long X 1.75mm Wide

PFI6000LRT



TiNi Coated Composite Instrument #L/R-T, 1.6mm Wide X 11.3mm Long, Left/Right Curvature

PADDLE/PLUGGERS

PFI6003T



TiNi Coated Composite Instrument #3-T, 1.15mm Plugger/7.5mm Long X 2.0mm Wide, Flared

PFI6002T



TiNi Coated Composite Instrument #2-T, 1.6mm Plugger/9.5mm Long X 3.15mm Wide, Flared

PADDLE/BURNISHERS

PFI6023T



TiNi Coated Composite Instrument #23-T, 2.75mm Acorn/11mm Long X 2.0mm Wide Paddle

PFI6014T



TiNi Coated Composite Instrument #14-T, 2.0mm Burnisher/9.5mm Long X 3.15mm Wide, Flared


PFI6534T



TiNi Coated Composite Instrument #534-T, 1.5mm/2.0mm Burnisher

FILLING INSTRUMENTS


PLASTIC FILLING INSTRUMENTS

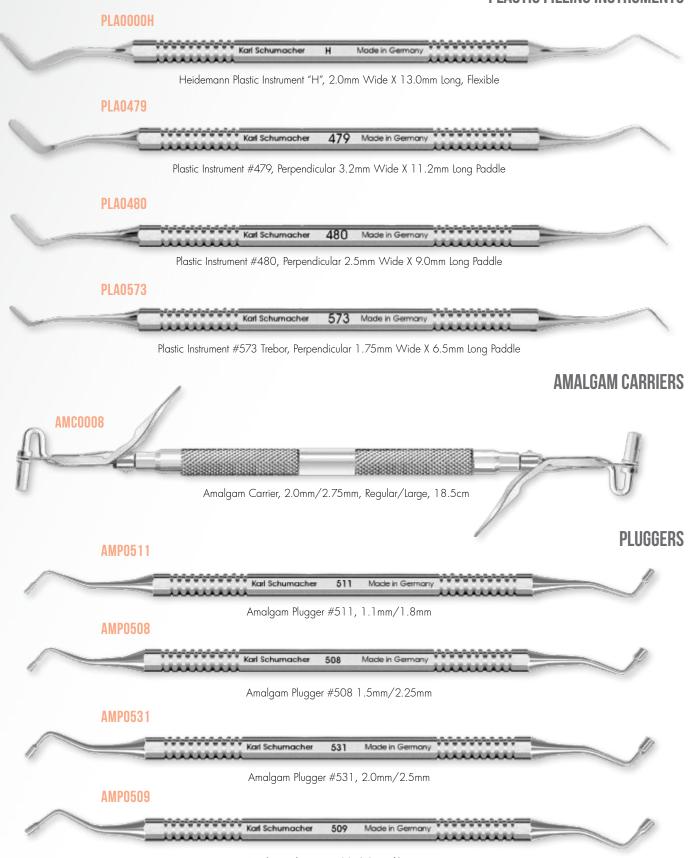

Amalgam Plugger #509, 2.0mm/2.75mm

BURNISHERS & CARVERS

BURNISHERS

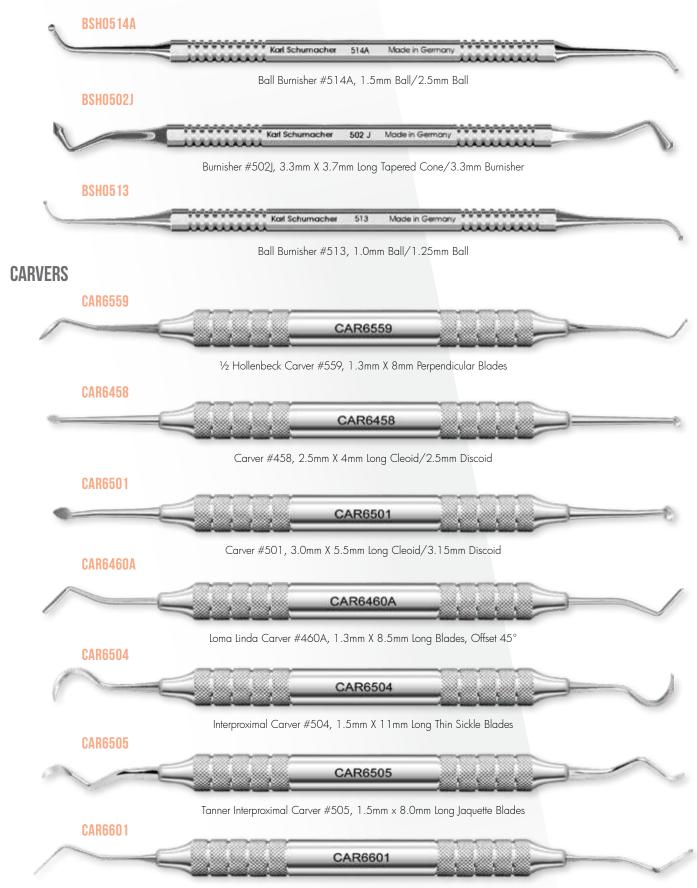

I.P.C. Interproximal Carver #601, 1.4 mm X 10.5mm Long Perp Blade

ENDODONTIC INSTRUMENTS


The EFM Handle enables fast and convenient adjustments to endodontic locator stops on the back of a cone socket mirror handle. With a matte finish and precision laser etched marking at 1/2mm increments, the EFM Handles' centering cut and stop holder allows you to quickly measure and adjust the stop in one motion, eliminating the need to measure-adjust-confirm with traditional finger rulers or slide gauges.








Endodontic Excavator #031L, 1.6mm Spoon, Long Shank

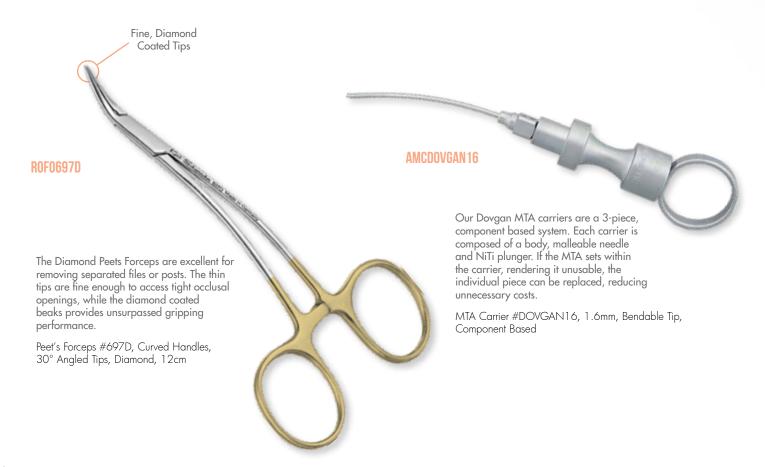

CROWN REMOVERS



CROWN REMOVERS

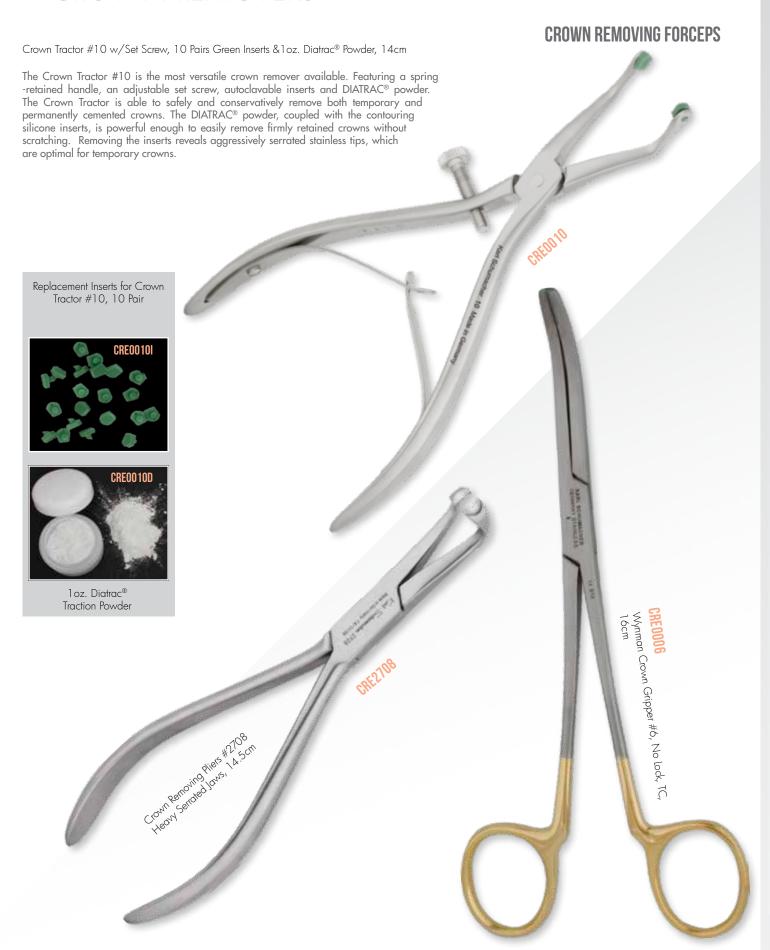

Karl Schumacher

PAYMENT

We accept cheques, direct deposit, Visa, Mastercard, Amex and payment will be processed in CAD.

PRICING

We offer various promotions and discounts through the year, including volume discounts. Prices are subject to change without notice. For current promotions please visit: HesiraMed.com or email us info@hesiramed.com.

AVAILABILITY

Our goal is to have your product in stock. In the event of a backorder, we will ship the product as soon as it is available at no additional shipping cost.

INSTRUMENT WARRANTY

Karl Schumacher Extraction Forceps are guaranteed for life. All other instruments are warrantied for one year. Special instrument coatings and inserts fall under the standard one-year warranty. Instruments that are damaged from mishandling, misuse, improper maintenance or sharpening, or are improperly sterilized may void any warranty.

RETURNS & EXCHANGES

We accept product returns and exchanges within forty five (45) days from receipt of product. A Return Materials Authorization (RMA) number is required prior to returning any item for credit or exchange. Unopened products may be returned for full credit or exchange. Opened instruments will incur a 25% processing fee.

ALPHABETICAL INDEX

#	/\LI I I	MDL	
	204 Scaler	<u>G</u>	0 1 /0 0 0 0 0
	300 Series Extraction Forceps		Garguilo/Bruno Periosteal 33, 34
	see: Deep Grip Extraction Forceps 77R Elevator		Goldman-Fox Scalers & Curettes
Α	// K Lievaloi		Scissors
_	AB/AS Extraction Forceps		Gracey Curettes
	Abutment Drivers		,
	Abutment Forceps		Handles
	Adson-Brown Tissue Forceps 37, 45		Mirror
	Aetranox TM Edition		Scalpel 33, 48 Heidbrink Elevators 13
	Extraction Forceps		Heidemann Plastic Instrument
	Scissors		ricidentalin Flashe instrument
	Amalgam Carriers		Interproximal Carvers
	Anterior Scalers & Curettes 60 - 62		Iris Scissors
	Apical Retention Extraction Forceps 27	Ī	
	Aspirators		Jaquette Scalers
D	Austin Retractor	<u>K</u>	Kally
<u>B</u>	Bard-Parker Scalpel Blade Handle 33		Kelly Forceps
	Barnhart Curettes		Scissors
	Beck Elevators		Kirkland Periodontal Knife & Chisel 40
	Bionik Implant Hygiene Instruments 69		Kirsch Bone Spoon/Plugger 51, 55
	Blumenthal Rongeur		Kits see: Kits
	Bone Calipers		Abutment
	Chisels		Basic Sinus Lift
	Condensors		Bionik Hygiene
	Curettes		Bone Grafting
	Delivery 51, 52		Implant Site
	Files		Dr. Nekky Jamal 5
	Harvester		Dr. Reznick
	Burnishers		Extraction
<u>C</u>	buser remosiedi		Luxating Elevator
_	Calipers		Micro Surgery
	Carriers		Oral Surgery'
	Amalgam		Osteotome
	Bone		Periotome
	MTA		Post-Op
	Castro-Viejo		Ridge Splitting
	Caliper		Soft Tissue
	Needle Holders		Stiller Sinus Lift
	Scissors		Tunneling 4, 49, 50
	Chisels		Kois Modified Wedelstaedt Chisel 44
	Cogswell Elevator	Ŀ	10 10 10 10
	Columbia Curettes		LaGrange Scissors
	Cord Packers		Locklin Scissors
	Corn Suture Forceps		Lucas Bone Curettes
	Cotton/College Pliers		Luxating Elevators 20, 21
	Cowhorn Extraction Forceps 30		
	Crane Pick Elevator		Mallet
	Crown Pamours / Sanadars 91 92		Matrix Bands & Retainers
	Crown Removers/Spreaders81 - 82 Cryer Elevators		McCall Scalers & Curettes 65 65
	Curettes		Needle Holders
	Bone		Periosteals
	Hygiene see: Scalers & Curettes	;	Scissors
_	Sinus Lift		Tissue Forceps
₫	Dean Scissors		Miller-Colburn Bone Files
	Deep Grip Extraction Forceps		Mirrors
	Diamond		Mixing Bowl
	Peets Forceps80		Molar Furcation Extraction Forceps30
_	Tipped Forceps		Molt Periosteal
Ē	Elevente en		Mosquito Forceps
	Elevators East/West	N	Mucosa Punches
	Luxating	17	Needle Holders
	Root Tip 13, 14, 15		Standard
	Straight		Castro-Viejo
	Endodontic Instruments		Non-Stick Composite Instruments 77
г	Excavators	<u>O</u>	Occasion Consulting Town
<u>F</u>	File		Osseous Coagulum Trap
	Bone	<u> P</u>	
	Periodontal		Pediatric Extraction Forceps 29
	Flex-Ex Ligament Knife		Peets Forceps
	Forceps		Periodontal Periodontal
	Abutment		Hygiene Instruments see: Scalers & Curettes
	Extraction 26, 29, 31		Surgery Instruments 32 - 33, 44 - 45
	Haemostatic/Mosquito		Probes
	Root Tip		Periotomes
	Tissue		
	Friedman Ronguers 40		

	Plastic Instruments Stainless	72 ts
	Amalgam	51 58
	Pritchard Periosteal	14
<u>R</u> S	Remington Scalers	38 57
21	Scalers & Curettes	48 43 33
	External	56 54
	Spade Elevators. 1 Proximators® . 1 Root Tip Forceps. 2 Spatulas, Operative 7	10 15
	Spoons 36, 4 Bone Curettes 36, 4 Excavators 76, 8 Grafting 5 Stiller Sinus Lift Curettes 4 Sugarman Periodontal File 4	30 51 55
Ţ	Syringe, Aspriating	32 50
	Tunneling Instruments	50 50
W		
X	Westcott Scissors 4 Woodson Periosteal 5	14 33
<u>Y</u> <u>Z</u>	Younger-Good Curettes	56



Proudly 🍁 Owned & Operated

HesiraMed.com | info@hesiramed.com | 844-HESIRA-1





