

Extension installation

1. Double-click the installer, as shown in Figure 1:

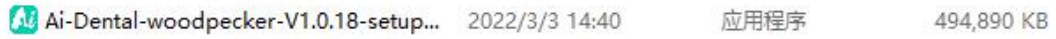


figure 1

2. Select the installation language, as shown in Figure 2:



figure 2

3. After the installation program starts, click the "Browse" button to select the installation path.

After the path is selected, click the "Next" button, as shown in Figure 3 :

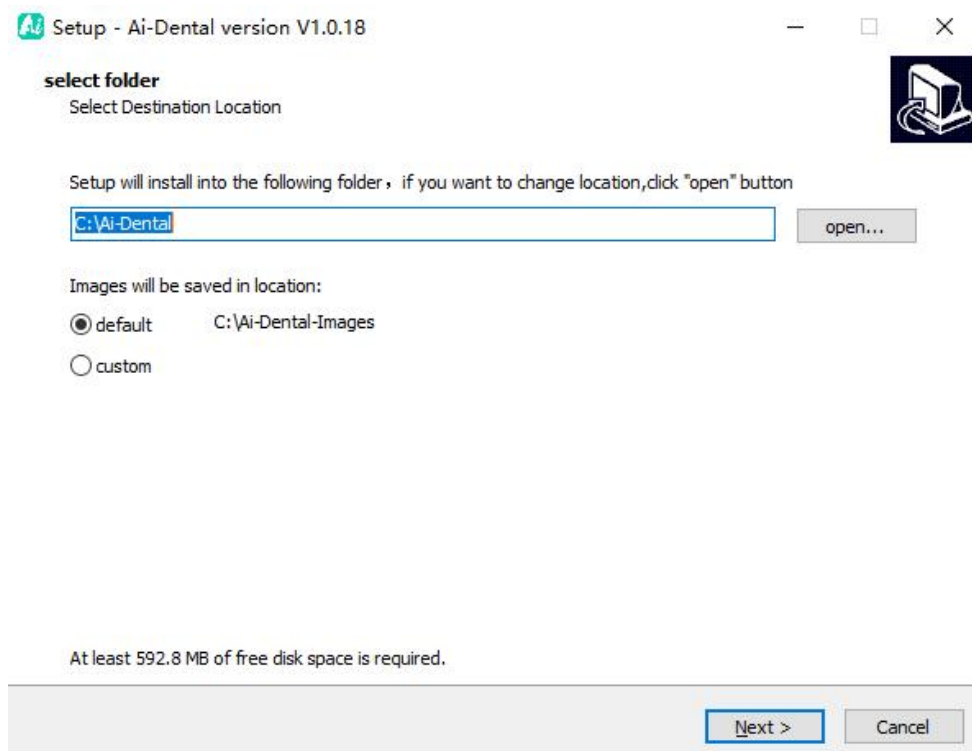


image 3

4. Read the software license agreement, agree to this agreement, click "I agree to this agreement", click Next to continue the installation, if you do not agree with this agreement, click "Cancel" to exit the installation program, as shown in Figure 4.

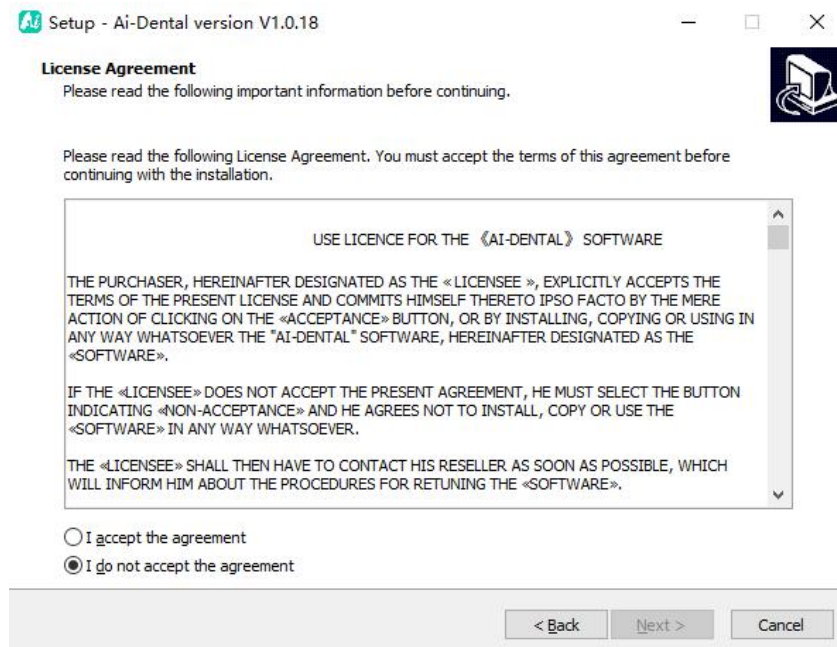


Figure 4

5. Select the component, the user will check the client as needed, after selecting the component, click the "Next" button, as shown in Figure 5:

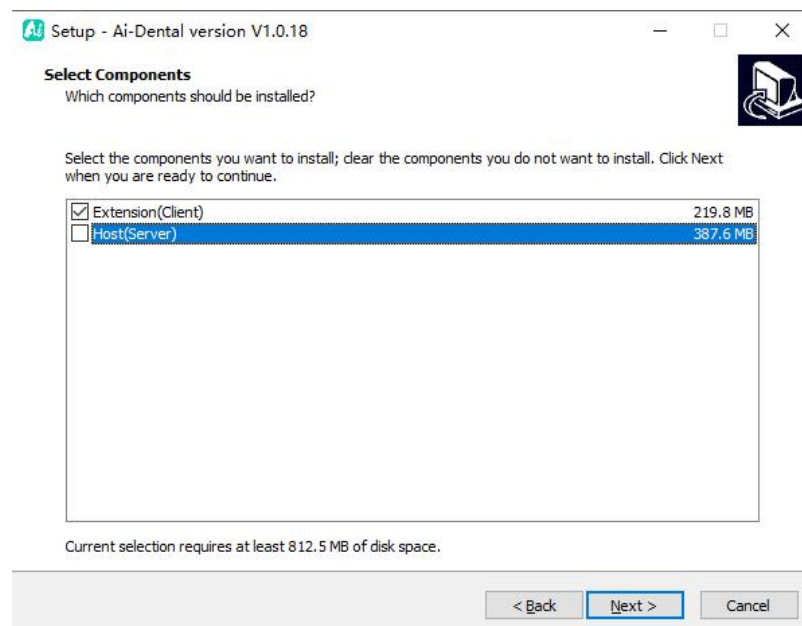


Figure 5

6. Set whether to create a shortcut and start the startup, click the "Next" button after completion, as shown in Figure 6:

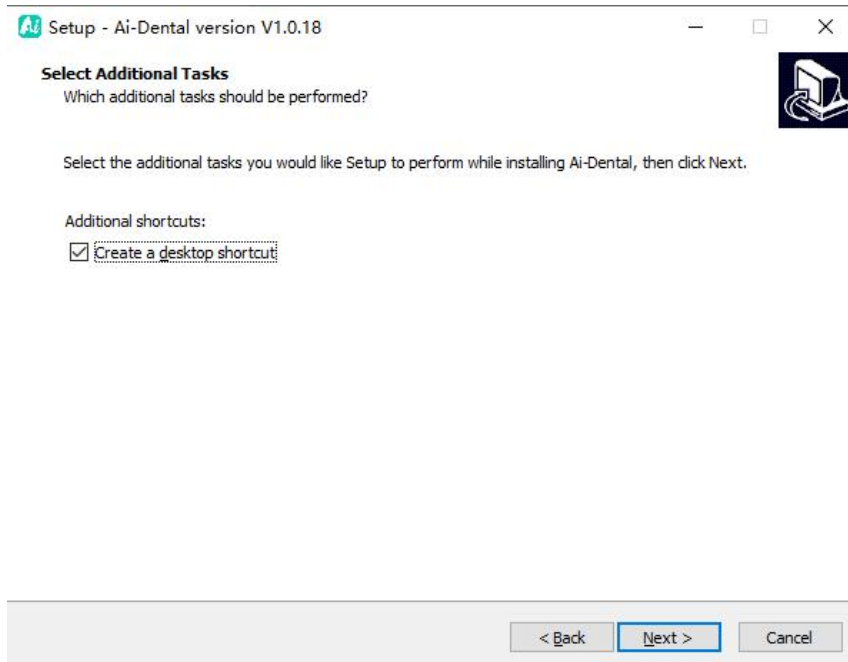


Image 6

7. Click the "Install" button to start the installation, as shown in Figure 7:

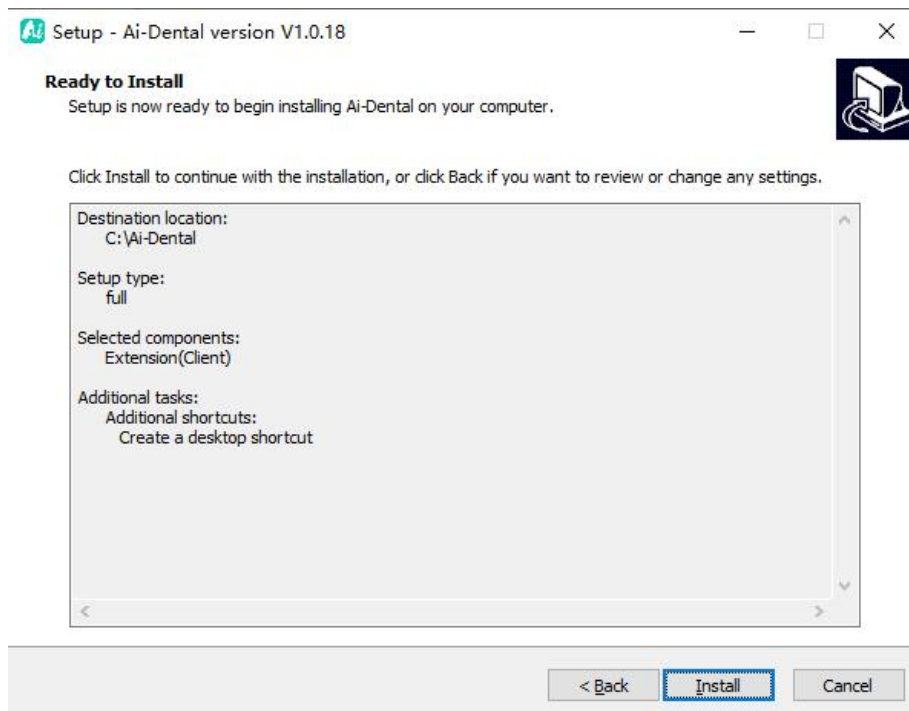


Figure 7

8. After clicking the "Install" button, the program starts to install, and the user can wait for the installation to complete, as shown in Figure 8 :

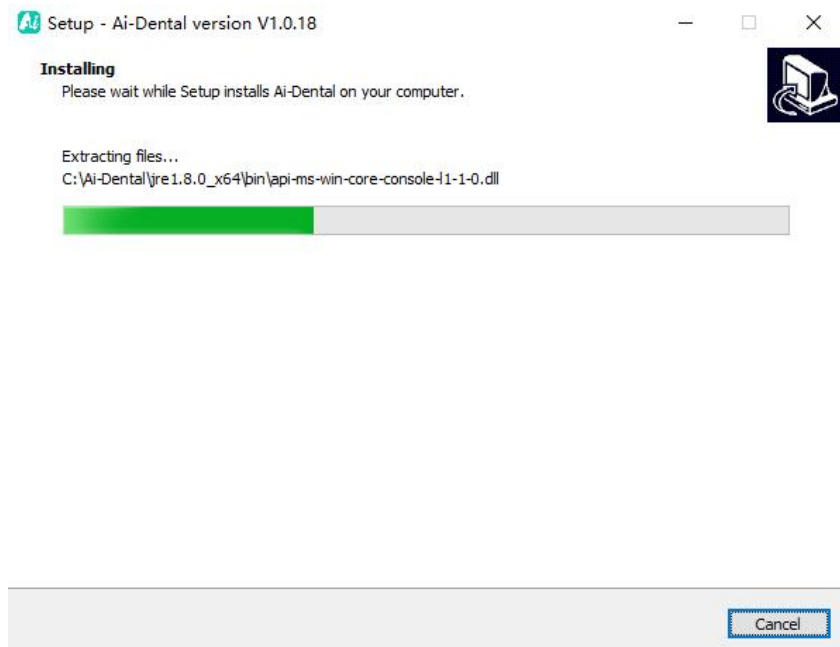


Figure 8

9. After the software installation is complete, click "Finish", as shown in Figure 9:

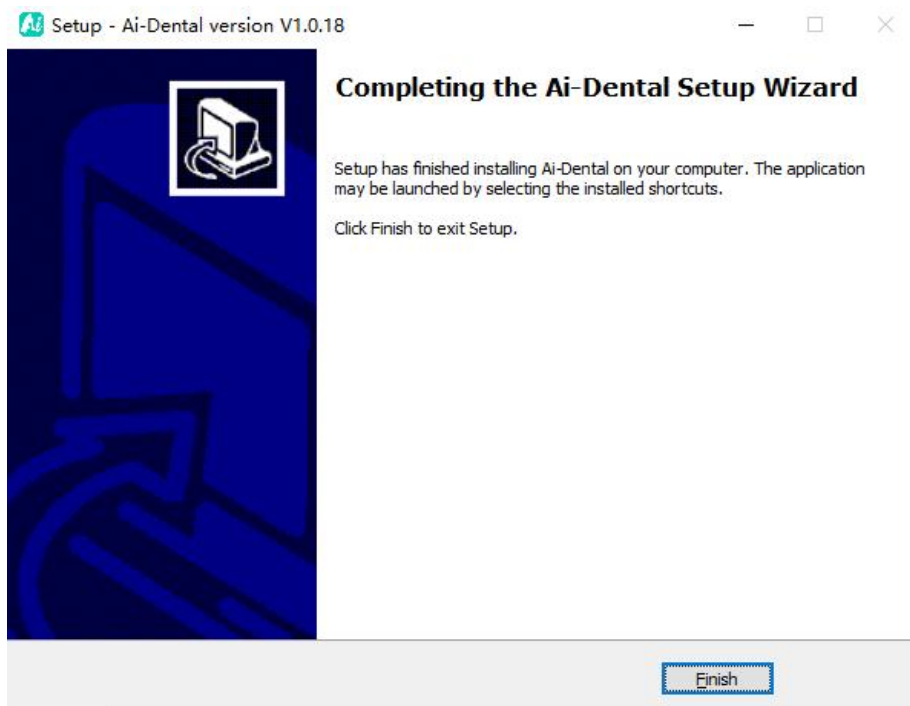


Figure 9

10. Open the Ai -Dental client, as shown in Figure 10



Figure 10

11. After starting the client, method 1 : Go to the computer of the host, find the server, open the interface, find the IP address, Figure 11. enter the IP address input box in the above pop-up window , click " Connection Test " , after the connection is successful, click " Modify " button to enter the login interface. Figure 1 2

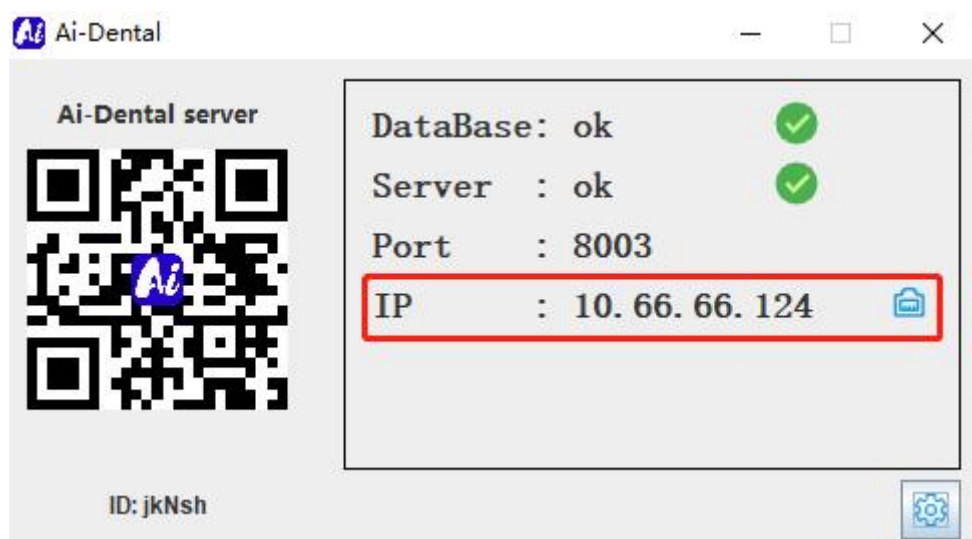


Figure 1 1

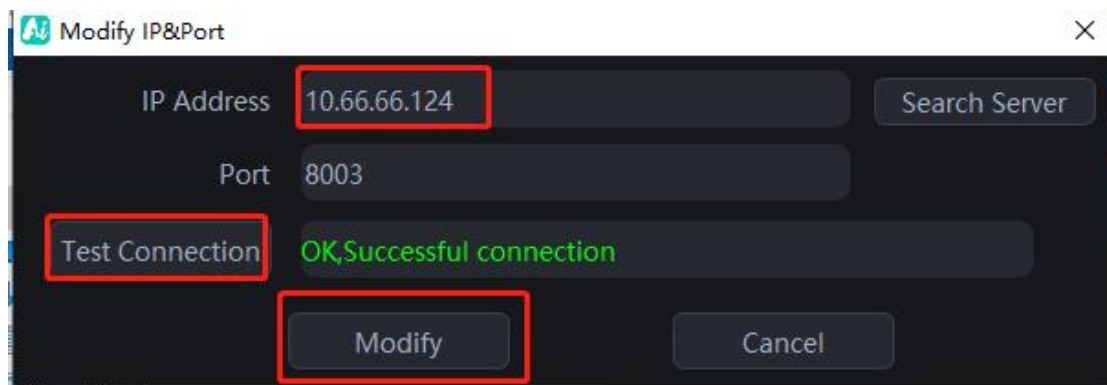


Figure 1 2

12. Method 2 : Click "Search Server" to search for the corresponding server IP address, click "OK", and then click "Connection Test". After the connection is successful, click the " Modify " button to enter the login interface. Figure 1 3 , 1 4

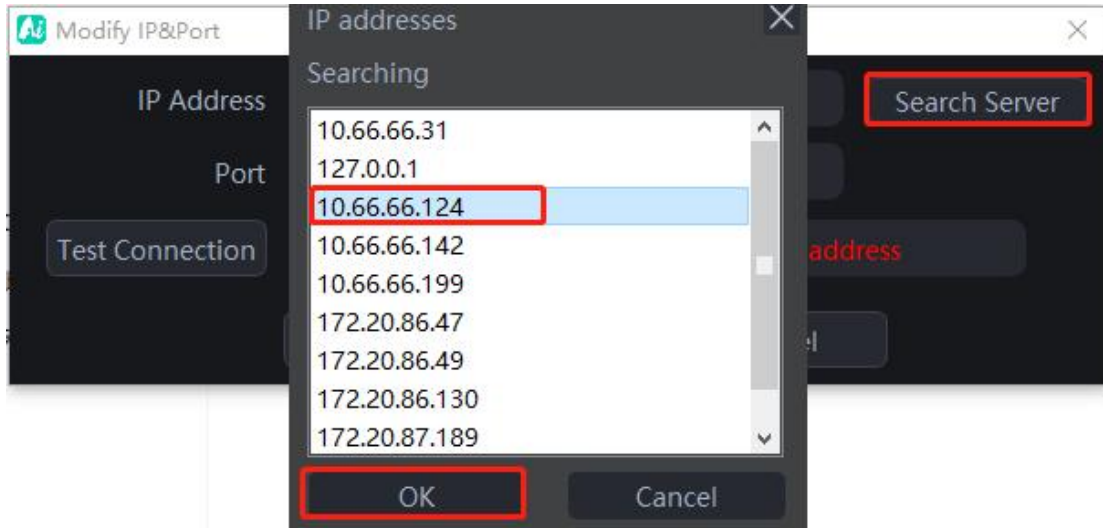


Figure 1 3

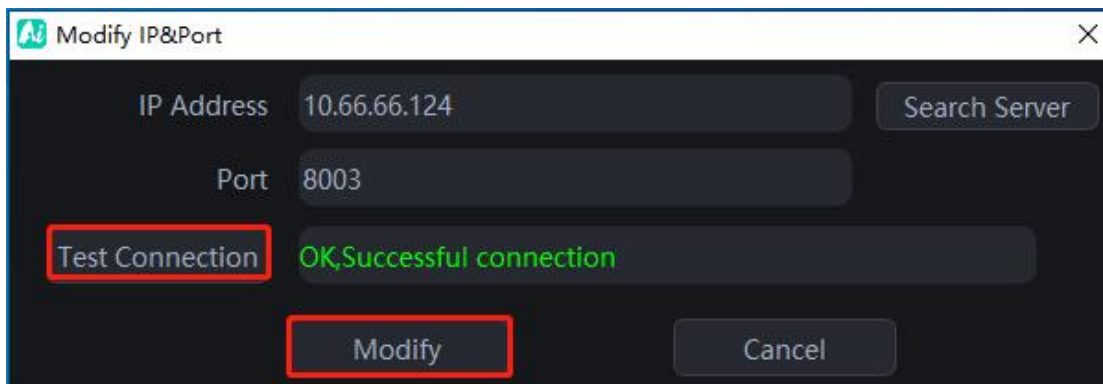


Figure 1 4