ES

Descripción:

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EN

Description:

Alpha-TCem™ NE is a non-eugenol temporary cement. **Alpha-TCem™ NE** is designed for temporary cementation of crowns, bridges, inlays and onlays. It offers strong adhesion to the tooth yet allows easy removal of the temporary restoration. Available in 7g cartridges.

Classification:

ISO 3107 Type I

Indications for Use:

Temporary cementation of provisional crowns, bridges, inlays, and onlays.

Warning:

Contains ingredients that may be irritating to the skin and eyes, see SDS.

General Precautions:

- U. S. Federal Law restricts this device to sale by or on the order of a dentist.
- Review instructions and SDS (www.dentaltech.com/resource) prior to beginning treatment. Use only as directed.

Working & Setting Times:

- Working Time is 2:00 Minutes
- Setting Time is 4:00 to 5:00 Minutes

Instructions for Use:

1. Insert the plunger into the back end of the cartridge.
 2. Remove cartridge cap by twisting it 90° counterclockwise and discard.
 3. When using a cartridge for the first time, extrude a "pea" size amount of material and discard. Use caution not to cross-contaminate orifices.
 4. Attach a new mixing tip by lining up internal cap stems with those on the cartridge. Twist mixing tip 90° clockwise to lock. The material is now ready to be dispensed directly with no hand mixing necessary.
 5. Clean, rinse, and dry preparation. Apply the desired amount of the mixed cement into the internal surfaces of the temporary restoration.
 6. Insert the temporary restoration within 2:00 minutes and attach to the dry, prepared tooth using slight pressure.
 7. Curing in the mouth takes approximately 4-5 minutes or less, and once complete remove any excess, unset cement with a wet cotton pellet or other instrument. Interdental excess is removed by flossing.
 8. Always leave mixing tip in place after use. It will serve as the cartridge cap.
- Storage:**
- Recommended storage conditions are between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (23°C/73°F). Do not freeze. See outer package for expiry date.
 - Keep cartridge closed when not in use.

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