

## INSTRUCTIONS AND SAFETY PRECAUTIONS

**WARNING:** To minimize the risk of injury from improper and unsafe use of this cane, read these instructions and safety precautions before using. Also, go to [HurryCane.com](http://HurryCane.com) and view the tips and tricks videos.

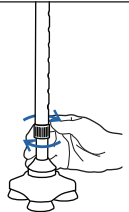
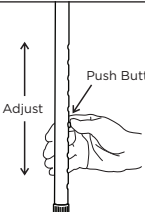
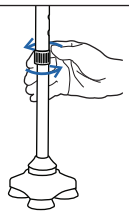

### Intended Use and Limitations:

- This cane is a walking aid and is not intended to absorb the full weight of the user.
- Not to be used by persons heavier than 350 lbs (159 kg).
- Not intended for use on ice or other slippery surfaces as it has the potential to slip.


### Safety Precautions:

- Consult a physician before using any assisted mobility device.
- Do not use unless push button is fully engaged.
- If the bottom base becomes loose, stop using cane immediately and contact HurryCane customer service.

## SETUP INSTRUCTIONS

<p><b>1 Release Lock</b> Turn left to unlock</p> 	<p><b>2 Adjust Height</b> After height adjust, make sure button is released in locked position <b>Note:</b> Cane will collapse if button is not fully engaged</p> 
<p><b>3 Tighten Lock</b> Tighten by turning right</p> 	<p><b>4 Ready for Use</b></p> 

## FEET CHECK INSTRUCTIONS

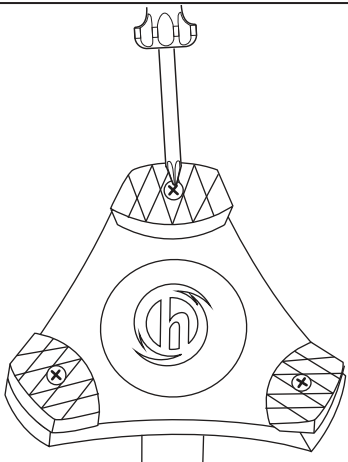

**!** Periodically check the screws on the feet to be sure they are tight.

## MAINTENANCE

- Periodically check the screws on the feet to be sure they are tight.
- Replace the feet every six months. See [HurryCane.com](http://HurryCane.com) to purchase new feet.
- Enhance your mobility at [HurryCane.com](http://HurryCane.com) for official HurryCane accessories.

### SPECIFICATIONS




- FDA-listed
- Eight adjustable heights:  
From 29 " to 38 "

### Comments, Feedback, or Questions

Toll-Free Number: 1-800-933-1136  
E-mail: [success@hurrycane.com](mailto:success@hurrycane.com)  
[HurryCane.com](http://HurryCane.com)  
Like us on Facebook: [facebook.com/AllTerrainCane](https://facebook.com/AllTerrainCane)

## RESET THE FLEX

When you want the cane to stand beside you without holding on to it, simply 'reset the flex' by following these 3 easy steps:

<p><b>1 Pivot</b></p>  <p>Keeping the base firmly on the ground, move handle in a full circle.</p>	<p><b>2 Center</b></p>  <p>Move the handle directly above the base.</p>	<p><b>3 Release</b></p>  <p>Carefully let go.</p>
--	---	---

NOTE: If the cane falls over, you can raise it easily by putting your foot on the base and pressing down.

## SURFACE TIPS

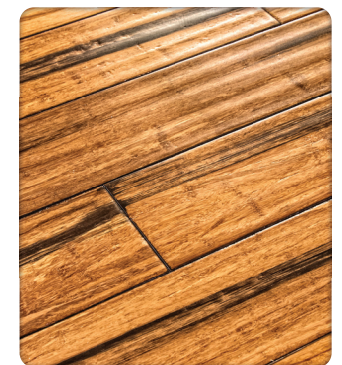
Although the HurryCane® is famous for providing superior stability on any terrain, it won't stand alone on uneven surfaces like these:



Plush Carpet



Throw Rugs



Uneven Wood Floors