

# PIG Surgical Floor Skimmer

**Collects and extracts fluid like blood and saline for safe and dry O.R. floors.**

**Features / Benefits:**

- **Shaped for efficiency** — captures liquids from all angles during high fluid surgeries for fast, easy cleanup
- **High-absorbency wicking pad** — proprietary bottom wicking pad removes 60 liters per hour of blood, saline and other fluids
- **Easy-slide floor pad** — easily reposition with your foot; keeps arms and hands free
- **Safe, Dry Floors** — helps maintain clean and dry O.R. floors to prevent risk of slips, trips and falls
- **Low-profile design** — it's so thin, the surgeon will never know it's there
- **Silent suction** — helps keep the operating room quiet
- **12-foot suction tubing** — long tubing length allows access to all areas around the O.R. table (included with HC200)



9" x 9" Footprint



Bottom Wicking Pad

**NOTE:** PIG Surgical Floor Skimmers are designed for one-time use. Please dispose of properly.

**PIG™ Surgical Floor Skimmer**

Item #	Description
HC200	10 Skimmers w/tubing
HC201	20 Skimmers


*Flip over for instructions >*




**New Pig**

## HOW TO USE


# PIG Surgical Floor Skimmer

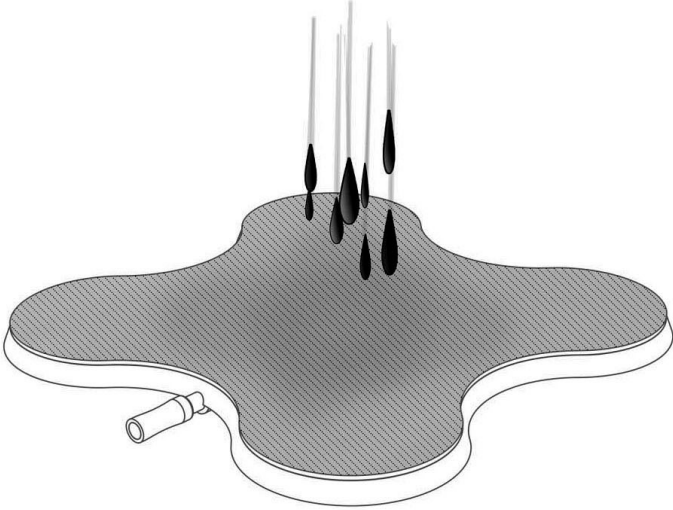


- 1.** Wet entire surface of bottom pad before use.

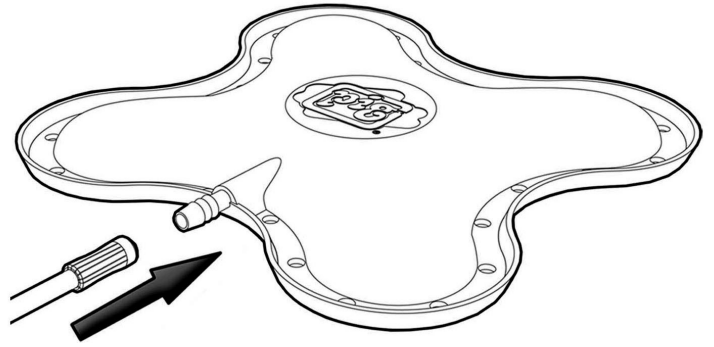

- 2.** Securely attach either end of suction tubing to barb fitting.

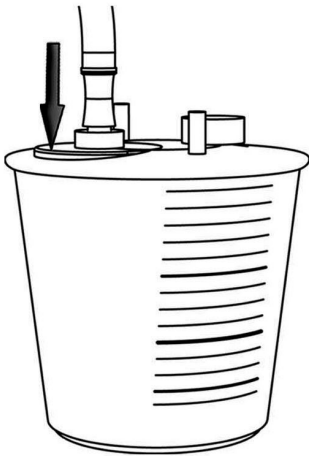

- 3.** Securely connect the other end of tubing to your fluid collection canister. Turn on your vacuum source when ready.

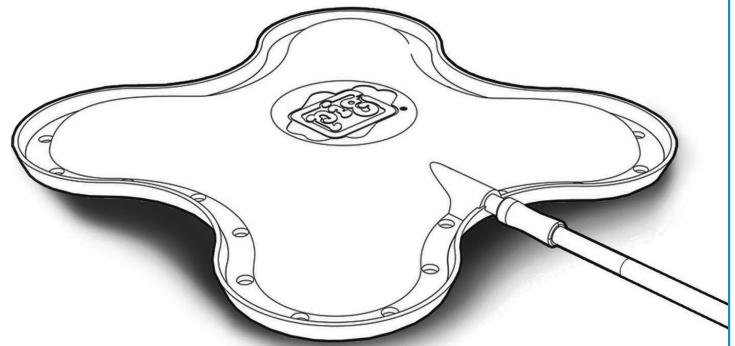

- 4.** Place skimmer wherever fluids are puddling — before, during or after a procedure.

**NOTE:** PIG Surgical Floor Skimmers are designed for one-time use. Please dispose of properly.



**New Pig**

*Flip over for product details >*