

PIG Surgical Absorbent Mat

Absorbs saline and blood quickly to keep O.R. floors dry and safe during high-fluid surgeries.

Features / Benefits:

- **Highly absorbent** — absorbs up to 3 liters per pad of blood, urine, vomit, saline and other O.R. fluids
- **Quick and efficient** — mat pads are more sanitary, faster and easier to use than towels, paper towels and other O.R. clean-up methods
- **Hygienic** — disposable mats keep surgical environment clean; helps reduce cross-contamination
- **Floor-gripping fibers** — lies flat on both wet and dry floors eliminating risk for falls; microfibers grip wet floor so mat doesn't slide around
- **Versatile** — mat pads can be placed on dry floors to catch liquids when they fall or placed on a wet floor to soak up liquids
- **Durable** — mat pads won't rip, tear or fray under light foot traffic or mayo stands and other carts
- **Perforated** — easily create smaller pads by tearing at perforations
- **Leakproof** — liquids will not soak through to the floor when saturated



NOTE: PIG Surgical Absorbent Mats are designed for one-time use. Please dispose of properly.

PIG® Surgical Absorbent Mat

Item #	Description
HC100	20 Mats • 30" x 40"
HC103	1 Roll • 30" x 150'
HC300	Roll Dispenser

HC300
(Mat not included)




Available in either rolls or mats

HC103



HC100

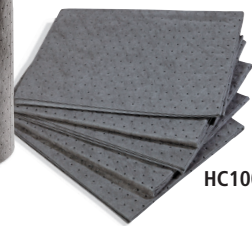

[Flip over for instructions >](#)



New Pig

PIG Surgical Absorbent Mat

PIG Surgical Absorbent Mat can be applied to your floor before, during or after procedures. Reposition mats as needed throughout the surgery.

1. Tear mat along perforations to fit your space.
2. Place mat on dry floor to catch liquids as they accumulate; place mat on wet floor to soak up existing liquids.
3. When mat is saturated, pick it up and throw it away.
4. Replace saturated mats with new mats until procedure is complete.

NOTE: PIG Surgical Absorbent Mats are designed for one-time use. Please dispose of properly.

WARNING: Used absorbents can take on the hazardous characteristics of liquids absorbed. Follow local, state and federal regulations when storing or disposing of used absorbents.



PIG Surgical Absorbent Mat absorbs high volumes of liquids to keep O.R. floors dry and safe.



New Pig

Flip over for product details >