Relief Insert® Walker 2.0

Relief Insert® Walker 2.0 Achill

Relief Insert® Walker 2.0

EN Lower Leg / Foot Orthosis
DE Unterschenkel-Fußorthese
ES Ortesis de pierna y pie
FR Orthèse de jambe-pied
IT Ortesi per piede-gamba
NL Onderbeen-voetorthese
DA Underbens-fodortose
SV Underben-fotortos
PL Orteza podudzia i stopy
SL Ortoza meča-stopalo
HR Ortoza za goljenicu i stopalo
PT Órtese de pé e perna inferior


Relief Insert® Walker 2.0 Achill

EN Lower Leg / Foot Orthosis with
Achilles tendon relief
DE Unterschenkel-Fußorthese mit
Achillessehnenentlastung
ES Ortesis de pierna y pie con
descarga del tendón de Aquiles
FR Orthèse de jambe-pied avec
décharge du tendon d'Achille
IT Ortesi per piede-gamba con
sgravio del tendine di Achille
NL Onderbeen-voetorthese met
ontlasting van de achillespees
DA Underbens-fodortose med
aflastning af achillesenen
SV Underben-fotortos med avlast-
ning för hälsenan
PL Orteza podudzia i stopy z
odciążeniem ścięgna Achillesa
SL Ortoza meča-stopalo z razbre-
menitvijo Ahilove tetive
HR Ortoza za goljenicu i stopalo s
rasterećenjem Ahilove tetive
PT Órtese de pé e perna inferior
com alívio do tendão de Aquiles

DARCO

Relief Dual® Plus

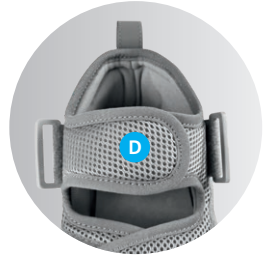Relief Insert® 2.0

E

EN Strap loops
 DE Umlenkschlaufen
 ES Lazos de desviación
 FR Boucles de sangles
 IT Passanti
 NL Doorhaallus
 DA Stropper
 SV Remögla
 PL Klamry do przewlekania
 SL Zanke
 HR Skretne omče
 PT Fivelas

H

EN Valve for releasing the air from the air chambers
 DE Ventil zum Ablassen der Luft aus den Luftkammern
 ES Válvula para desinflar las cámaras de aire
 FR Valve d'évacuation de l'air des alvéoles
 IT Valvola per lo scarico dell'aria dalle camere d'aria
 NL Ventiel waardoor lucht uit de luchtkamers kan ontsnappen
 DA Ventil til udledning af luft fra luftkamrene
 SV Ventil för att släppa ut luft ur luftkammarna
 PL Zaworek do spuszczenia powietrza z klamer powietrznych
 SL Ventil za izpuščanje zraka iz zračnih komor
 HR Ventil za ispuštanje zraka iz zračnih komora
 PT Válvula de saída do ar das câmaras de ar

K

EN Calf liner
 DE Waden-Liner
 ES Forro para pantorrilla
 FR Doublure du mollet
 IT Rivestimento del polpaccio
 NL Kuitvoering
 DA Læg-liner
 SV Vadliner
 PL Wkładka na łydce
 SL Vložek za meča
 HR Podstava za list
 PT Revestimento dos gêmeos

F

EN Forefoot strap
 DE Vorfußgurt
 ES Correa del antepié
 FR Sangle d'avant-pied
 IT Cinturino sull'avampiede
 NL Voorvoetband
 DA Forfodsrem
 SV Framfotsrem
 PL Pasek na przedniej części stopy
 SL Trak za sprednji del stopala
 HR Pojas za prednji dio stopala
 PT Correira de antepé

I

EN Calf splint with integrated air liner with air chambers
 DE Waden-Schiene mit fest integrierten Air-Liner mit Luftkammern
 ES Férula para pantorrilla con forro inflable integrado fijo y cámaras de aire
 FR Attelle de mollet avec Air-Liner intégré et alvéoles
 IT Stecca per polpaccio con Air-Liner integrato fisso e dotato di camere d'aria
 NL Kuitbrace met vast geïntegreerde air-liner met luchtkamers
 DA Læg-skinne med fast integreret Air-liner med luftkamre
 SV Vadskenamed fast Air-Liner med luftkammare
 PL Szyna łydkowa z umocowaną na stałe wkładką powietrzną z komorami powietrznymi
 SL Opornica za meča s fiksno integriranim vložkom Air-Liner z zračnimi komorami
 HR Udlaga za list s čvrsto ugrađenom zračnom podstavom sa zračnim komorama
 PT Tala dos gêmeos com revestimento Air integrado de forma fixa com câmaras de ar

L

EN Air liner with air chambers
 DE Air-Liner mit Luftkammern
 ES Forro inflable con cámaras de aire
 FR Air-Liner avec alvéoles
 IT Air-Liner con camere d'aria
 NL Air-liner met luchtkamers
 DA Air-Liner med luftkamre
 SV Air-Liner med luftkammare
 PL Wkładka powietrzna z komorami
 SL Air-Liner z zračnimi komorami
 HR Zračna podstava sa zračnim komorama
 PT Revestimento Air com câmaras de ar

G

EN Pump to fill the air liner air chambers
 DE Pumpe zum Befüllen der Luftkammern des Air-Liners
 ES Bomba para inflar las cámaras de aire del forro inflable
 FR Pompe de gonflage des alvéoles de l'Air-Liner
 IT Pompa per il riempimento delle camere d'aria dell'Air-Liner
 NL Pomp voor het vullen van de luchtkamers van de air-liner
 DA Pumpe til at fylde luftkamrene i Air-lineren
 SV Pump för påfyllning av luftkammarna i Air-Liner
 PL Pompka do napełniania komór powietrznych wkładki powietrznej
 SL Črpalka za polnjenje zračnih komor na vložku Air-Liner
 HR Pumpa za punjenje zračnih komora zračne podstave
 PT Bomba para encher as câmaras de ar do revestimento Air

J

EN Automatic modification of the width: For unhooking, if necessary
 DE Automatische Weitenregulierung: Im Bedarfsfall zum Aushaken
 ES Regulación automática de la anchura: Se puede desenganchar en caso necesario
 FR Réglage automatique de la largeur : Si nécessaire pour décrocher
 IT Regolazione automatica della larghezza: Se necessario per sganciare
 NL Automatische wijzteaanpassing: Waar nodig los te haken
 DA Automatisk bredderegulering: Kan hæftes af efter behov
 SV Automatisk viddreglering: kan hakas loss vid behov
 PL Automatyczna regulacja szerokości: W razie potrzeby można rozpiąć
 SL Samodejna regulacija širine: Za odpenjanje po potrebi
 HR Automatska regulacija širine: Može se otkopčati po potrebi
 PT Regulação automática em largura: Se necessário, para desenganchar

M

EN Front splint with front liner
 DE Front-Schiene mit Front-Liner
 ES Férula frontal con forro frontal
 FR Attelle avant avec doublure avant
 IT Stecca anteriore con rivestimento anteriore
 NL Frontbrace met frontvoering
 DA Front-skinne med Front-liner
 SV Frontskena med frontliner
 PL Szyna frontowa z wkładką frontową
 SL Sprednja opornica s sprednjim vložkom
 HR Prednja udlaga s prednjom podstavom
 PT Tala dianteira com revestimento dianteiro

Relief Insert® Walker 2.0 | Lower Leg / Foot Orthosis
Relief Insert® Walker 2.0 Achill | Lower Leg / Foot Orthosis with Achilles tendon relief

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities. Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

The Relief Insert® Splinting System may only be used in combination with the Relief Dual® Plus Off-loading Shoe for walking. The splint is not designed for walking without a shoe. There is a risk of thrombosis due to the restricted ankle movement. There is a risk that the orthosis (splint) may break if the load is too heavy. Max. permitted load: 150 kg. The air chambers must not be over inflated to prevent pressure points on the leg and to prevent the air chambers from bursting. The air liner with pump cannot be washed.

Purpose

Relief Insert® Walker 2.0: Lower leg / foot orthosis for immobilisation in a given position, as well as a foot-positioning orthosis

Relief Insert® Walker 2.0 Achill: Lower leg / foot orthosis and foot-positioning orthosis with Achilles tendon relief

Design features

The lower leg / foot orthosis is available in five sizes with a universal fit for right and left.

The shoe can be customized to the left or right foot by individually adjusting the hook and loop straps. The lower leg / foot orthosis consists of the following components: One shoe (Relief Dual® Plus | Off-loading shoe with high ankle support), a splint (Relief Insert® 2.0 | Splinting System) with a grey air liner and integrated pump and a blue lining (consists of a forefoot strap, front and calf liner). The Relief Insert® Walker 2.0 Achill also includes heel wedges (DAW | DARCO Achilles Wedges).

The splinting system is used to position the lower leg / foot. In addition, a walker can be assembled from the splinting system using a therapeutic shoe to immobilise the lower leg / foot with simultaneous mobilisation. The exchangeable insole enables patients to individually adjust the fit.

Maintenance

The product does not require maintenance.

Relief Insert® Walker 2.0

Indications

- > Arthrodesis of the first TMT joint (lapidus arthrodesis)
- > After metatarsal fracture surgery
- > After hallux valgus surgery
- > After hallux rigidus surgery
- > After hammer and claw toe surgery
- > Fractures of the phalanges
- > After surgery on the rearfoot
- > For ligament rupture of the ankle joint
- > Postoperative use after removal of Haglund's exostosis (with intact Achilles tendon)
- > Pre- and postoperative / post-traumatic immobilisation of the lower leg / foot, stable foot or ankle fractures, severe ankle sprains
- > Upper ankle joint arthrodesis
- > Postoperatively after Weber fractures

Contraindications

- > Acute Weber fractures (B, C)
- > Unstable fractures

Relief Insert® Walker 2.0 Achill

Indications

- > Achilles tendon ruptures
- > Postoperative use for Achilles tendon suture
- > Postoperative use after removal of Haglund's exostosis

Contraindications

- > Forefoot conditions
- > Plantar foot ulcers

Combination with other products

- > PegAssist® Insole
- > Puzzle Insole®
- > PegContour® Insole (blue)
- > Relief Contour Insole
- > PegContour® Insole Uni
- > Toe Cap
- > DAW (DARCO Achilles Wedges)
- > Heel Wedges
- > Relief Insert® Walker 2.0 Lining

Technical data

Sizes

- XS 37.0 – 38.5
- S 39.0 – 41.0
- M 41.5 – 43.0
- L 43.5 – 45.0
- XL 45.5 – 47.0

Colour

Grey-Blue

Material

Outsole: Rubber, EVA

Insole: EVA

Upper material of shoe: Polyester foam, microfibre

Lining and forefoot strap: Polyester

Hook and loop straps: Nylon

Adhesive: Polyurethane glue

Splints: Polyamide

Pump: TPU

DAW (heel wedges): EVA

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

To avoid the hook and loop straps getting in the way during the individual application steps, attach them behind onto the respective fleece side.

Using the Relief Insert® 2.0 as a splinting system:

When used as a splinting system, the Relief Dual® Plus Off-loading Shoe is not required. Set the shoe aside and proceed with opening the splinting system.

1. Fold the front splint away to the side.
2. If desired, another compatible DARCO insole can be placed in the splinting system. To do this, remove the grey insole. Then place and fix the desired insole analogously to the grey insole in the splinting system.
3. Place the leg on the calf liner of the calf splint. Make sure that the foot is in contact with the insole.
4. The width of the guide splint automatically adjusts to the patient's needs by means of the adjuster on the back of the guide splint.
5. Once the leg is positioned, close the splinting system. To do this, place the front splint on the shin. Make sure that the front splint is positioned centrally within the calf splint. Guide the three hook and loop straps through the loops. Pull the hook and loop straps as tight as necessary. Ensure that the leg is supported comfortably. Optionally, the blue forefoot strap (F) can be attached to the forefoot for additional support.

Using the Relief Insert® Walker 2.0 as a lower leg / foot orthosis:

Perform steps 1 – 5 as described. After that, please continue as follows.

6. Let the patient step into the opened foot section of the Relief Dual® Plus Off-loading Shoe with the orthosis (splint) on. For particularly easy entry, open the Relief Dual® Plus completely using the hook and loop straps (A, B, C). Make sure that the insole is positioned securely at the front in the shoe.
7. Now close the off-loading shoe with the two hook and loop straps C of the heel cap.
8. The inner shoe tabs can be placed over each other either way (D).
9. Close the two hook and loop straps A, B as described below. Fold the strap A on the outer side (left or right) around the tab and secure it firmly. Close the shoe with the strap on the other side. If needed, the length of the upper hook and loop strap A can be customised to fit. Now close the shoe with the lower hook and loop fastener B such that the DARCO logo is on the outside of the shoe.
10. To achieve additional stabilisation of the ankle joint, the volume of the air chambers can be increased using the pump attached to the side. To reduce the volume, release the air using the silver valve. This allows for individual filling and adjustment. Ensure that the leg is supported comfortably. If impaired sensation, swelling or pain occurs, the pressure should be reduced by releasing air from the chambers using the valve. Should persistent pain occur, the treatment should be interrupted and a doctor should be consulted.

Using the Relief Insert® Walker 2.0 Achill as an Achilles walker:

11. Fold the front splint to the side.
 12. Leave the insole in the splint.
 13. Place the wedges in the heel area of the splint on the insole, corresponding to the degree of plantar flexion that is therapeutically required. A wedge has a height of approx. 11 mm. Depending on the number of wedges used, the height varies and the angle of plantar flexion can, thus, be adjusted individually. 3 wedges are equivalent to approx. 120° slant.
 14. Fix the wedges to the insole using the adhesive dots.
 15. Depending on the therapy course, the wedges can be removed and, thus, the height adjusted.
- Once the wedges are positioned, please carry out steps 3 to 10.

Changing the lining

Removing the lining:

16. Remove the blue calf liner attached to the calf splint with hook and loops.
17. The grey air liner with integrated air chambers and pump remains in the product and must not be removed.
18. Now carefully remove the blue front liner attached to the front splint with hook and loop fasteners.
19. The calf liner, front liner and forefoot strap can be washed, if required.

Fitting the lining:

After cleaning the blue lining, please carry out the steps described below:

20. First attach the front liner to the front splint using the hook and loop fasteners. To do this, fold the liner in the middle lengthwise and place it centrally (according to the shape of the front splint) on the front splint. By unfolding the liner, it clings to the front splint in the correct position. Gently press the front liner against the splint.
21. The calf liner is also folded lengthwise in the middle for attachment to the splint. Place the calf liner in the centre of the calf splint with the notches at the level of the ankle. By unfolding the liner, it clings to the calf splint in the correct position. Gently press the calf liner.

To close the splinting system or walker, follow the steps described from step 1.

Note

The Relief Insert® Splinting System may only be used in combination with the Relief Dual® Plus Off-loading Shoe for walking. The splint is not designed for walking without a shoe.

Cleaning instructions

Washing

Shoe, splint, grey insole, wedges:

If these are slightly soiled, wipe with a damp cloth or clean with commercially available disinfectants.

Liners:

To wash, the blue calf and front liners are removed from the splint and then washed at approx. 60° C in a washing machine. The optional forefoot strap can also be washed at 60° C. The air liner with air chambers and the pump cannot be washed.

Drying

Do not dry any of the components of the Relief Insert® Walker 2.0 | Achill in a dryer or on a radiator, but dry them at room temperature.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

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ALBO-Healthcare GmbH