

AMPHIBIOUS TOURNIQUET®

Gen 2 Features

- Wearable, Rugged, Lightweight, & Compact
- Ocean-Rated, 100% Marine-Grade
- Self-Locking, & Non-Slip
- Bite-Strap to Aid 1-Handed Applications
- Instructions For Use (IFU) on Exterior
- Place to Record TQ Application Time
- Marine Ratcheting Buckle - Stainless Steel
- Ratchet Max Load 550 lbs (250 Kgs)
- Designed for Repetitive Ocean / Training Use

MEDICAL DEVICE SYMBOL LEGEND

SYMBOL

DESCRIPTION



MANUFACTURER: Indicates the medical device manufacturer.



CAUTION: Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.



DO NOT REUSE: Indicates a medical device that is intended for one use or for use on a single patient during a single procedure.



CE MARKING: Signifies European technical conformity.



AUTHORIZED EUROPEAN REPRESENTATIVE: The requirements for accreditation and market surveillance relating to the marketing of products; Medical Device Directive.



BATCH CODE: Indicates the manufacturer's batch code so that the batch or lot can be identified.



Preventative Maintenance:

- Fresh Water rinse after each use in or exposure to saltwater
- Inspection & Testing of components should be performed regularly
- Familiarization should be performed regularly & before actual application to reduce the effects of stress

Contraindications:

- Do not use for minor bleeding
- Do not apply on a joint
- Do not apply over items in pockets



To register your Tourniquet & validate your warranty log on at www.omnainc.com/pages/contact

Manufactured for:

Manufacturer Symbol

OMNA Inc.

1400 63rd Ave S

St. Petersburg, FL, 33705, USA

Phone: +1 727.222.6429 Email: sales@omnainc.com

Patents: www.omnainc.com


EC	REP	Australian Sponsor
EMERGO EUROPE Prinsessgracht 20 The Hague, 2514 AP, The Netherlands		EMERGO AUSTRALIA Level 20, Tower II Darling Park, 201 Sussex St Sydney, NSW, 2000, AU

Severe Bleeding on Arm or Leg:

Apply OMNA Tourniquet

Two-Handed Application

1. **PULL** - removing stored strap
2. **Position High & Tight - WRAP** around limb
3. **Route strap through the red ring**
4. **PULL Down Tight** - removing all slack
5. **LIFT** ratchet lever until bleeding stops and distal pulse eliminated

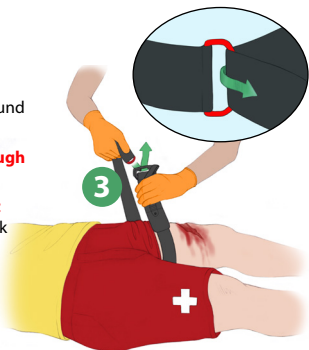

One-Handed Application

1. **Position High & Tight**
May also be applied 2" – 3"
(5 – 8 cm) above the wound



2. **Bite - bite strap**
Pull Down Tight - removing all slack



3. **Velcro**



4. **Confirm Slack Removed; Re-Tighten As Needed**



5. **LIFT Ratchet Lever until bleeding stops & pulse below tourniquet cannot be felt.** If bleeding is not controlled or distal pulse is present add more compression, or apply another tourniquet above and/or side-by-side the first

6. **Record time of tourniquet application. Seek medical attention**

