

# OMNA MARINE TOURNIQUET®

## Gen 2 Features

- Rugged, Lightweight, & Compact
- Ocean-Rated, 100% Marine-Grade
- Self-Locking, & Non-Slip
- Bite-Strap to Aid 1-Handed Applications
- Instructions For Use (IFU) on Exterior
- Place to Record TQ Application Time
- Marine Ratcheting Buckle - Stainless Steel
- Ratchet Max Load 550 lbs (250 Kgs)
- Designed for Repetitive Ocean / Training Use

### MEDICAL DEVICE SYMBOL LEGEND

#### SYMBOL

#### DESCRIPTION



**MANUFACTURER:** Indicates the medical device manufacturer.



**CAUTION:** Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.



**DO NOT REUSE:** Indicates a medical device that is intended for one use or for use on a single patient during a single procedure.



**CE MARKING:** Signifies European technical conformity.



**AUTHORIZED EUROPEAN REPRESENTATIVE:** The requirements for accreditation and market surveillance relating to the marketing of products; Medical Device Directive.



**BATCH CODE:** Indicates the manufacturer's batch code so that the batch or lot can be identified.



### Preventative Maintenance:

- Fresh Water rinse after each use in or exposure to saltwater
- Inspection & Testing of components should be performed regularly
- Familiarization should be performed regularly & before actual application to reduce the effects of stress

### Contraindications:

- Do not use for minor bleeding
- Do not apply on a joint
- Do not apply over items in pockets



To register your Tourniquet & validate your warranty log on at [www.omnainc.com/pages/contact](http://www.omnainc.com/pages/contact)

#### Manufactured for:

Manufacturer Symbol

**OMNA Inc.**

1400 63rd Ave S

St. Petersburg, FL, 33705, USA

Phone: +1 727.222.6429 Email: [sales@omnainc.com](mailto:sales@omnainc.com)

Patents: [www.omnainc.com](http://www.omnainc.com)


EC	REP	Australian Sponsor
EMERGO EUROPE Prinsessgracht 20 The Hague, 2514 AP, The Netherlands		EMERGO AUSTRALIA Level 20, Tower II Darling Park, 201 Sussex St Sydney, NSW, 2000, AU

# Severe Bleeding on Arm or Leg:

## Apply OMNA Tourniquet

### Two-Handed Application

1. **PULL** - removing stored strap
2. **Position High & Tight - WRAP** around limb
3. **Route strap through the ring**
4. **PULL Down Tight** - removing all slack
5. **LIFT** ratchet lever until bleeding stops and distal pulse eliminated

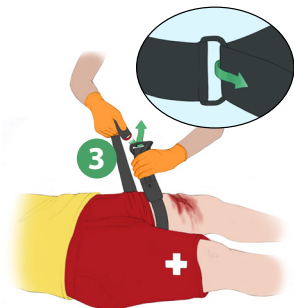

### One-Handed Application



1. **Position High & Tight**  
May also be applied 2" – 3"  
(5 – 8 cm) above the wound



2. **Bite - bite strap**  
**Pull Down Tight** - removing all slack



3. **Velcro**



4. **Confirm Slack Removed;**  
**Re-Tighten As Needed**

5. **LIFT Ratchet Lever until bleeding stops & pulse below tourniquet cannot be felt.** If bleeding is not controlled or distal pulse is present add more compression, or apply another tourniquet above and/or side-by-side the first

6. **Record time of tourniquet application.**  
**Seek medical attention**



**To release / reset tourniquet for training.**  
Pull red button upwards.

