

EASY SETUP

FREQUENCY PRECISION[®]
SENSORS & PAGERS

Bed Pressure Mat - Quickstart Guide

1 - Unroll Bed Mat

Open bed mat packet and lay mat flat - wait 20mins to inflate

2 - Insert batteries

Insert 2x AA batteries in back of control box.

3 - Set Switch

Move selector on control box to BED.

4 - Position Control Box

Use rubber case and straps or place in safe position.

5 - Connect Air Tube

Connect black air pipe/tube between control box and mat.

6 - Place Bed Mat

Position under mattress.

7 - Finally...

FOR CALL SYSTEM

Connect Call System Lead between Control Box and call point. Test Mat. It will alert through existing call system.

OR

FOR PAGER

Test Pager by pressing yellow button on control box, if the pager is on it should alert.