

Instructions for use

Urine test strips

Early detection of a Urinary Track Infection (UTI)



1. Collect a fresh sample of urine in a clean vessel.
2. Dip a test strip into the urine for 1 second, ensuring all test squares are covered.
3. Test strip will show test results within 60 seconds.
4. Compare the used test strip to the strip on the container.

Results:

LEU - a purple colour suggests a possible infection - Positive UTI.

NIT - a pink colour suggests a possible bacterial infection - Positive UTI.

UBG & ERY - indicators represent blood in urine, if significant change consult a doctor.

LEU and NIT changes indicate a UTI, **you must seek medical attention promptly.**

Note: for a detailed review consult insert leaflet.

