

INSTRUCTIONS FORM

FOR FILLING TICKER® TRANSDERMAL APPLICATORS

Wear appropriate protective equipment as determined by your company, USP 800, or other regulating bodies when handling HDs. Gowns; head, hair, and shoe covers; and two pairs of chemotherapy gloves are required for both sterile and non-sterile HDs.



1
FILL

Step 1. Fill the applicator using a spatula, EMP jar or other filling system. Tap the applicator against the counter to settle the cream uniformly while minimizing air pockets. Remember to add additional product to compensate for residual volume.




2
ALIGN

Step 2. Press the *dispenser lid* (cap with hole) on the top of the applicator by aligning the insertion guides with the slots on the top of the chamber and snap-on tight until it clicks shut.



3
PRIME

Step 3. Prime the applicator by turning the *base* clockwise until the cream is dispensed through the outlet. Continue to prime the applicator to set a proper starting point for the patient. Wipe-off any excess amount dispensed. *Each Click dispenses 0.05 mL*

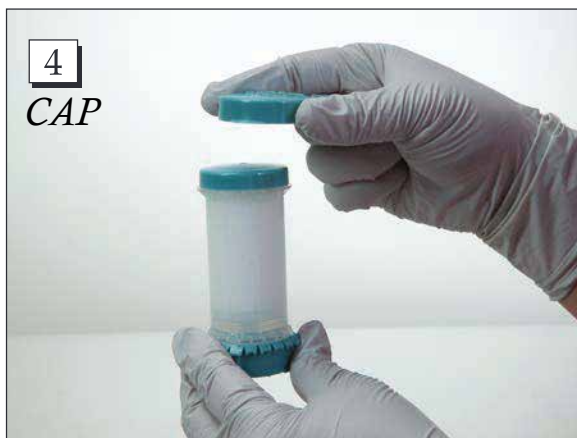

4
CAP

Step 4. Press the *safety cap* until it snaps-on to a closed position. If the cap doesn't fasten, it may be an indication that the dispenser lid (in step two) was not snapped-on properly.

SPECIFICATIONS:

Capacity: 37 mL
Dispensation/CLICK: 0.05 mL
Residual Volume: 2.5 mL
Packaging: 40 units per BOX
SKU: T37-Teal-40

Featured above: Teal TICKER Classic

TICKER Filling Instructions X.01

CLICK to ML Conversion Chart for TICKER, TICKER Unlimited, and TICKERmini

| | | | |
|--------------------|---------------------|---------------------|---------------------|
| 1 CLICK = 0.05 mL | 6 CLICKS = 0.30 mL | 11 CLICKS = 0.55 mL | 16 CLICKS = 0.80 mL |
| 2 CLICKS = 0.10 mL | 7 CLICKS = 0.35 mL | 12 CLICKS = 0.60 mL | 17 CLICKS = 0.85 mL |
| 3 CLICKS = 0.15 mL | 8 CLICKS = 0.40 mL | 13 CLICKS = 0.65 mL | 18 CLICKS = 0.90 mL |
| 4 CLICKS = 0.20 mL | 9 CLICKS = 0.45 mL | 14 CLICKS = 0.70 mL | 19 CLICKS = 0.95 mL |
| 5 CLICKS = 0.25 mL | 10 CLICKS = 0.50 mL | 15 CLICKS = 0.75 mL | 20 CLICKS = 1.0 mL |

*Conversion chart based accuracy studies for commonly used HRT Bases sold across the USA to compounding pharmacies. The results obtained are average values obtained from HRT bases with similar densities.

INSTRUCTIONS FORM

FOR APPLYING TRANSDERMAL PREPARATIONS W/ TICKER[®]



Step 5. Remove the safety cap, then turn the base clockwise to dispense the dosage. Use the proper number of *Clicks* as instructed by the practitioner. Note: After dialing the dosage, wait 5-10 seconds to give ample time for the product to exit through the chamber.



Step 6. Always follow the doctor's or pharmacist's instructions on how to apply the medication. Apply the cream/ointment directly with the applicator by rubbing the dome applicator pad onto the recommended sites in a semi-circular motion or as directed.



Note: Prevent evaporation of the next dose by snapping on the safety cap completely. (TIP: Some of the water that makes the bulk of the preparation may evaporate without the lid).

Featured above: Teal TICKER Classic

| CLICK to ML Conversion Chart for TICKER, TICKER Unlimited, and TICKERmini | | | |
|---|---------------------|---------------------|---------------------|
| 1 CLICK = 0.05 mL | 6 CLICKS = 0.30 mL | 11 CLICKS = 0.55 mL | 16 CLICKS = 0.80 mL |
| 2 CLICKS = 0.10 mL | 7 CLICKS = 0.35 mL | 12 CLICKS = 0.60 mL | 17 CLICKS = 0.85 mL |
| 3 CLICKS = 0.15 mL | 8 CLICKS = 0.40 mL | 13 CLICKS = 0.65 mL | 18 CLICKS = 0.90 mL |
| 4 CLICKS = 0.20 mL | 9 CLICKS = 0.45 mL | 14 CLICKS = 0.70 mL | 19 CLICKS = 0.95 mL |
| 5 CLICKS = 0.25 mL | 10 CLICKS = 0.50 mL | 15 CLICKS = 0.75 mL | 20 CLICKS = 1.0 mL |

**Conversion chart based accuracy studies for commonly used HRT Bases sold across the USA to compounding pharmacies. The results obtained are average values obtained from HRT bases with similar densities.*