



# PRE-MOISTENED ISOPROPYL ALCOHOL WIPES

ALCOWIPES ARE A DISINFECTANT WIPE FOR USE ON HARD SURFACES THAT OTHERWISE THE DISINFECTION WILL NOT BE EFFECTIVE. USE CLOTH IMMEDIATELY AFTER REMOVING FROM THE CONTAINER WHILST STILL MOIST.

**CAN BE USED FOR CLEANING OF:**

- ALL HARD SURFACES IN PATHOLOGY, DENTAL CLINIC AND GP SURGERIES, HOSPITALS AND LABORATORIES
- STAINLESS STEEL BENCHES
- TILE SURFACES
- SINKS
- EQUIPMENT SURFACES
- TELEPHONES
- FOR USE BY DENTIST, PHYSIOTHERAPIST, GP'S
- OPTOMETRISTS

**PRECAUTIONARY STATEMENTS:**

KEEP OUT OF REACH OF CHILDREN, HIGHLY FLAMMABLE, KEEP AWAY FROM SOURCES OF IGNITION.

CONTACT WITH EYES MAY CAUSE IRRITATION, IN CASE OF CONTACT, RINSE EYES IMMEDIATELY WITH WATER AND IF IRRITATION PERSISTS SEEK MEDICAL ADVICE.

**CONTAINS:** ISOPROPYL ALCOHOL 70% V/V

**STORAGE:** STORE IN A WELL-VENTILATED PLACE. KEEP CONTAINER TIGHTLY CLOSED. PROTECT FROM SUNLIGHT, KEEP COOL

**PRODUCT SPECIFICATIONS:**

- 75 WIPES - 140MM X 170MM

**DIRECTIONS FOR USE:**

1. OPEN THE LID
2. REMOVE THE FOIL COVERING
3. USE THE TAB THAT IS PART OF THE LID TO PUSH THE WIPE THROUGH THE SLIT IN THE CENTRE OF THE LID
4. TO TEAR OFF ONE SHEET PULL STRAIGHT UP AND THEN TO THE SIDE



RE-ORDER CODE

**171436**

PUREGIENE AUSTRALASIA

freecall: 1800 177 399

email: [sales@puregiene.com.au](mailto:sales@puregiene.com.au)

[www.puregiene.com.au](http://www.puregiene.com.au)

**abco**