

REPAIR-IT® DENTURE REPAIR KIT

DIRECTIONS FOR USE

IMPORTANT: Read all directions and cautions carefully before starting.

- **Do Not use for loose caps or lost fillings. (Use Dentemp OS).**
- **Do Not Use On a Valplast Denture.**
- **Warning: Contains methacrylates. Do not use if allergic or sensitive to methacrylates.**
- **Warning: Denture repair liquid is flammable. Keep away from heat, open flames or sparks.**
- **Warning: Avoid getting denture repair liquid on skin. If you do, wash area with water immediately.**
- **Caution: Do not put repaired denture back into your mouth until the denture repair material has hardened.**
- **Caution: Fit pieces together properly. Once this product hardens it can not be removed.**
- **Hint: If, while mixing, the material hardens too fast for the mixture to become smooth and creamy then please try again but first put the liquid into a freezer for ½ hour to slow hardening.**

Directions for use

For Broken Dentures:

1. Clean and dry the broken edges of the denture. Make sure there are no missing pieces. If you cannot do this, the denture will not fit when repaired. Do not proceed. See your dentist immediately.
2. Mix denture repair liquid and powder as follows:
 - a. Place contents of one packet into one of the enclosed mixing cups.
 - b. Remove the cap from the tube, reverse it and use the point to open the tip of the tube. Add 22 drops of liquid to the powder in the mixing cup. Be careful not to get any denture repair liquid onto your skin.
This liquid is flammable. Do not use near sparks or an open flame. Use in a well ventilated area.
 - c. Screw cap onto tube tightly to store liquid for future repair.
 - d. Using the wooden spatula, mix and if necessary, continue adding 1 drop at a time until the mixture becomes smooth with a creamy consistency, like syrup.
3. Using the wooden spatula apply a small amount of the denture repair mixture to each broken edge of the denture. Make sure that each broken edge has a thin coating of the denture repair mixture.
4. Immediately fit the broken pieces of the denture together. Make absolutely sure that the broken pieces of the denture fit together properly and are in correct alignment.
5. Press the pieces together and hold in place for five (5) minutes. Be sure that the pieces stay in perfect alignment.
6. Carefully scrape away any extra material that has squeezed out from between the pieces of the broken denture.

7. Soak repaired denture in hot tap water (not boiling) for 10 minutes.
8. Remove denture from water. Dry carefully. Using sandpaper, smooth out any rough spots.
9. Wash denture with soap and water. Clean wooden spatula and put back in kit. Wash hands thoroughly.

For Cracked But Not Completely Broken Dentures:

10. Mix liquid and powder as in number 2 above.
11. Using the wooden spatula apply a small amount of the denture repair mixture directly on the crack on both sides of the denture.
12. Allow to set for (5) minutes.
13. Carefully scrape away any extra material.
14. Soak repaired denture in hot tap water (not boiling) for 10 minutes.
15. Remove denture from water. Dry carefully. Using sand paper, smooth out any rough spots.
16. Wash denture with soap and water. Clean wooden spatula and put back in kit. Wash hands thoroughly.

For Loose or Broken Denture Teeth:

17. Clean and dry the missing tooth area on the denture and the denture tooth.
18. Make sure the tooth fits correctly back into the denture.
19. Mix the denture repair material as in step 2 above.
20. Place the denture repair material into the missing tooth area and press the tooth into position.
21. Allow tooth to set for one (1) minute.
22. Remove excess material carefully using a wet cotton tipped applicator.
23. Allow tooth to set for an additional four (4) minutes.
24. Soak repaired denture in hot tap water (not boiling) for 10 minutes.
25. Remove denture from water. Wash denture with soap and water. Clean wooden spatula and put back in kit. Wash hands thoroughly.

NOTE: After repairs have been completed, if for any reason the denture is uncomfortable or does not fit, do not use it. See your dentist immediately.

IMPORTANT: Special training and tools are needed to repair dentures to fit properly. Home-repaired dentures may cause irritation to the gums and discomfort and tiredness while eating. Long-term use may lead to more troubles, even permanent changes in bones, teeth and gums, which may make it impossible to wear dentures in the future. For these reasons, dentures repaired with this kit should be used only in an emergency until a dentist can be seen. Dentures that don't fit properly cause irritation and injury to the gums and faster bone loss, which is permanent. Dentures that don't fit properly cause gum changes that may require surgery for correction. Continuing irritation and injury may lead to tumors in the mouth. You must see your dentist as soon as possible.

WARNING: For emergency repairs only. Long-term use of home repaired dentures may cause faster bone loss, continuing irritation, sores and tumors. This kit is for emergency use only. See dentist without delay. A new denture may be required.

Any problems or questions - call us at (800) 238-0220 or (845) 436-0011,
M-TH, 8:00 AM to 4:00 PM E.T., FRIDAY, 8:00 AM to 3:00 PM E.T.

e-mail us at: Custserv@majesticdrug.com or visit our web site at www.majesticdrug.com

KEEP OUT OF THE REACH OF CHILDREN

© 2010 Majestic Drug Co., Inc., P.O. Box 490, South Fallsburg, NY 12779