


Pinnacle BioLabs COVID-19 Novel Coronavirus Dual IgG/IgM Quick Reference Guide

Sample Collection

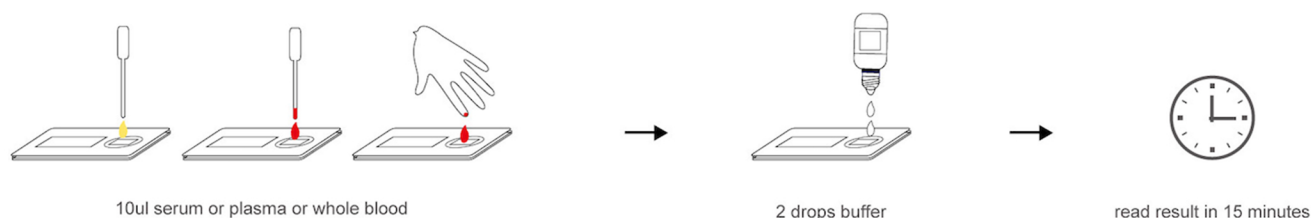

Results Interpretation



In the United States of America, this test is shipped and sold under Emergency Use Authorization by the US FDA. This test has not been reviewed by the FDA.

Negative results do not rule out SARS-CoV-2 infection, particularly in those who have been in contact with the virus. Follow-up testing with a molecular diagnostic should be considered to rule out infection in these individuals.

Results from antibody testing should not be used as the sole basis to diagnose or exclude SARS-CoV-2 infection or to inform infection status.

Positive results may be due to past or present infection with non-SARS-CoV-2 coronavirus strains, such as coronavirus HKU1, NL63, OC43, or 229E.

Not for the screening of donated blood

For Professional Use Only