



A*C*T*S® Non-Foaming

Item No. 34501

Autoclave Cleaning Treatment System

A*C*T*S® is a safe, efficient and quick revitalizing solution. A*C*T*S® will remove all rust, spotting, staining and corrosion, as well as scale, mineral and alkaline deposits on the inside walls of the autoclave and from stainless steel carts and trays. Simultaneously eliminates all noxious and unpleasant odors. A*C*T*S® is free-rinsing and leaves no residue. Non-aerosol pump is environmentally safe.

Features

- Removes all rust, stains and corrosion from walls of autoclaves and from stainless steel carts, and trays
- Also removes scale, alkaline, and mineral deposits
- Eliminates all noxious odors
- Restores original finish to autoclaves, carts, and trays
- Keeps carts and trays looking new
- Non-corrosive; will not harm quality stainless steel

Directions for Use

Autoclave Cleaning and Deodorization

1. Autoclave walls must be cool.
2. Spray A*C*T*S® generously on inside walls of autoclave.
3. Allow to remain wet for 15-30 minutes.
4. Brushing with non-metallic brush will speed the action.
Use of gloves is recommended.
5. Rinse walls with copious amounts of water.

Stainless Steel Carts and Trays

1. Spray A*C*T*S® generously on cart or tray surfaces.
2. Allow to remain wet for 15-30 minutes.
3. Brushing with non-metallic brush will speed the action.
Use of gloves is recommended.
4. Rinse cart or tray with copious amounts of water.

Repeat process as often as desired. Regular use of A*C*T*S® will prolong the life of your autoclave and stainless steel carts. May be used on quality stainless steel instruments.

Cautions: A*C*T*S® is a mild acid solution that may be harmful if swallowed or sprayed directly into face or eyes. Avoid skin contact. Keep out of reach of children.

**FOR USE ON STAINLESS STEEL ONLY.
DO NOT USE ON CHROME OR SILVER PLATED EQUIPMENT.**