

Endoscopy Care Products

AIR/WATER CLEANING ADAPTER

Item No. 345AWV

The device is intended, as part of the pre-cleaning process, to help clean the air/water channel of Olympus® GI endoscopes post procedure.



FEATURES

- Individually packaged
- Disposable, one-time use to prevent cross contamination
- Eliminates the need for manual cleaning and reprocessing reusable cleaning adapters
- Helps create consistent practices; Reduces the potential for errors
- For use with Olympus® GI Endoscopes (EXCLUDING Olympus® echo-endoscopes that utilize a balloon channel)

INSTRUCTIONS FOR USE (Read carefully prior to use.)

1. Prior to clinical use, familiarize yourself with the device and read all the Instructions for Use.
2. Remove the device from the package.
3. Turn off the airflow regulator on the light source after initial rinsing of scope (per facility guidelines).
4. Remove the air/water valve from the endoscope.
5. Insert the Air/Water Cleaning Adapter into the air/water cylinder on the control body of the endoscope.
6. Place the distal end of the scope into a basin of water.
7. On the light source, turn on the airflow regulator and set it to a maximum output of "high".
8. Depress the Air/Water Cleaning Adapter for 30 seconds. Note: This flushes the air/water channel with water.
9. Release the Air/Water Cleaning Adapter.
10. Keep the distal end of the scope in the basin of water.
11. Flush air for 10 seconds. Note: Air bubbles will be seen in the basin of water.
12. Turn off the air/flow regulator on the light source.
13. Remove distal end of endoscope from basin of water.
14. Remove the Air/Water Cleaning Adapter from endoscope after pre-cleaning step is completed.

PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

WARNINGS AND PRECAUTIONS

- These devices are intended to be used with Olympus® GI Endoscopes **EXCLUDING Olympus® echo-endoscopes that utilize a balloon channel**
- Do Not Use the Air/Water Cleaning Adapter during patient procedures. Continuous air flow could cause patient injury.
- Endoscopic cleaning procedures should only be performed by persons having adequate training with endoscope cleaning techniques.
- Do not attempt to reuse, reprocess, refurbish, re-manufacture or re-sterilize this device. This device is not intended to be reused, reprocessed, refurbished, re-manufactured, or re-sterilized. Performing such activities on this disposable medical device presents a safety risk to patients (i.e.. compromised device integrity, cross contamination, infection).

Olympus® is a registered trademark of Olympus® Corporation.
Rx Only (U.S.A.)

Federal law (U.S.A.) restricts this device to sale, distribution and use by or on the order of a physician.