

Delyme®

Item No. 34507

Lime and Mineral Scale Remover

*Specifically Designed for use in all Automatic Washers,
Washer Sterilizers, Cart Washers and Tunnel Washers.*

DELYME® removes all scale and alkaline mineral deposits from inside walls of automatic washers. Regular use of DELYME® prevents build-up caused by hard water, keeping high quality washers in top working condition, and aids in the cleaning process. DELYME® prevents, spotting and leaves washer walls sparkling clean.

Features

- Removes hard water scale, mineral scale, alkaline mineral deposits, rust, stains and organic waste from the inside walls of automatic washers
- Regular use will prevent the build up of lime and mineral scale
- Helps maintain the condition of your washer disinfectant equipment
- Improves the cleaning process

Directions for Use

For Soaking and Manual Cleaning

Add 2 ounces (60ml) to 4 ounces (120ml) of Delyme® to 1 U.S. gallon (3.8 liters) of hot water and soak for 10 minutes. Rinse thoroughly with hot water.

Recommended use: Weekly

Cautions: Avoid prolonged contact with skin. Do not swallow. Keep away from children.



For Automatic Washing Machine

Light Scale and Daily Use:

Use 1/2 ounce (15ml) of DELYME® per U.S. gallon (3.8 liters) of tank capacity. Circulate on Acid Rinse cycle for 10 minutes. Dispense where possible via an automatic dispensing system to prevent contact with the product

Heavy Scale:

Use 8 ounces (240ml) of DELYME® per U.S. gallon (3.8 liters) of tank capacity. Circulate on Acid Rinse cycle for 10 minutes. Then use a hot water rinse for at least 10 minutes. Dispense where possible via an automatic dispensing system to prevent contact with the product.