

## Ruhof Dry Sponge

Item No. 345DRYSPG

### *Absorbent Contoured Sponge for Pre-Cleaning Flexible and Rigid Scopes & Endoscope Tip Protector*

Ruhof's super absorbent Dry Sponges are designed to remove gross contaminants from the outside sheath of all scopes and tubular instruments during the pre-cleaning and manual cleaning process. Ruhof Dry Sponges can also be used to fit securely around the distal tip of a scope to protect delicate lenses and fiber-optics from damage during transportation. In addition, Ruhof Dry Sponges protect the distal tip of Tee Probes along with other delicate components of various medical/surgical instruments. Ruhof Dry Sponges are individually wrapped for one time use to help reduce the risk of cross contamination.

#### Features

- Super absorbent contoured sponge for all flexible and rigid scopes
- Perfect for pre-cleaning and manual cleaning
- Protects delicate scopes during transportation
- Lint and latex free
- Individually wrapped for single use to help decrease cross contamination
- Will not harm delicate endoscopic parts, such as lenses, lighting fixtures or air/water nozzles.

#### Directions for Use

##### Scope Cleaning:

1. Remove sponge from plastic bag.
2. Place sponge around the insertion tube of scope or tubular instrument.
3. Run sponge down length of scope or instrument to remove gross soil.
4. Rinse sponge and repeat until surface is clean.
5. Discard sponge.

##### Tip Protection:

1. Place the Ruhof Dry Sponge around the distal end of the scope or medical/surgical instrument.
2. Remove the Ruhof Dry Sponge prior to use of scope or medical/surgical instrument.
3. Dispose used sponge. This product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

**WARNING:** Never use Ruhof Dry Sponge on a disinfected or sterilized endoscope and/or medical/surgical instrument.

