

ScopeValet™ Dual End Valve Brush

Item No. 345VB

ScopeValet™ Dual End Valve Brush is used to safely remove debris from the valve openings and dials of the control head of flexible scopes.

Features

- Variable brush head sizes fit all buttons and valves
- Extended length and ergonomic grip handle facilitates brushing of biopsy port
- Optimized bristle density and stiffness ensure gentle and effective scrubbing action
- Disposable, one-time use

Instructions for Use

1. Remove valve(s) from the control head of the scope.
2. Using the Ruhof enzymatic detergent of choice, use the brush to gently cleanse the valve openings. Avoid twisting or torquing the brush handle.
3. Using the Ruhof enzymatic detergent of choice, clean the control head and control head dials.

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

