

Endozime® AW

Item No. 34511

Dual Enzymatic Detergent

Specifically Designed for Washers, Washer Sterilizers, Ultrasonics and Manual Cleaning.

Endozime® AW is a low-sudsing, dual enzymatic cleaner that removes blood, protein, mucous, vomit and fecal matter. This formulation of Protease and Amylase enzymes, with buffers and non-ionic detergents, is safe for all scopes and instruments and extends usable lifetime. Endozime® AW will leave scopes and instruments free of any unpleasant odors or baked-on blood.



Features

- Dual enzymatic cleaner Protease and Amylase
- Low-sudsing Designed for use in all washer, washer sterilizers, ultlrasonics and for manual cleaning.
- Safe for use on all surgical instruments and scopes
- Neutral pH, non-abrasive, free rinsing and 100% bio-degradable
- Will not harm any metals, plastic, rubber, glass or corrugated tubing
- Safe for all instruments and scopes Extends usable lifetime

Directions for Use

Surgical Instruments and Scopes

- 1. Add one (1) ounce of Endozime® AW to one (1) gallon (8ml/liter) of water.
- 2. Submerge instruments and scopes to be cleaned. For scopes: suction or flush through channels before soaking.
- 3. Soak for 2 minutes to remove all organic matter.
- 4. Rinse thoroughly with tap, distilled or sterile water. Works in all temperatures.

Discard used solution.

Cautions: Avoid prolonged contact with skin. Do not swallow. Keep away from children.

Automatic Cleaning Devices:

- 1. Pour directly into detergent reservoir full strength, or
- 2. Dilute 1 ounce per 1 gallon (8ml/liter) of water.

