

Endozime®

Item No. 34509

Dual Enzymatic Cleaner

Highly recommended for all scopes (flexible/rigid) and biopsy forceps.

Leaves instruments free of blood and protein in 2 minutes.

Endozime® is a dual enzymatic cleaner that removes blood, protein, mucus, vomit and fecal matter. This formulation of Protease and Amylase enzymes, with buffers and non-ionic detergents, is perfect for all scopes and instruments and extends usable lifetime. Endozime® will solve any problem with clogged channels, sticky forceps or clouded lenses and will leave both scopes and instruments free of any unpleasant odors or baked-on blood.

ENDOZINE COUNT PAIN A request and presents formatter than the part of the country of the count

Features

- Dual enzymatic cleaner Protease and Amylase
- Designed for universal applications: For use in all endoscope washers, washer disinfectors, ultrasonics and for manual cleaning
- Safe for use on all instruments and scopes
- Neutral pH, non-abrasive, free rinsing and biodegradable
- Will not harm any metals, plastics, rubber or corrugated tubing

Directions for Use

Surgical Instruments and Scopes

- Add one (1) ounce of Endozime® to one (1) gallon (8ml/liter) of water.
- Submerge instruments and scopes to be cleaned.For scopes: suction or flush through channels before soaking.
- 3. Soak for 2 minutes to remove all organic soils.
- 4. Rinse thoroughly with tap, distilled or sterile water.

Automatic Cleaning Devices:

- 1. Pour directly into detergent reservoir full strength, or
- 2. Dilute 1 ounce per 1 gallon (8ml/liter) of water.

Discard used solution. Works in all temperatures.

Cautions: Avoid prolonged contact with skin. Do not swallow. Keep away from children.

