

ScopeValet™ Endo Brush

Item No. 345EBS - 5MM Brush (Channel size of 2mm-4mm)

Item No. 345EBL - 6MM Brush (Channel size of 4mm-5mm)

STERILE

Item No. 345EBSST - 5MM Brush (Channel size of 2mm-4mm)

Item No. 345EBLST - 6MM Brush (Channel size of 4mm-5mm)

ScopeValet™ ENDO BRUSH is a non-sterile (also available as a sterilized product for both testing and cleaning purposes), one-time use brush on a flexible wand that allows for cleaning the inside channel of an Endoscope.

Features

- Disposable, one-time use
- Available in a 5mm size (for scopes and instruments with a channel size of 2mm-4mm) and a 6mm brush size (for scopes and instruments with a channel size of 4mm-5mm).
- Wand length is 230cm; Catheter diameter is 1.8mm
- Premium catheter stiffness provides smooth scope passage
- Double-end provides pull through design, ensuring maximum channel coverage
- Unique rounded tip helps to prevent damage to the inside channels of scopes

Instructions for Use

Intended Use:


The ScopeValet™ ENDO BRUSH is designed to provide mechanical scrubbing of channels to safely remove debris from flexible endoscopes.

Warnings & Precautions:

- The cleaning brush should never be used as a cytology brush.
- Original Equipment Manufacturer's guidelines regarding the care and cleaning of individual endoscope channels must be reviewed prior to the use of any cleaning brush.
- Dispose of the cleaning brush after initial use. The mechanical attachment of the brush(es) to the plastic sheath cannot be guaranteed to be reliable or secure following initial use.
- If sharp bends in the plastic sheath or wire cable are observed during insertion of the brush, the brush should be disposed to avoid causing damage to the endoscope's operating channels.
- **NEVER TRANSFER** a used cleaning brush from one endoscope to another endoscope, since cross contamination could occur and result in patient complications. Disposable cleaning brushes are designed to safely remove debris, some of which can be retained in the bristle structure.
- Ruhof Healthcare did not design this device to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and /or sterilize or maintain the structural integrity of the device to ensure patient and /or user safety.
- These products are intended for single patient use only. Any institution, practitioner, or third party who reprocesses, refurbishes, remanufactures, resterilizes, and /or reuses these disposable devices must bear full responsibility for their safety and effectiveness.

Contraindication:

If the channel(s) are known to be occluded or resistance is met in attempting to pass the brush through the endoscope, **DO NOT FORCE** the cleaning brush through the endoscope, as this may result in damage to the channel.

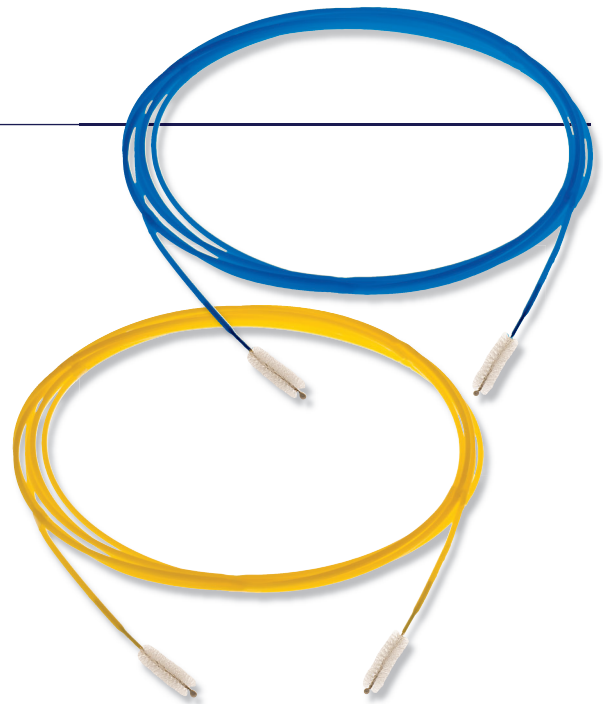

Endo Brush's unique rounded tip helps prevent damage to inside channels of scopes.