

ScopeValet™

Endoscopy Care Products

ScopeValet™ EndoTote™

Item No. 345TBC

The ScopeValet™ EndoTote™ is a safe and easy way to transport an endoscope from the procedure room to the reprocessing room after the endoscopy. Simply fold the sides of EndoTote™ and gather the reinforced handles to convert from a surface pad to an endoscope transport container. The disposable bag's Velcro closure system and tamper proof "zip-tie" tag ensures staff safety by securely keeping the soiled scope enclosed during transfer.

Features

- Easy set up before procedures providing full surface coverage for large and small cart surfaces; 43" x 34 9/16" (109 cm x 88 cm) bag size with 16 9/16" (42 cm) wide inner pad
- A strong leak-proof liner and absorbent inner pad which keep bio-hazard fluid contained and protects your endoscope from being damaged during transportation
- Ample bag size accommodates all endoscopes and keeps them safe from being impaired when the flexible shafts are coiled too tightly
- A red tamper-proof, disposable tag easily secures bag tightly; it features a unique serial number for traceability, simplifying transportation and assisting with cleaning room organization
- Conforms to all endoscopic disinfection guidelines including the regulations that endoscopes be protected from the environment, be fully traceable and be kept moist after pre-cleaning endoscope
- Disposable, one time use

Instructions for Use


1. Remove from packaging and spread the ScopeValet™ EndoTote™ across the procedure cart surface.
2. Place the soiled scope in the center of the EndoTote™ pad.
3. Fold the sides of the EndoTote™ (without handles) inward towards the scope and secure the Velcro fastening system.
4. Bring the reinforced handles together over the top of the endoscope and hold in one hand.
5. Place the red "zip-tie" tag through the reinforced handles and lock into position.
6. Simply break the red tag open to remove the used scope for reprocessing.

Contraindication:

This product is for one time use only. Its reuse may present risks to the safety of individuals, due to possible uncontrolled contamination.



Tamper-proof tag secures bag tightly and features unique serial number for traceability.

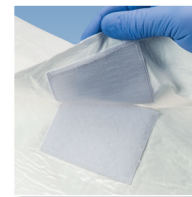

Velcro closure system keeps soiled scope enclosed during transfer.

