

ForeverPrep™

Item No. 345FP

Quick & Easy Application of Prepzyme® Forever Wet for the Pretreatment of Soiled Instruments

The Ruhof ForeverPrep™ Pretreatment Sprayer is an automated application system for Prepzyme® Forever Wet Humectant Pre-Cleaner which makes the job of coating surgical instruments fast and easy.

Prepzyme® Forever Wet is a neutral pH, multi-tiered enzymatic humectant spray which promotes the long lasting retention of moisture on soiled instruments and scopes thus helping to prevent the adhesion of bio-burden. This unique formulation gently coats instruments to maintain moisture making it an ideal pre-cleaner for soiled instruments during transport or when left for an extended period of time.

FEATURES

- Applies Prepzyme® Forever Wet which keeps instruments moist for up to **72HOURS**, helping to prevent the adhesion of bio-burden
- Quickly coats instrument trays in less than 8 seconds
- Eliminates manual pumping with trigger sprayers
- Ready to operate out of the box
- Safe 24 volt operation
- Can be wall mounted or placed on the counter; Convenient carry handle for mobile application
- System houses 1 gallon bottle of Forever Wet inside

INSTRUCTIONS FOR USE (IFU)

1. Check chemical level in bottle.
2. Turn power on.
3. Hold applicator wand 6" – 8" over instruments and squeeze lever.
4. Coat instruments thoroughly, moving the applicator side to side.

CAUTION:

Personal protective equipment should be worn when operating ForeverPrep™. Always point the ForeverPrep™ applicator wand away from you.



For use with Prepzyme® Forever Wet
Item No. 34577



Wide spray pattern to cover
all instruments evenly