

## ISO-GON®

Item No. 34537

**Versatile Cleaner, Descaler, Decontaminant, and  
Radioactive Contaminant Remover**

**Specifically designed for use in washer-disinfectors,  
tunnel washers and cart washers.**



ISO-GON® contains powerful and unique sequestering, chelating, and emulsifying agents, assuring complete cleanliness and free rinsing of all surgical carts, trays, washers, plastics, aluminum, glassware and rubber tubing. ISO-GON® is excellent for eliminating scale and mineral deposits caused by hard water.

### Features

- One-step cleaning and scale prevention
- ISO-GON® is mildly acidic (pH 5-5.5) yet does not require a neutralizer.
- Bio-degradable
- Extremely fast wetting, while maintaining high detergency and cleaning performance in hard, soft, cold and hot water
- Instantly and constantly active (sequestering)
- Free Rinsing, ISO-GON® leaves no residue or pyrogens

### Directions for Use

- Dose Ruhof's ISO-GON® via the washer's auto dispensing pump or pour directly into the reservoir. Works in all water temperatures.
- Recommended dilution: 1-2 oz. per gallon of water (8-16ml per liter)  
(On lightly soiled carts and trays ISO-GON® can be diluted at 1/2 oz. per gallon of water (4ml per liter).

Cautions: Product contains citric acid; Avoid prolonged contact with skin. Do not swallow. Keep away from children. Do not store in carbon steel containers. Do not use concentrated product on soft metals and carbon steel.