

Liquizime[®]

Item No. 34547

Multi-Tiered Enzymatic, Alkaline Detergent

Specifically designed for use in all washer-disinfectors, tunnel washers, cart washers and for manual cleaning.

Liquizime® is a premium detergent that utilizes the power of enzymes in an alkaline solution. With its Multi-tiered formulation of enzymes, Liquizime® removes all blood, fat, carbohydrates, starches and protein, while its powerful alkaline solution eliminates scale and mineral deposits left from hard water. Liquizime®'s unique sequestering and emulsifying ability assures complete cleanliness and free rinsing of surgical instruments and scopes as well as all carts, trays and inside washer walls.



Features

- One-step cleaning and scale prevention
- Proprietary synergistic mixture of enzymes and alkaline
- Protease, Lipase, Amylase, Carbohydrase and proprietary enzymes
- Excellent for hard water problems, eliminating mineral deposits from stainless steel instruments, optics, common plastics as well as materials used in anesthetic utensils
- Liquizime®'s mild alkaline with approximate pH 9, yet does not require a neutralizer.
- Extremely fast wetting and odorless, while maintaining high detergency and cleaning performance in hard, soft, cold and hot water.
- Free Rinsing, Liquizime® leaves no residue or pyrogens
- Non-Foaming Non-Sudsing
- Bio-degradable
- · Will not harm any metals, plastics, rubber or corrugated tubing

Directions for Use

Liquizime® can be diluted as low as 1/4 oz. (2ml/liter) per (1) gallon of water up to two (2) oz. per one (1) gallon of water, but for optimal results, the following is recommended.

Manual Cleaning

- For lightly soiled scopes and instruments dilute Liquizime[®] at a 1/2 ounce per one (1) gallon (4ml/liter of water.
 For heavily soiled scopes and instruments dilute Liquizime[®] at one (1) ounce per one (1) gallon (8ml/liter) of water.
- 2. Submerge instruments and scopes to be cleaned.
 For scopes: suction or flush through channels before soaking.
- 3. Soak for two (2) minutes to remove all organic soils.
- 4. Rinse thoroughly with tap, distilled or sterile water.

Discard used solution. Works in all water temperatures.

Cautions: Avoid prolonged contact with skin. Do not swallow. Keep away from children.

CE-marks: Liquizime[®] conforms to the requirements of the legislation for medical devices and the European directive 93/42/EC. Hazard warnings and safely advice: Liquizime[®] needs not to be classified according to the Dangerous Preparation Guideline 99/45/EC.

Automatic Cleaning Devices:

- 1. Pour directly into detergent reservoir full strength or
- 2. Calibrate value at 1/2 ounce to 1 ounce per one (1) gallon (4ml/liter to 8ml/liter) of water.



