

Premixslip®

Instrument Lubricant and Rust Inhibitor

The Only Instrument Lubricant and Rust Inhibitor Clinically Tested to be Thoroughly Steam Penetrable and Steam Sterilizable

Premixslip® is a non-sticky, non-toxic, silicone free and water-soluble mineral oil lubricant that can withstand the high temperatures of autoclaving without becoming gummy. Forming a protective barrier on surgical instruments, Premixslip® prevents rusting, staining, spotting and restores articulation to sticky box-locks and joints. Used routinely as part of the Ruhof Instrument Care System, Premixslip® will greatly reduce repair and replacement costs.

Premixslip® provides permanent lubrication and protection for all surgical instruments.

Features

- High-grade mineral oil lubricant and rust inhibitor.
- Prevents rust, spots, stains and corrosion thus prolonging the life of the instrument.
- Prevents damage due to the misuse of stiff or grinding instruments thus reducing repair and replacement costs.
- Steam penetrable, steam sterilizable and ETO sterilizable.
- Non-oily, non-sticky, non-toxic and silicone free.

Directions for Use

Use as is – **Do Not Dilute** – Shake gently prior to use.

For Spray Applications:

Item No. 34552-18

1. Restrict the use of the lubricant to the area of the instrument requiring lubrication using the sprayer or dropper.
2. Allow the Premixslip® to penetrate the joint for about 30 seconds. Test the action of the lubricated joint to ensure penetration.
3. For exceptionally stiff instruments re-apply and actively open and shut the instrument, until loosened.
4. If the lubricant dislodges dried contaminants from the joint, the instrument will require re-cleaning.
5. Seized instruments may require treatment by immersion in a bath of Premixslip®.

For Immersion Applications:

Item No. 34552-27

1. Dip previously cleaned instruments into Premixslip® solution.
2. Remove instruments and let drain about 30 seconds.
3. For exceptionally stiff instruments:
 - Place single opened instrument in solution for 5 minutes.
 - Actively open and shut clamp or scissors in solution for 30 seconds.
 - Proceed as directed.
4. Drained instruments are ready for preparation and terminal sterilization. Do not rinse, wipe or dry.
5. Always cover Premixslip® solution when not in use.

Cautions: Avoid prolonged contact with skin. Do not swallow. Keep away from children.



Item No.
34552-27



Item No.
34552-18