

## Prepzyme® XF (eXtreme Foam)

Item No. 34561

### Multi-Tiered Enzymatic Foam Spray with Applicator Tip

**Extreme foaming action prevents bio-burden from adhering to the surface of all surgical instruments and scopes.**

Prepzyme® XF provides a thick, long lasting foam that keeps soiled instruments moist and begins breaking down all bio-burden - blood, fat, carbohydrates, starches and protein - prior to decontamination. Prepzyme® XF makes cleaning easier and helps speed up turn-around time by preventing bio-burden from drying on the surface of instruments. The Applicator Spray Tip is designed to reach into cannulas and lumens providing a deeper more thorough cleaning. Prepzyme® XF is a great alternative to water based solutions that cause rust after prolonged soaking of instruments.

Perfect for use in Operating Rooms, Endoscopy Suites, Outpatient Surgery, Dental and other departments where instrument and scope cleaning is not immediately available. No more messy spills. Stops splashing during transportation. Instruments arrive ready for decontamination. Prepzyme® XF is clinically tested to be free rinsing and 100% biodegradable.

### Features

- Longer lasting foam allows for better penetration of bio-burden
- Thicker foam completely covers instruments
- Keeps soiled instruments moist
- Eliminates dried on bio-burden
- No messy spills and splashing
- Helps reduce cleaning time
- Easy and safe to use
- Neutral ph
- Free rinsing
- Will not harm any metals, plastics, rubber or corrugated tubing

### Directions for Use

1. Spray directly on soiled instruments and scopes immediately after use. Use applicator spray tip for deeper penetration of cannulated instruments and scopes.
2. Allow instruments to soak in foam until ready for cleaning. Apply more as needed to keep moist.
3. Rinse instruments before cleaning.
4. Discard used solution.

Cautions: Avoid prolonged contact with skin. Do not swallow. Keep away from children.



14.5oz. can  
Item No. 34561-48

3oz. can  
Item No. 34561-49

