

Prepzyme®

Item No. 34559

Multi-Tiered Enzymatic Foam Spray

Helps to Prevent The Adhesion Of Bio-Burden To The Surfaces of All Surgical Instruments and Scopes

Prepzyme® is a safe and ideal method for keeping soiled instruments and scopes moist, helping to prevent bio-burden from adhering to the surface prior to decontamination. Prepzyme® covers the instruments and begins breaking down blood, fat, protein and carbohydrates before they arrive in the central service/decontamination room.

Perfect for use in Operating Rooms, Endoscopy Suites, Outpatient Surgery, Dental and other departments before instruments are transported to decontamination. Prepzyme® prevents splashing during transportation, meaning no more messy spills. By eliminating dried on bio-burden, it helps reduce cleaning time. Prepzyme® is clinically tested to be free rinsing and 100% biodegradable.

Features

- Instantly active - Constantly active
- Begins breaking down blood, fat, protein and carbohydrates
- No more messy spills or splashing
- Helps to prevent dried on bio-burden
- Easy and safe to use
- Safe for all instruments and scopes - Extends usable lifetime
- Neutral pH (non-abrasive)
- 100% Biodegradable
- Free rinsing
- Will not harm any metals, plastics, rubber or corrugated tubing

Directions for Use

1. Spray directly on soiled instruments and scopes immediately after use.
2. Allow foam to stay on instruments and scopes until ready for cleaning.
Apply more Prepzyme® as needed to keep moist.
3. Rinse instruments thoroughly before cleaning.

Discard used solution.

Cautions: Avoid prolonged contact with skin. Do not swallow. Keep away from children.

